

Tissue-Tek Prisma[®] Plus

Automated Slide Stainer

Operating Manual

Part Number	M01-019E-04
Revision Letter	B
Revised	2018-02-01



continuous innovation for pathology



Introduction

The automated slide stainer DRS series "Tissue-Tek Prisma® *Plus* (hereinafter referred to as the "instrument")" is an instrument that automatically stains paraffin strips, frozen strips, cells and other tissues pasted on slides.

Read this Operating Manual carefully so that you can fully utilize the performance of the instrument based on correct procedures. Since the instrument uses organic solvents, exercise due caution when operating the instrument.



Manufactured for:

Sakura Finetek U.S.A., Inc. Torrance, CA 90501, USA
Sakura Finetek Europe B.V., 2408 AV Alphen aan den Rijn, NL
Sakura Finetek Japan Co., Ltd. Tokyo, Japan 103-0023

Manufactured by:

Sakura Seiki Co., Ltd. Nagano, Japan

Table of Contents

Safety Precautions	(e)
Explanation of Terms.....	(p)

Chapter 1 Basics of the System

Installation Method.....	1-1
Installing the Instrument	1-1
Flow of Installation	1-1
Required Tools	1-1
Installation Environment	1-1
Checking the Installation Conditions	1-2
Checking the Work Area.....	1-3
Transportation	1-3
Unpacking Procedure	1-4
Unpacking the System.....	1-4
Removing the Securing Members Inside the Instrument	1-6
Switching the Power-supply Voltage Setting.....	1-7
Checking the Accessories	1-7
Checking the Installation (Securing the Instrument)	1-9
Installing the parts around the instrument	1-9
Installation of Accessories	1-14
Turning on the Power	1-16
Specification.....	1-17
Name of Each Part.....	1-20
Standard Accessories/Options.....	1-22
Basics.....	1-24
Basics on This Instrument	1-24
Instrument Overview	1-24
Stain Process Modes.....	1-24
User Management by Password	1-24
Logon/Logoff Function	1-25
Safety Mechanism of the Instrument	1-25
How to Open and Close the Cover/Door.....	1-25
Handling the Touch-panel Display	1-26
Various Gateways	1-27
Relationship of Solution Configurations and Station Numbers.....	1-29
Explanation of Menu Screens	1-32
Stain Process Menu	1-32
Explanation of Icons in the Stain Process Menu	1-33
Utility Menu.....	1-34

Explanation of Icons in the Utility Menu	1-34
Edit Menu	1-35
Explanation of Icons in the Edit Menu	1-35
System Setup.....	1-36
System Setup	1-36
Setting Date and Time.....	1-38
Entering Year	1-38
Entering Month	1-38
Entering Day	1-38
Entering Hours and Minutes	1-38
am/pm.....	1-38
Setting User IDs, Passwords and Accessible Functions.....	1-39
Setting an ID	1-39
Setting a Password.....	1-39
Setting Accessible Functions	1-40
Stain Process.....	1-40
Utility	1-40
Edit.....	1-40
Fume Filter Management	1-41
Whether or Not to Use Fume Filter Management.....	1-41
Setting a Time Limit	1-41
Resetting the Actual Time	1-41
Adjusting the Water Flow.....	1-42
Export Data	1-43
Names of Files Exported to a CF Card.....	1-44

Chapter 2 Operating Procedure

Operating Procedure.....	2-1
Flow of Stain Process Operation.....	2-1
Flow of Stain Process Operation	2-1
Preparation for Stain Process	2-2
Opening the Water Supply.....	2-2
Logging On to the Instrument	2-2
Selecting and Saving a Solution Configuration and Setting Solutions.....	2-3
Setting Slides into Baskets.....	2-4
Start of Stain Process.....	2-5
Operations Permitted during Stain Process	2-7
Adding a Basket.....	2-7
Setting Priority Start.....	2-8
Checking the End Station Status	2-8
Checking the Staining Status	2-9
Menu Operations Permitted during Staining Process	2-10
Pausing and Aborting Stain Process	2-11
Removing a Basket during Stain Process	2-12

End of Stain Process.....	2-12
Removing Stained Baskets.....	2-12
End of Operation	2-12
How to Start Stain Process by Specifying the Start and End Steps	2-13
Setting a Solution Configuration	2-15
Creating a New Solution Configuration	2-15
Saving a Staining Program Name	2-15
Setting Up a Solution Configuration	2-16
Setting an Edit Solution Configuration.....	2-17
Mix Parameter Setup	2-20
Enhanced Wash Parameter Setup	2-20
Basket Lifting Speed and Water Wash Cycle Time Parameter Setup.....	2-21
Temperature Settings.....	2-21
Editing a Solution Configuration	2-22
Copying a Solution Configuration.....	2-22
Editing a Solution Name.....	2-23
Creating and Editing a Solution Name	2-23
Copying a Solution Name	2-24
Setting a Staining Program	2-25
Creating a New Staining Program.....	2-25
Saving a Staining Program Name	2-25
Creating a Staining Program	2-26
Editing a Staining Program.....	2-31
Copying a Staining Program	2-32
Checking a Staining Program.....	2-33
How to Use the Bar Code Reader, and It's Function (Options)	2-34
Overview.....	2-34
Applicable Solutions	2-34
How to Use Sakura Staining Solutions	2-34
Editing a Solution Configuration	2-34
How to Enter Sakura Solution Bar Code Data.....	2-35
How to Use Sakura Staining Kit	2-36
Editing a Solution Configuration	2-36
How to Enter Kit Component Solution Bar Code Data	2-37
Resetting the Solution Usage Information.....	2-39
Outputting Process Report	2-40
Solution Usage Information Report	2-40
Process Report	2-41
Solution Configuration Report.....	2-42
Barcode History Report	2-43
Deleting a Solution Configuration/Staining Program/Solution Name/Process Report	2-44
Deleting a Solution Configuration	2-44
Deleting a Staining Program.....	2-45
Deleting a Solution Name.....	2-45
Deleting a Process Report.....	2-45

Action to Be Taken upon Power Outage	2-46
Power Recovery Operation	2-46
Action to Be Taken upon Error	2-47
Error Handling	2-47
Errors with a Number	2-47
Other Errors	2-47

Chapter 3 Maintenance and Inspection





Daily Inspection and Cleaning	3-1
Inspection and Cleaning Methods	3-1
Cleaning the Solution Reservoirs and Baskets	3-1
Cleaning the Drying Stations	3-1
Cleaning the Heating Stations	3-1
Replacing Activated Carbon Filters	3-2
Cleaning the Arm Holder and Basket Adapter	3-2
Cleaning the Water Supply Strainer	3-3
Cleaning the Cabinet	3-3
Cleaning the Wash Station Nozzle	3-4
Cleaning the Drain Hose	3-4

Chapter 4 Troubleshooting


Troubleshooting	4-1
Error Codes Displayed during System Operation	4-1
Instrument Conditions and Recovery Actions	4-7


Safety Precautions

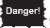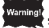
Designation of "Instrument Control Manager"

- Operation of the instrument requires expert knowledge of the target application, method of use, and so on. So that the instrument can be used correctly and safely, designate a "Instrument Control Manager."
- When the instrument is delivered, the Instrument Control Manager should receive explanation on the handling of the instrument directly from our sales representative.
- * Before using the instrument, read "Safety Precautions" to ensure the correct use of the instrument.
- * The cautionary instructions provided herein are intended to ensure that the instrument will be used safely in a manner preventing operator from injury and property damage.
- * In this manual, instructions pertaining to different levels of potential hazards are classified as and indicated under , ,  and .

The definition of each hazard is defined below.

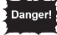
 **Danger!** pertains to a potential hazard where failure to observe the specified instruction may result in death or serious injury of the operator or other person.

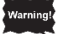
 **Warning!** pertains to a potential hazard where failure to observe the specified instruction may result in a serious injury of the operator or other person.


 **Caution!** pertains to a potential hazard where failure to observe the specified instruction may result in damage to this instrument or other property, or affect the process results.


 **Note** pertains to an item to pay attention to or other useful information.

- * The symbols used on the labels attached to the instrument are explained.
The labels bearing one of the following symbols provide particularly important information you must know in order to ensure safety of the operator, improve work efficiency and protect the instrument from damage. Be sure to check these labels and understand the specified instructions before commencing your work.

 A label bearing this symbol specifies an action that must be taken. Always follow the instructions.

 A label bearing this symbol specifies a prohibited item. Always follow the instructions.

 A label bearing this symbol specifies a precaution. Wrong handling against the instruction may potentially expose the operator to danger or result in damage to the instrument. Always follow the instructions.

 A label bearing this symbol is provided near a high-temperature area. Exercise caution against burns caused by high temperatures.

Safety Precautions



WARNING

- **Read the unpacking instructions carefully before unpacking the instrument.**
When removing the instrument from the pallet, two or more persons should work together because the instrument weighs 150 kg. When removing the instrument from the pallet this way, indicate so in the shipping document and inspect the cargo for damage. As a rule, the instrument is unpacked/installed by Sakura Finetek or our local representative.
- **Those who have experience operating a Sakura Finetek product or products other than the instrument must also read this manual. Also, those who will be operating the instrument should receive training first.**
Sakura Finetek will assume no responsibility for any accident that may result from an accident caused by operating the instrument without reading this manual first. If the instrument is operated in a manner not specified in this document, the protective functions of the equipment may be lost.
- **Do not apply unnecessary external forces on the instrument and accessories.**
The instrument may malfunction or cause accidents.
- **Avoid connecting the instrument to a transformer not manufactured by Sakura Finetek.**
When moving the instrument to a region where different voltage is used, contact the Sakura Finetek Technical Support representative or local representative.
- **Safety mechanisms installed in the instrument only represent the minimum measures taken to prevent accidents. The responsibility for safe operation of the instrument lies primarily with the owner of the instrument, and persons in charge who operate the instrument.**
- **Do not use the instrument, if damaged, even when operating it is possible.**
Injury or electric shock may result.
- **Should the instrument malfunction, turn off the power and disconnect the power plug from the power outlet. If processing is still in progress, immediately protect the tissues.**
Should malfunction occur, turn off the power and contact the Sakura Finetek Technical Support representative or local representative. Disconnecting the power plug also cuts off the main power supply. When installing the instrument, select a place where the power outlet is not concealed and the power plug can be disconnected any time in case of emergency.
- **Do not disassemble or modify the instrument.**
The instrument may malfunction or cause accident, thus creating a dangerous situation.
- **Only the specified solutions can be used with this instrument.**
If any other solution is used, processing may not be performed properly and fire or failure may result.
- **Do not wet the instrument.**
If areas other than those instructed by "Daily Inspection and Care" get wet, current leak may occur and cause fire or electric shock. If wetted, the board, power switch, CF card and CF card slot, connection parts of various terminals are located on the right side of the instrument may cause instrument failure or data damage.
If the terminal connection part LAN or bar code terminal is not used, put a protective cap over it.

Safety Precautions



WARNING

- **A wrong operation may stop or interrupt the tissue processing. Be careful not to let unauthorized personnel manipulate or tamper with the instrument while operating.**

A wrong operation may stop or interrupt the tissue processing. (If the power is turned off by unauthorized personnel, the stain process may be interrupted and the staining problems may occur. Arrange the work environment so that only the operators with expert knowledge can use the instrument. (Also consider using external alarms, etc., just in case.)
- **Do not bring fire sources close to the instrument.**

The instrument uses organic solvents that may ignite upon contact with fire.
- **Do not let foreign matter go inside the instrument.**

If water, metal, paper or other foreign matter enters the air inlets (to the fan, etc.), fire, electric shock, injury of the user, instrument failure, etc., may occur. If foreign matter entered the instrument, immediately stop the instrument and contact the Sakura Finetek Technical Support representative or local representative.
- **Do not operate the instrument while wearing metal accessories.**

Conductors (metal pen, accessories, etc.) worn on your body may contact the instrument and cause electric shock.
- **Do not block the intake/exhaust ports and air inlet around the instrument.**

Do not block the intake/exhaust ports (air inlets) with a wall or object or use the instrument in a poorly ventilated place or with the dustproof covers still attached, because heat will be trapped in the instrument and cause the capacity of the instrument to drop, while fire, accident, failure, etc., may also occur due to abnormal overheating.
- **Periodically check the power cord and cables and power outlet for damage and accumulation of dust.**

Foreign particle and dust accumulating on the power outlet may cause fire. Clean the power outlet periodically.
- **Do not let tissue with flammable solvent attached go inside the drying station.**

Deposited flammable solvent may be heated by the drying station and vaporize, causing fire or explosion.
- **Do not install the instrument in a place used as a living area.**

If a common area is used, an exhaust system is required. Provide electrostatic discharge protection in the installation location.
- **Install the instrument indoors. Do not place the instrument in a location exposed to direct sunlight, snow or rain.**

This product is specified for indoor use, so do not use it outdoors. Ultraviolet rays and high temperature may damage the instrument. Never use the instrument while exposing it to snow or rain because it may cause electric shock or short-circuit.

Safety Precautions



WARNING

- **Unplug the power cord from the power outlet and store it while the instrument is not in use.**
For safety, if the instrument will not be used for an extended period of time, clean each part and store the instrument in a place free from humidity and from dust and foreign matter.
Using the instrument in a humid place, place exposed to oil smoke or steam, or dusty place, may cause its parts to deteriorate, short-circuit, fire, etc. The installation humidity range is 30 to 85% (non-condensing). If you wish to install the instrument in a special place, consult the Sakura Finetek Technical Support representative or local representative.
- **If dews are formed on the instrument, do not turn on the power.**
If the power is turned on while dews are formed on the instrument, current leak may occur and cause fire or electric shock. Do not connect the instrument to a power-supply facility until dews have evaporated and the instrument has become completely dry.
- **Do not operate the power plug and switches with a wet hand. When connecting/ removing the power plug, do so by holding the plug.**
Failure to observe these instructions may result in electric shock.
- **Be sure to use the supplied power cord. Also, be sure to use a correct power cord that is suitable for the current capacity of region where the instrument is used.**
If an inappropriate power cord or voltage is used, instrument malfunction due to short-circuit, fire or voltage drop, unexpected accidents or injuries due to electric shock, etc., can occur. The preset voltage cannot be changed by the user.
- **For the power cord, a standard cord specified by local law may have to be used.**
If a power cord not specified by the PSE standard, UL/CSA standard or EC standard is required, contact the Sakura Finetek Technical Support representative or local representative.
- **Do not forcibly bend, pull, twist or bundle the power cord and cables or otherwise damage the power cord and cables.**
Fire or electric shock may occur.
- **Do not overload the outlet and do not use an extension cord.**
Doing so may stop the operation of the instrument due to voltage drop, or fire can occur due to heat generation.
- **This instrument has been designed and tested according to CISPR11 Class A. When the instrument is used in a home, it may cause electromagnetic interference, in which case measures to reduce interference may have to be taken.**
- **Radiant waves may prevent this instrument from operating properly, so do not use the instrument near a radiation source of strong electromagnetic waves.**
- **Use the instrument under the conditions (such as voltage within 10% of the rated voltage) stated in the Specification.**

Safety Precautions



CAUTION

- **Do not pinch the drain hose or allow it to slacken.**
Water will not be drained properly and overflow from the instrument.
- **Set the solution station (tray) in the correct location.**
If not, the tissues may not be soaked in the solution and staining problems may occur.
- **Store the slides evenly in the basket.**
If the slides are stored unevenly, basket movements are affected during the staining operation.
- **Before staining is started, confirm that the end station has been set.**
If the end station is not set, the basket cannot be put down and will remain gripped. If the next basket is processed in this condition, the baskets may collide with each other and the tissues may be damaged.
- **Do not touch the moving arm.**
The arm should stop automatically when the cover is opened, but it may continue to operate if a series of operations is still ongoing. Touching the moving arm may result in injury. If you must move the arm by hand, confirm that the arm has stopped completely and then move the arm with due caution by following the onscreen messages on the display.
- **Do not open the cover or door unless necessary.**
Open the cover or door only when required by the operations explained in this manual. If the cover or door remains open for a long period of time, organic gas may be released from the instrument. Also, opening the cover or door during staining stops the arm, which may cause the tissues to dry up or extend the staining time and eventually affect the stained quality of tissues.
- **Be careful not to pinch your hand when opening/closing the cover or door.**
Be careful not to pinch your hand between the fixed part and moving part of the cover or door or between the cover and door.
- **Do not move the instrument while operating. Move the instrument after removing all solution bottles, paraffin containers and other components carrying liquid inside.**
Moving the instrument while operating may cause the solution or paraffin to leak. Accident or failure may also occur.
- **If the straining process is suspended, the process may be negatively affected depending on the situation. Remember to resume operation.**
Prolonged suspension may negatively affect the stain process. Keep the suspension time to the minimum.
- **When the expiration date of the current solution is passed, a message appears to urge exchanging the solution. Exchange the solution properly in order to maintain the staining quality.**
- **The safety data sheet of each solution used can be obtained from the supplier/manufacturer of the solution. You can also download it from the website of respective supplier/manufacturer of the solution.**
- **Do not insert a metal bar or other metal object into the hot air outlet of any drying station or use a metal product to reset an over temperature protector.**
Metal contact may cause electric shock.

Safety Precautions



CAUTION

- **Do not allow any foreign object to be caught between the cover/door and the stainer.**
It may cause an unexpected problem.
- **Use tap water for the water supply.**
Use of water containing pollutants may result in staining problems.
- **Confirm that no paper or foreign object is present at the cover or outlet of any drying station.**
It may result in insufficient drying.
- **Do not touch the drying stations and heating stations while the instrument is operating.**
These stations become very hot during operation, so touching them may result in burns.
- **Securely connect the water supply hose and drain hose to make sure water will not leak.**
If water leaks, the facility may be damaged.
- **Before using the instrument, inspect the water supply hose and drain hose to ensure that they are free from damage and connected properly.**
It may cause an unexpected problem.
- **Close the water tap after using the instrument.**
If the water tap remains open, the water supply hose may accidentally come off due to water pressure and cause water leaks.
- **Clean the water supply strainer regularly. Also, close the supply source water tap before cleaning.**
If the water supply strainer is not cleaned regularly, the water supply volume will drop and washing problems may occur. Check the flow rate after cleaning. If the water supply strainer is cleaned with the supply source tap open, water will overflow when the strainer is removed.
- **Clean the drying station regularly.**
If the drying solution is not cleaned regularly, paraffin will accumulate in the pan and cause the temperature to vary, resulting in staining problems.
- **Clean the water supply nozzle regularly.**
If the water supply nozzle is not cleaned regularly, contaminants may attach to the nozzle and enter the tissues, causing staining problems.
- **Clean the instrument thoroughly after using corrosive solutions.**
Some solutions have corrosivity and other aggressive properties that damage the instrument. It is best to avoid using these corrosive solutions, but if you must use a corrosive solution, remove the solution from the instrument and store it properly after use and also clean the instrument thoroughly. If this is not performed, the metal parts will corrode (rust, etc.).

[Corrosive Solutions]

Acids [hydrochloric acid (HCl), hydrobromic acid (HBr), hydrofluoric acid (HF), etc.], staining solutions and reagent solutions containing hydrochloric acid (Schiff's reagent, cold Schiff, sulfurous acid solution, resorcin-fuchsin staining solution, etc.), oxidizing chlorides [ferric chloride (FeCl₃), mercuric chloride (HgCl₂), cupric chloride (CuCl₂), sodium hypochlorite (NaOCl), etc.], tioculfates [sodium tioculfate (Na₂S₂O₃), etc.], organic acids [formic acid (HCOOH), acetic acid (CH₃COOH), bromic acid (C₂O₄H₂), lactic acid (CH₃CH(OH)COOH), etc.]

Safety Precautions



CAUTION

- **When the pull-up speed is increased, more solution will be brought into the next station and accelerate solution deterioration, resulting in lower staining quality. There are also risks of tissue contamination due to dropping solution.**
- **Do not stare at the laser beam from the 2D bar code reader.**
For details, refer to the operating manual. The operating manual can be downloaded from the website of the 2D bar code reader manufacturer.
- **Observe the rated water pressure stated in the Specification of the instrument.**
If the water pressure is high, water may leak and cause electric shock or the instrument may stop.
- **Do not use any CF card other than the one that comes with the instrument. Also, run a virus check before inserting the CF card.**
- **Turn off the power before performing cleaning.**
Unless otherwise instructed in this manual, always turn off the power and disconnect the power plug before cleaning the instrument. Do not spill liquid inside the instrument while cleaning it. If cleaner is used, follow the safety precautions provided by its manufacturer and safety rules set by each test laboratory.
- **Secure the instrument with seismic anchorage (optional) to prevent it from moving/falling in case of earthquake.**
Provide fall prevention measure to prevent the instrument from moving should a disaster occur.
- **Do not apply excessive force on the display.**
The display may break and cause injury.
- **Do not remove the warning/caution labels.**
Without these labels, the operators cannot be reminded of the necessary warning/cautionary information in the day-to-day handling of the instrument, which may result in unexpected problems.
- **Do not use damaged accessories (reservoirs, baskets, basket adapters, etc.).**
The accessories deteriorate from use over time. If deteriorated accessories are used, unexpected problems may occur. Check the accessories before use.
- **Do not put solution, etc., on the instrument. Also, do not spill solution on the painted surfaces, caution/warning labels or touch screen.**
If solution is spilled over the instrument, the instrument may fail or sustain damage.
- **Do not spill solution inside or outside the instrument.**
If solution was spilled, turn off the power to the instrument and then quickly wipe off the spilled solution. The instrument may stop or the facility may be damaged, etc. Also, other solution may be contaminated. Do not use any contaminated solution.
- **Put the lids on other stations when exchanging solution.**
If solution is spilled during exchange, other stations may be contaminated. Also, do not use, but always replace, any contaminated station.
- **Do not put any obstacle within the moving range of the arm.**
Failure may occur.

Safety Precautions



CAUTION

- **Do not clean the solution reservoirs and baskets using hot water or heat the solution reservoirs containing solutions in a microwave oven.**
Heating the solution reservoirs and baskets using hot water or in a microwave oven may cause the reservoirs/baskets to deform. Severely deformed baskets and solution reservoirs may rub against each other on side faces or at corners, thereby inhibiting the operation.
- **Do not use any ultraviolet sterilizing lamp around the instrument.**
Ultraviolet light may cause the instrument characteristics to change or deteriorate.
- **Fill each solution reservoir with a specified amount of solution.**
Failure to do so may cause tissues to dry up or result in staining problems.
- **Replace the activated carbon filters in the fume control unit periodically.**
Replace the activated carbon filters regularly in order to maintain a better environment around the instrument. Expired activated carbon filters cannot maintain exhaust gas at an appropriate level of concentration. When replacing the activated carbon filters, put the lids on the solution stations or remove the solution stations by considering the possible scattering of activated carbon particles, to prevent solution contamination.
*If the instrument is connected to an external exhaust device, the activated carbon filters need not be replaced.
- **Use dedicated activated carbon filters for the fume control unit.**
Use of inappropriate activated carbon filters may cause explosion of powder dust generated from activated carbon powder.
- **Inspect the instrument every 6 months.**
Maintain/inspect the instrument periodically so that it operates correctly at all times. For details on periodic inspection, contact the Sakura Finetek sales representative.
- **If the instrument must be moved after installation, contact the Sakura Finetek Technical Support representative or local representative.**
Unexpected accident may occur by relocating the instrument, so contact the Sakura Finetek Technical Support representative or local representative.
- **Use designated consumable parts.**
When replacing consumable parts, use those specified in this manual. Use of any other consumable part may result in malfunction or failure.
- **Close the lids of solution reservoirs whenever the instrument is not operating.**
If the solution reservoir lids remain open for a prolonged period of time, organic gas may be released to the room.
- **Provide sufficient ventilation around the instrument.**
Some of the solutions used in the instrument are toxic/harmful to the human body. Accordingly, provide sufficient ventilation around the instrument.
- **Install the instrument away from other electrical equipment.**
Electromagnetic waves and vibration, heat and solution may affect and be affected by other systems located nearby.

Safety Precautions



CAUTION

- **Provide a proper space around the instrument.**
Provide at least 10 cm of clearance around the instrument and make sure the power plug is installed so that it can be accessed at any time.
- **Thoroughly check the installation location before installing the instrument.**
Install the instrument on a flat, non-slippery sturdy floor having a sufficient size. If the installation location is tilted or too small, unexpected accidents may occur.
- **Do not use the instrument in a place subject to too low or high temperatures.**
Use it in a place where the temperature is stable in a range of 10°C to 40°C.
- **Be sure to connect the power plug to a power outlet with grounding bar.**
Connect the power plug alone to a power outlet with grounding bar conforming to at least the Class D grounding specification. If the instrument is not grounded properly, fire or electric shock may result due to leak current.
- **Lightning and power outage may cause the instrument to fail and stain process may not be able to continue and adversely affect staining. If the power supply is unstable, use an uninterruptible power supply.**
Lightning may cause the instrument to fail and damage the tissues. If the instrument will be used in a region where lightning damage is a possibility, use an uninterruptible power supply. If lightning has caused an irreparable damage to the instrument, take out the basket from the instrument and take an appropriate measure.
- **Do not perform stain process if an extended period of power outage is scheduled. Staining quality may drop. Staining problems may occur depending on when the power outage occurs.**
In this case, use a uninterruptible power supply. If you find that the stain process has stopped due to an unexpected power outage lasting for an extended period of time, remove the baskets from the instrument and take appropriate action.
- **Install the power cord and cables carefully by making sure the cord and cables will not interfere with the work.**
Failure to do so may result in the operator tripping over the power cord and cables and sustaining injury.
- **Securely connect the power cord and power outlet. If the power cord came off or the power switch was turned off inadvertently, connect the power cord or turn on the power switch without delay.**
If they are loosely connected, heat may be generated at the connected part, voltage drop can occur and the instrument may not operate normally. Furthermore, unexpected accidents or injuries can occur. Also, the tissues may be damaged when the power is cut off during processing.
- **Handle the power cord with care. Also, be aware of damage caused by mice and other small animals.**
To unplug the power cord from the power outlet, do not pull the cord, but hold the plug and then pull it out. Inspect the power cord periodically. If any damage is found in the exterior of the power cord, stop using it and contact the Sakura Finetek Technical Support representative or local representative.
- **Before operating the instrument, be sure to check the electromagnetic environment.**
Example: Remove electrostatics from the operator and humidify the room.

Safety Precautions



CAUTION

- **If an error (trouble) occurs, staining problems may occur. Immediately resolve the error, protect the tissues, and contact the Sakura Finetek Technical Support representative or local representative.**

Connect an external alarm to prevent staining problems from happening when the instrument experiences problems. When introducing an external alarm, contact the Sakura Finetek Technical Support representative or local representative.
- **Conduct a staining test before determining the staining conditions.**
- **Check the staining program before the operation. If a wrong staining program is used, staining problems may occur.**

A wrong operation may interrupt the stain process or edit the program to a wrong one. If staining is started with a wrong program, staining problems may occur. Read this manual carefully and use the instrument correctly.
- **Operating the instrument with an inappropriate temperature setting or time setting (long/short) may cause staining problems. Conduct a verification test beforehand to confirm that staining can be performed properly. If you have any question regarding staining programs, contact the Sakura Finetek Technical Support representative or local representative.**
- **The allowable concentration limit of each substance in the work environment varies depending on the air temperature, size of the room, additional load, if any, ventilation efficiency, etc.**

If danger is suspected, the Instrument Control Manager should measure the concentration of the applicable substance in the area and confirm that the allowable limit in the work environment is not exceeded. Follow the standard concentration limits specified by the environmental regulation of the country/region in which the instrument is used.
- **Solution may scatter and attach to your body while working. Be sure to wear protective gears while working.**

In all operations including stain process, solution exchange, opening/closing of doors, cleaning and daily maintenance, the solutions used may scatter and attach to your body (eye, mucous membrane, skin, mouth, nose, etc.) or you may inhale their gases. Be sure to wear protective gears (gloves, organic solvent mask, goggles) while working.
- **If any solution harmful to human health attached to your body or entered your mouth, take an appropriate action according to the SDS of the solution.**

Wear protective gears (mask, gloves, goggles) during work in case anything goes wrong. Use protective gears that are resistant to all solutions on the solution list.
- **You may accidentally touch an infectious source and operate the instrument with the contaminated hands. Be sure to wear protective gears when operating the instrument.**

If you have operated the instrument after handling a pathogen, disinfect the instrument properly. The disinfection method varies depending on the type of pathogen, so choose a method appropriate for the pathogen. Also, disinfect/sterilize or otherwise properly treat the protective gears used and then dispose of them as infectious waste. For the disposal method, follow the applicable regulation and guideline specified by each institution, country or region.

Safety Precautions



CAUTION

- **Exercise due caution when handling solutions. Follow the relevant regulations and guidelines in your country or region and take proper measures to protect the operator, such as wearing gloves, mask and/or safety goggles.**
Some solutions are toxic or harmful to the human body. Some solutions are very flammable, combustible, and/or harmful to human body. Use protective gears that are resistant to all solutions on the solution list.
- **Sterilize any potentially infectious waste material or infectious medical waste and then dispose of it according to the applicable regulation or guideline in the country or region.**
- **When the instrument is used to handle any substance presenting biological hazard, ensure safety according to the applicable regulation or guideline in the country or region. When performing operations that involve any such substance, also wear gloves, mask, safety goggles and/or other protective gears and perform the operations inside a safety cabinet by taking appropriate measures to protect yourself as the operator.**
- **Handling of solutions requires expert knowledge on associated hazards due to their toxicity and volatility, etc.**
Appoint a person responsible for solutions and use solutions under the guidance/supervision of the responsible person.
- **If any additional requirement on accident prevention or environmental preservation is specified in the country/region where this instrument is used, follow the instructions under the applicable law/regulation in addition to the content of this manual to ensure compliance with the requirement in the applicable country/region.**
- **When disposing of used solutions, activated carbon cartridges, consumable accessories (solution reservoirs, baskets, basket adapters, etc.), follow the applicable regulation in the country or region.**

Explanation of Terms

- **Mix**
To cause a basket to move up and down inside a solution-filled station before or after the basket is transferred.
- **Enhanced Wash**
To repeat the process of immersing a basket into a solution and then removing it completely out of the solution inside a station.
- **Slide**
A slide made of glass on which tissues are pasted.
- **Basket**
A special staining basket in which slides are set. Two types of baskets, 10-slide type and 20-slide type, are available.
- **Batch mode**
An operation mode in which the next staining run cannot be started unless the current staining run is completed.
- **Continuous mode**
An operation mode in which staining runs of different tissues can be started during the current staining run.
- **Solution station**
A general term referring to both solution reservoirs and wash reservoirs.
- **Start station**
A solution station in which baskets are set at the start of staining.
- **End station**
A solution station in which stained baskets are stored.
- **Program End (PE) station**
A solution station in which baskets are temporarily held when the staining program is aborted in the middle.



Chapter 1

Basics of the System



Installation Method

Installing the Instrument

This section provides information on determining a location for, and installing the instrument. Installation should be performed by qualified service staff working exclusively with the instrument. The instrument must be installed correctly to ensure proper operation and service. Read this Operating Manual carefully before operating the instrument. Strictly observe all instructions provided in this document.



This is a precision instrument and must be handled with care. If the instrument is handled in a rough manner or dropped, its internal components may malfunction or get damaged. Always exercise due care when handling the instrument.

Flow of Installation

• Checking the Installation Location and Transporting the Instrument Details: P. 1-2

- Checking the installation conditions
- Checking the work area
- Transporting the instrument

• Unpacking and Checking the Installation Details: P. 1-4

- Unpacking and installation
- Installation check

• Setting before Commencing Operation Details: P. 1-16

- Turning on the power

Required Tools

Name	Size	Q'ty
Transport handle	-	4
Phillips screwdriver	ISO No. 2	1
Adjustable wrench	300 mm or more	1
Wrench or box wrench	24 mm across flats	2
Convex ruler (steel tape measure)	Capable of measuring objects of 2.5 m or larger	1
Level	-	1
Cutter	-	1
Pipe wrench	200 mm or more	1

Installation Environment

As is the case with all electronic devices, the instrument should not be installed in places subject to excessive temperature or humidity.

Temperature and humidity should be held at constant levels to achieve the highest degree of operating stability. The ambient operating temperature of the instrument is in a range of +10 to +40°C. Its ambient operating humidity is in a range of 30 to 85% in relative humidity (non-condensing).

Also, locate the instrument, with at least 10 cm away from walls, in a well-ventilated place not exposed to corrosive vapor or extremely high/low temperatures or where the instrument will not come in direct contact with air flow. Install the instrument away from direct sunlight and do not install it proximity to open windows, ovens, open flames, hot plates, radiators, and dry ice baths. The instrument should be installed away from any equipment carrying high voltage or current, such as a large refrigerator or oven. Since the instrument is very heavy, the experiment bench should have sufficient bearing strength, and the instrument should be laid flat.


The instrument should be installed near a power supply meeting the power facility requirements (voltage, current) specified in this Operating Manual. Be sure to ground the power outlet and wire the power properly using a dedicated cable.



If the instrument is used in a non-ventilated place, serious health risks may occur to the human body.

Installation Method

Checking the Installation Conditions

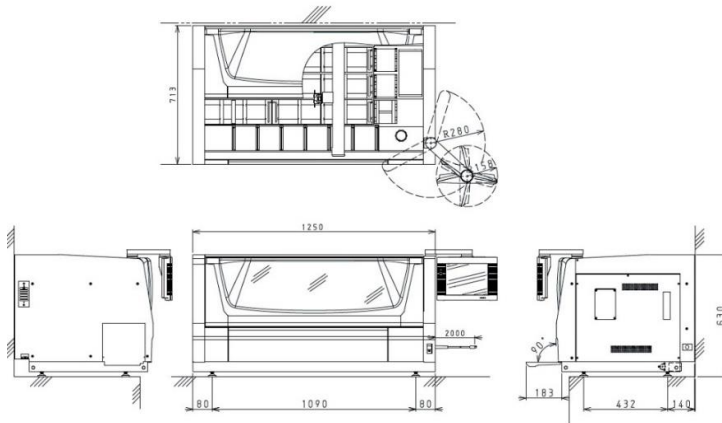

■ Checking the installation location

Confirm beforehand that the location is flat, strong and meets the dimensions of installation location (installation space) shown in the figure on the left. Confirm that no foreign objects that may block the vents of the instrument (such as hanging towels, etc.) are present in and around the installation location. Confirm that the facility-side power supply conforms to the requirements.

■ Checking the facility-side power supply

Confirm that the facility-side power supply conforms to the requirements.

Note The instrument can support the power supplies listed below using the selector switches. Switching of the voltage setting should only be performed by qualified service staff working exclusively with the instrument.

Facility requirements

- | | | |
|----------------------------|------------------------|---------------------------------|
| • Single-phase 100 VAC±10% | 50/60 Hz: 15 A or more | Power outlet with grounding bar |
| • Single-phase 115 VAC±10% | 50/60 Hz: 15 A or more | Power outlet with grounding bar |
| • Single-phase 230 VAC±10% | 50/60 Hz: 7 A or more | Power outlet with grounding bar |

Warning! If the above facility requirements are not satisfied (such as when the power supply used is 200 VAC), the instrument must not be used. If the instrument generates abnormality, the breaker on the facility side may actuate and other equipment may be affected; accordingly, connect each instrument to a dedicated power outlet (including a dedicated breaker on the facility side).

- Specification of the power cord supplied with the instrument

Type of power cord	Applicable standard	Length
For 100/115 VAC	UL/CSA, PSE	Approx. 2 m
For 230 VAC	Cord: CENELEC HD VDE Plug: EN IEC VDE Connector: EN VDE	Approx. 2 m

Warning! If the instrument is used in a region where the applicable standard above is not met, provide a different power cord conforming to the standard in the location of use.

- About grounding

Confirm with the person responsible for the facility that the power outlet to be used is Class-D grounded.

■ Confirm that a ventilation system, local exhaust system or exhaust port leading to the outside of the building is available on the facility side.

Although the volumes of volatile solution gases released from the instrument vary depending on the solutions and processing programs used and the environmental temperature, volumes of gases released per unit time are provided below for reference. Based on these values and the volumetric size of the installation location, check if the ventilation rate is sufficient to meet the requirements at the installation location pertaining to the concentrations of substances in the work environment. If no ventilation system is available, use a Ø38 or Ø75 exhaust duct adapter (optional) to connect the duct hose to the exhaust port of the instrument, and discharge the exhaust fume to the outside of the building.

Note Maximum volumes of gases released per unit time (reference)

- Ethanol: 3.9×10^{-3} (m³/hr)
Xylene: 4.8×10^{-4} (m³/hr)

Installation Method

Checking the Work Area

The dimensions and weight of the instrument in packed state are as follows. Check them beforehand so that sufficient space and safety can be ensured when "transporting, unpacking and installing" the instrument.



"Dimensions and weight in packed state"

W: Approx. 1500 mm
H: Approx. 930 mm
D: Approx. 900 mm
Weight: Approx. 210kg

"Instrument dimensions and weight"


W: Approx. 1250 mm (excluding the display)
H: Approx. 630 mm
D: Approx. 713 mm
Weight: Approx. 150kg

Transportation

Caution!

When transporting the instrument on a pallet truck, etc., one person should not work alone in order to prevent instrument damage or accident caused by rear-end collision, dropping of the instrument, etc. When transporting the instrument, place thin iron sheets, etc., to eliminate height gaps on the floor so that the instrument will not receive impact.

Pay due attention to safety. Particularly when transporting the instrument on a slope, remember that, because the instrument is heavy, stopping the instrument will not be easy once it began accelerating.

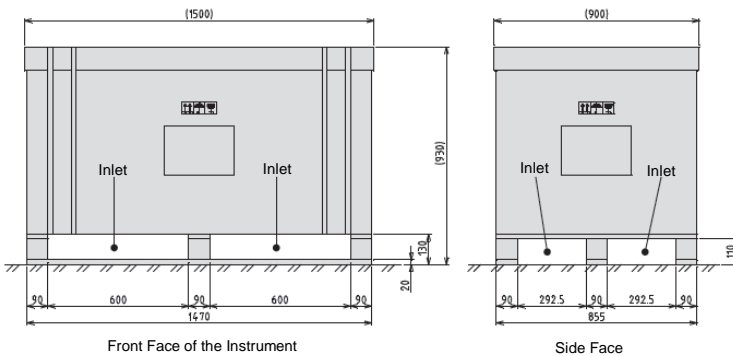

■ Transporting the instrument to the work area

• Transporting on a forklift or pallet truck

When transporting the instrument on a forklift or pallet truck, do so when the instrument is still packed, and insert the fork into the packing pallet as shown in the figure below. Refer to the palletizing size in the figure on the left and check the dimensions of the forklift or pallet truck beforehand.

■ Moving to the installation location

1. Unpack the instrument by referring to the next section, "Unpacking Procedure - Unpacking the system," and move the instrument to the installation location.
2. If the instrument can be transported on a pallet truck, etc., move the whole packing pallet to near the installation location.
3. Remove the L brackets securing the instrument to the packing pallet.
4. Screw in the transport handles into the treaded parts on the left and right side faces at the bottom of the instrument. Securely screw in the transport handles all the way.
5. Hold the transport handles to lift the instrument and move it to the installation location.

Caution!

When using a pallet truck, etc., for transport, confirm that the instrument and packing pallet are secured together with L brackets, because otherwise the instrument may shift and drop from the packing pallet. When the instrument must be pushed while being transported on a pallet truck, etc., pay attention to where on the instrument to be pushed, because the exterior panels may be deformed or the acrylic cover may be damaged. (Do this by holding the transport handles at the base of the system.) Pay due attention to safety, and particularly when transporting the instrument on a slope, remember that, because the instrument is heavy, stopping the instrument will not be easy once it began accelerating. When lifting the instrument to move, be sure to secure a line of movement or footing.

Installation Method


Unpacking Procedure

How to remove the instrument from the packing box that has been carried in, and remove the protective materials, is explained. To perform each work safely and correctly, be sure to follow the procedure specified herein.

Warning! Confirm before unpacking the instrument, that the packing materials have not been damaged due to external forces applied during transport. Two or more persons should always work together to unpack the instrument. Be careful not to damage/scratch the instrument when unpacking. When taking the accessory box out of the instrument, assume a natural posture.

Unpacking the System


1. Cut the bands with a cutting knife, etc. and remove the cap.

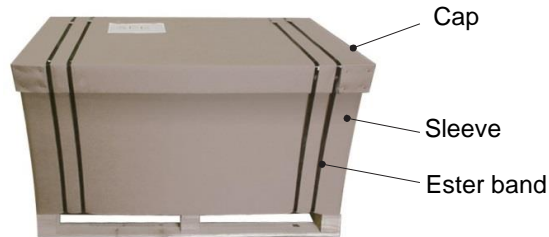

2. Remove the cushion materials put around the instrument, and take out the sleeves and polyethylene bags.




3. Remove the cushion materials inserted into the front cover from both sides, lift the front cover, and take out the cushion materials sandwiched between the front cover and the front door.

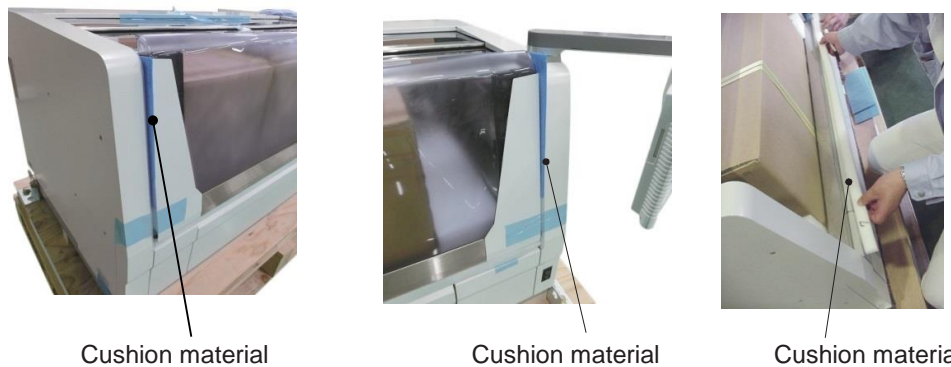


Installation Method


4. Remove the tape securing the top cover, and take out the cushion materials sandwiched between the cover and the accessory box.

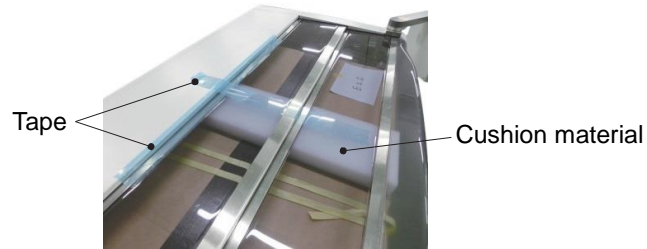

5. Cut the PP band and take out the accessory box.

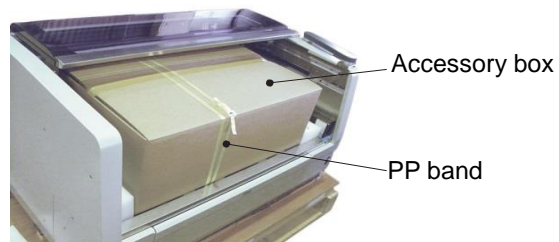


6. Remove the cushion materials inserted underneath the display arm, and the cushion foams wrapping the instrument.

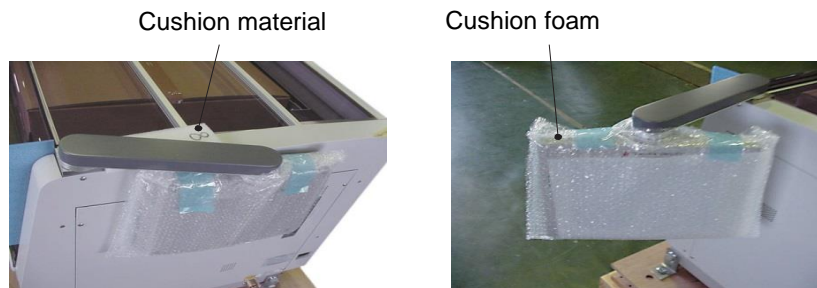

Installation Method


Removing the Securing Members Inside the Instrument

1. Remove the cushion materials sandwiched between the robot arm and the rail.

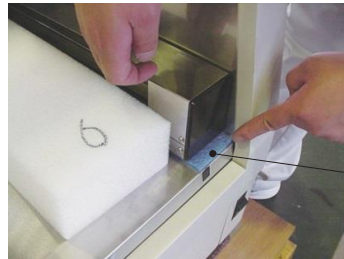


Cushion material

2. Remove the tape securing the holder, and remove the cushion materials inserted underneath the holder.

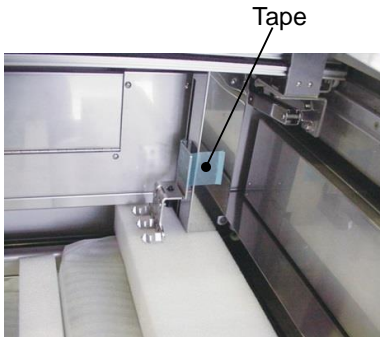

Tape

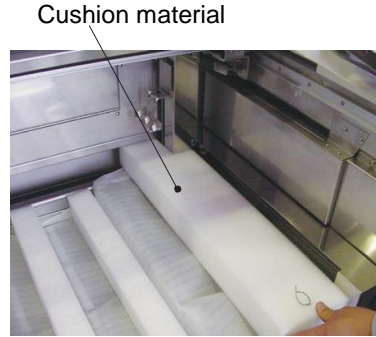

Cushion material

3. Remove the reinforcing cushion materials on the tray, and take out the cushion foam sheet and PP band.

Cushion material Cushion material



Cushion material




PP band Cushion foam sheet




4. Remove all 3-position and 4-position reservoir trays stored in the system, and remove the tape securing the cushion foam sheet and the water supply nozzle.

Cushion foam sheet

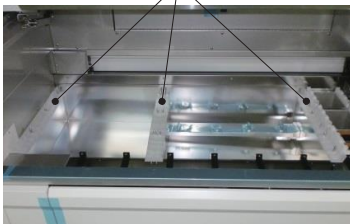

Sheet securing the water supply nozzle

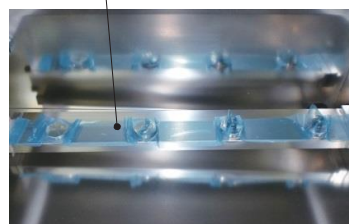

Installation Method

Switching the Power-supply Voltage Setting



This instrument supports multiple voltages, but the setting on the instrument side must be switched according to the power-supply voltage. If the instrument is used without switching the voltage setting, not only the instrument will break down, but the user may also be exposed to danger; accordingly, contact the Sakura Finetek Technical Support representative or local representative when the instrument is installed to switch the voltage setting according to the voltage on the facility side. The factory setting is 230 V.

Checking the Accessories

Product models in the instruction sheet are as shown below. Pay attention that accessories are different depending on the model.

Model	Overview
Tissue-Tek PrismaPlus (Model Prisma-P-AD)	w/ Drying Function
Tissue-Tek PrismaPlus (Model Prisma-P-AS)	w/ Drying/Heating/Special Staining Functions
Tissue-Tek PrismaPlus (Model Prisma-P-ED)	w/ Drying Function
Tissue-Tek PrismaPlus (Model Prisma-P-ES)	w/ Drying/Heating/Special Staining Functions
Tissue-Tek PrismaPlus (Model DRS-Prisma-P-JD)	w/ Drying Function
Tissue-Tek PrismaPlus (Model DRS-Prisma-P-JS)	w/ Drying/Heating/Special Staining Functions
Tissue-Tek PrismaPlus (Model DRS-Prisma-P-JCS)	w/ Drying/Heating/Special Staining Functions



Confirm that all accessories (refer to the table below) are present and that the instrument and accessories are free from damage, and if the instrument is damaged or any of the accessories is short or damaged, contact the Sakura Finetek sales representative.

No.	Name of part	Qty	Remarks	
1	3-position reservoir tray (2-phase stainless) *3	3	-AD, -AS, -JD, -JS, -JCS	
2	3-position reservoir tray	3	-ED, -ES	
3	4-position reservoir tray (2-phase stainless) *3	3	-AD, -AS, -JD, -JS, -JCS	
4	4-position reservoir tray	3	-ED, -ES	
5	Control panel protection sheet	1	Comes attached to the control panel	
6	Paraffin tray	2		
7	Hot air outlet panel, drying station	2		
8	Bushing (bolt cap)	4		
9	Heating solution reservoir	2	-AS, -ES, -JS, -JCS	
10	Water supply hose (faucet connection type)	1	-ED, -ES, -JD, -JS, -JCS	
11	Nipple (1/2 inch)	1(2)	2 for -ED, -ES	
12	Bushing (1/2 x 3/4 inch)	1	-ED, -ES, -JD, -JS, -JCS	
13	Bushing (machined)	1	-AD, -AS	
14	Drain hose unit (∅38 mm x 2 mm + cuffs)	1		
15	Drain elbow	1		
16	Spiral wire band	1		
17	Hose clamp	1		
18	Power cord 100/115 VAC power plug	1	Not applicable to -JCS	
19	Power cord 230 VAC power plug	1		
20	Ferrite core	1	For input in case of power outage	
21	Activated carbon filter	2		
22	20-slide basket	10		
23	20-slide basket adapter	10		
24	Nozzle	4	Of the four pieces, two are preinstalled on the instrument.	
25	Wash cap	4	Of the four pieces, two are preinstalled on the instrument.	
26	Cap for covering water supply port	4	Of the four pieces, two are preinstalled on the instrument.	
27	Cap for LAN port	1	For LAN port; installed on the port.	
28	Cap for RS232C port	1	For RS232C port; installed on the port.	
29	CF card	1		
30	Station labels (Start, End)	1	3 types/set	
31	Basket adapter label	1		
32	Operating Manual	1		
33	Package insert for medical equipment	1	-JD, -JS	
34	Excerpt from Unpacking/Installation Manual	1		
35	Warranty registration card	1	-JD, -JS	
36	10-slide basket	10	-AS, -ES, -JS, -JCS	
37	10-slide basket adapter	10	-AS, -ES, -JS, -JCS	
38	10-slide basket load adapter	3	-AS, -ES, -JS, -JCS	
39	Special stain solution reservoir	8	-AS, -ES, -JS, -JCS	
40	Lid, special stain solution reservoir	8	-AS, -ES, -JS, -JCS	
41	Special stain reservoir tray (2-phase stainless)	1	-AS, -JS, -JCS	
42	Special stain reservoir tray	1	-ES	
43	Standard solution reservoir	22	-AD, -AS, -ED, -ES, -JCS	
44	Solution reservoir with handle	6	-AD, -AS, -ED, -ES, -JCS	
45	Wash reservoir	4	-AD, -AS, -ED, -ES, -JCS	
46	Lid, standard solution reservoir	8	-AD, -AS, -ED, -ES, -JCS	
47	Large lid, 3-position reservoir tray (2-phase stainless)	3	-AD, -AS, -JCS	
48	Large lid, 3-position reservoir tray	3	-ED, -ES	
49	Large lid, 4-position reservoir tray (2-phase stainless)	3	-AD, -AS, -JCS	
50	Large lid, 4-position reservoir tray	3	-ED, -ES	
51	(Breakdown)	Standard solution reservoir set (sold as a set) *2	1	-JD, -JS
		Standard solution reservoir	22	
		Solution reservoir with handle	6	
		Wash reservoir	4	
		Lid, standard solution reservoir	28	
52	(Breakdown)	Small solution reservoir set (sold as a set) *2	1	-JD, -JS
		Solution reservoir with handle	6	
		Wash reservoir	4	
		Lid, standard solution reservoir	6	
		Small solution reservoir	44	
		Lid, small solution reservoir	44	
20-slide basket load/unload adapter	3			
53	Strainer	1	-ED, -ES	
54	Street elbow (1/2 inch)			

Installation Method

- *1 The accessories are compatible with all models, unless the applicable models are specified in the Remarks field.
- *2 Select either the standard solution reservoir set or small solution reservoir set. (-JD/-JS only)
- *3 Housed in the instrument.

Installation Method

Checking the Installation (Securing the Instrument)




1. Confirm that the location where the instrument will be installed is flat and sufficiently strong.
2. Two or more persons work together to lift the instrument and move it to the installation location. If the instrument rattles or is inclined too much when installed, adjust the level adjusters underneath the instrument. Also when the instrument is linked to the Tissue-Tek Film or Tissue-Tek Glass g2, use the level adjusters underneath the instrument to adjust the height and inclination of the instrument.
3. Once the instrument has been installed, remove the transport handles and install the attached bushing (bolt caps) in the threaded parts on the system.

Installing the parts around the instrument

- **Attach the water supply hose connect it to a water supply facility (tap water supply) meeting the following conditions:**

Pressure:	Dynamic pressure = 0.098 to 0.441 MPa Maximum static pressure = 0.74 MPa
Temperature:	30°C max. (non-freezing)
Faucet:	A faucet of general shape having a tip of 12 mm to 17 mm in size. (The water supply hose cannot be connected to a faucet of any other size or to a special shaped faucet such as chemical faucet or aerator faucet.)
Facility requirements:	Use a facility meeting the following items in order to prevent health damage caused by flow-back: [1] Water supply facility not directly connected to public water pipes [2] Water supply facility not directly connected to a water supply system used for drinking water, etc. [3] Water supply facility designed to prevent flow-back caused by reverse-siphoning • Water supply facility equipped with a flow-back protection mechanism meeting the law of each country/region

1. Connect a nipple of the water supply port on the instrument via a bushing, and then screw the threaded part of the water supply hose into the bushing.
2. Pull the flange on the other end of the water supply hose toward the hose and separate the water tap joint.
3. Loosen the four screws on the water tap joint.
4. Tighten the four screws uniformly while pressing the black rubber in the water tap joint against the tip of the water faucet.
5. Turn and tighten the nipple.
6. Insert the water supply hose into the water tap joint with the flange pulled back.
7. Gently pull the water supply hose by hand to confirm that it does not come off from the water tap joint.

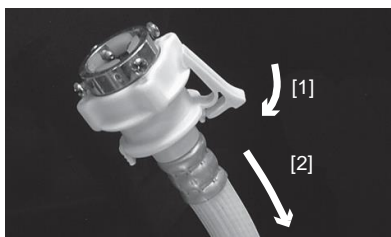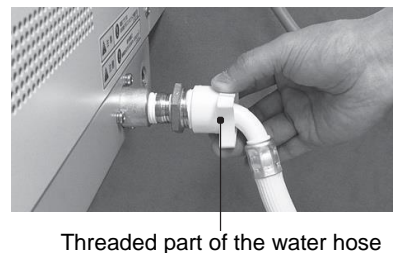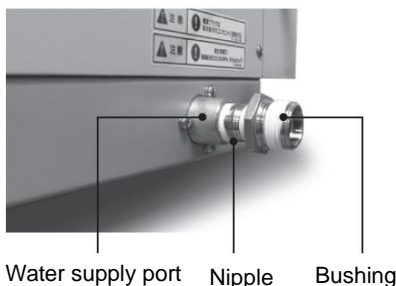

How to separate the water tap joint



Installation Method

● Attaching the water drain hose

Facility requirements: [1] Sink, drain ditch or other water drain facility not subject to back pressure
[2] Water drain facility that will not negatively affect the environment or human body even when chemicals used for staining enter the system

1. Install a spiral wire band on one end of the water drain hose, connect a drain elbow to the drain hose, and then tighten the screws.
2. Install a hose clamp to the drain port on the instrument, and then insert the drain elbow into the drain port.
3. Tighten the screws on the hose clamp using a flathead screwdriver.



4. Connect the other end of the drain hose to a sink having a minimum drain capacity of 15 L/min. (Connect the hose securely so that it will not separate from the sink.)
5. Ensure that the drain port on the instrument is positioned higher than the sink, that the drain hose has no slack in the middle, and that the outlet of the drain hose is away from the water level of the sink by at least 100 mm.

Note Do not extend the drain hose using a different hose. The drain elbow can be oriented to either right or left.
The drain hose contains vinyl chloride, so be careful not to spill organic solvents onto the hose.
After connecting the water supply hose and drain hose, confirm by referring to P. 1-42 that water is not leaking.

● Installation of exhaust system

The instrument generates volatile solution gases, so install it where a ventilation system, local exhaust system or other exhaust facility, or exhaust port leading to the outside of the building, is available. Also, estimate the exhaust gas concentration from the reference value of maximum volume of gas released and the volumetric size of the installation location, and confirm that the ventilation rate of the installed ventilation facility is sufficient to meet the requirements pertaining to the concentrations of substances in the work environment. Ideally the instrument is installed directly near a ventilation facility, if it must be installed away from a ventilation facility; however, use a $\varnothing 38$ or $\varnothing 75$ exhaust duct adapter (optional) to connect the duct hose to the exhaust port of the instrument, and discharge the exhaust fume near the exhaust facility. Even when no exhaust facility is available, use a duct hose to discharge the exhaust fume to the outside of the building.

Maximum volumes of gases released per unit time (reference value)

- Note**
- Ethanol: 3.9×10^{-3} (m³/hr)
 - Xylene: 4.8×10^{-4} (m³/hr)

○ Facility Check

- Ventilation system
- Local exhaust system
- Exhaust port

○ Required Options

- Exhaust duct adapter, 38 mm dia.
- Exhaust duct adapter, 75 mm dia.

○ How to Connect

1. Attach the $\varnothing 38$ or $\varnothing 75$ exhaust duct adapter to the exhaust port located at the back of the system on the left side.
2. Insert a duct hose into the adapter and tighten it with a wire clamp, etc.
3. Bring the open end of the duct hose to near a ventilation system or local exhaust system or to the outside of the building.

The duct hose should not be longer than 10 m, and confirm that the hose is not bent or crushed in the middle. If the ambient temperature is low, drain water may collect inside the duct hose; accordingly, make sure drain water will not collect in any part of the duct hose and also install a drain pan at the tip of the duct hose or take other appropriate measure.


Installation Method

- **Installing the water supply strainer (Prisma-P-ED, ES Only)**


Depending on the condition of the maintenance space available on the right side of the instrument, install the attached water supply strainer on the water supply port, and then connect the water supply hose.

1. Enough maintenance space of 200 mm or more is available on the right side face of the instrument.


(1) Screw the connection nipple and bushing into the attached strainer.

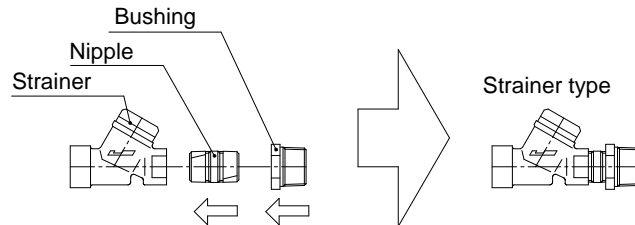

(2) Screw the attached nipple, and the strainer set assembled in (1), into the water supply port on the right side face of the instrument.

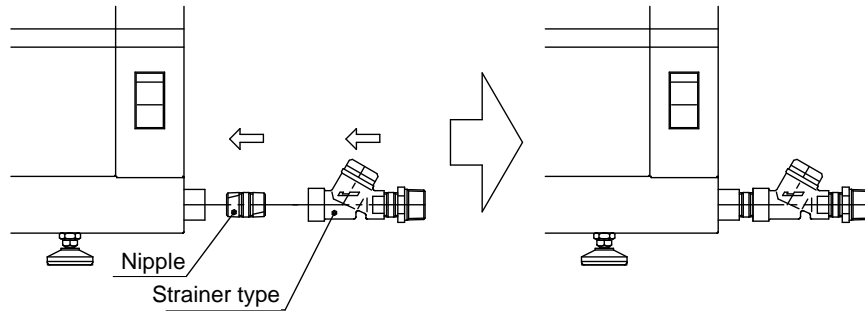


(3) Screw in the water supply hose, and connect the water supply hose to the tap water supply according to P. 1-9.

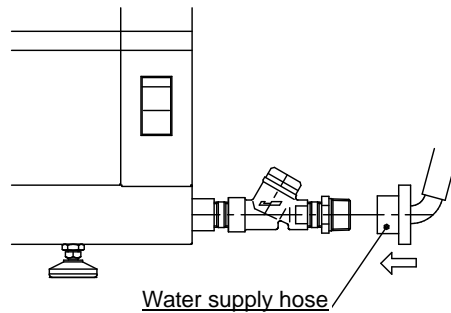


Installation Method

2. No more than 100 to 200 mm of maintenance space is available on the right side face of the instrument
In this case, approx. 35 mm of space may be required at the back of the instrument.


(1) Screw the connection nipple and bushing into the attached strainer.

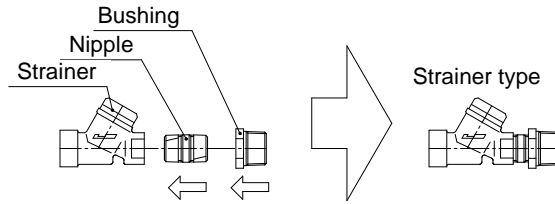


(2) Screw the attached nipple and street elbow into the water supply port on the right side face of the instrument.

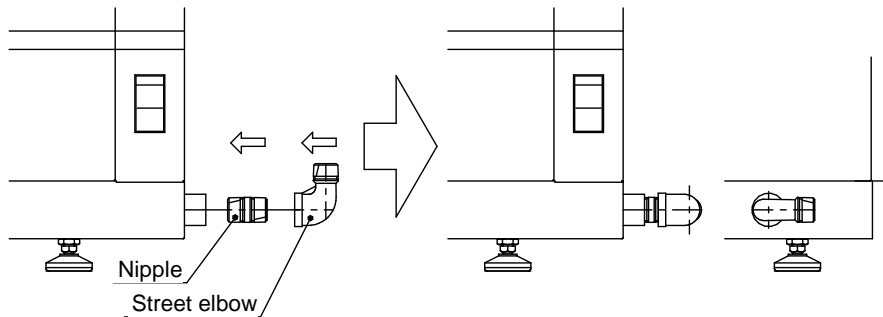

(3) Screw in the strainer set assembled in (1).

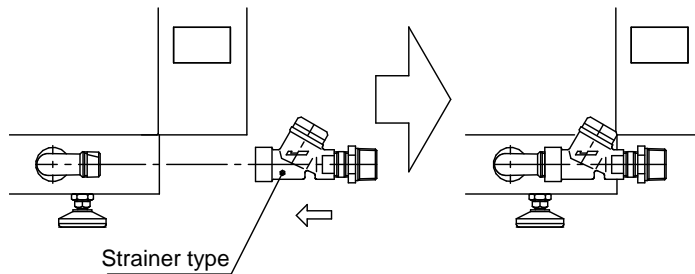

(4) Screw in the water supply hose, and connect the water supply hose to the tap water supply according to P. 1-9.

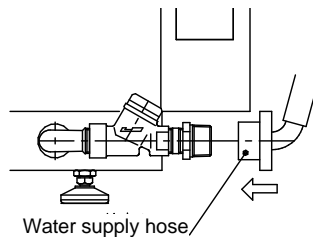

Installation Method

- **Setting the water supply nozzle**

The wash stations are set in the 4-position reservoir tray at the rearmost position of the cabinet. When the 4-position reservoir tray is removed, you will see four water supply nozzles. If you have changed the positions or quantity of wash stations, change the nozzle positions accordingly. The water supply nozzle can be replaced as follows:


1. Remove the 4-position reservoir tray at the installed position of the wash reservoir.
2. Remove the cap or wash station nozzle currently set.
The cap can be removed by turning it counterclockwise. To remove the nozzle, turn the wash cap counterclockwise to remove it, and then remove the wash station nozzle by pulling it straight up.
3. Set the cap if you want to seal the water supply part. At this time, confirm that a waterproof rubber gasket is placed inside the hole, and then turn the cap clockwise to secure it.
4. To install the wash station nozzle, first align the small hole in the waterproof rubber gasket with the small hole in the water supply outlet. Next, put the small projection on the bottom face of the wash station nozzle into the small hole in the water supply outlet. Lastly, put the wash cap on and turn it clockwise to secure the cap.



Rubber gasket

Wash station nozzle


Wash cap

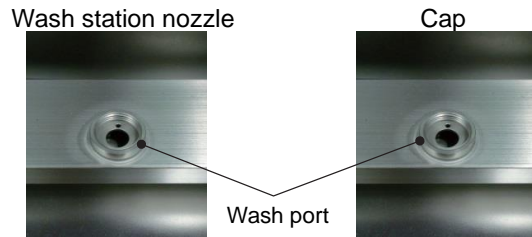


Wash station nozzle

Cap

Wash port

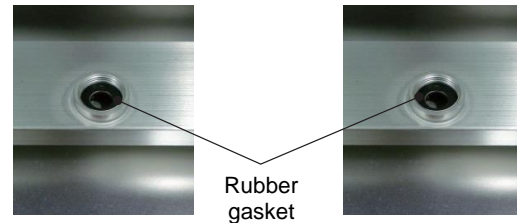


Rubber gasket

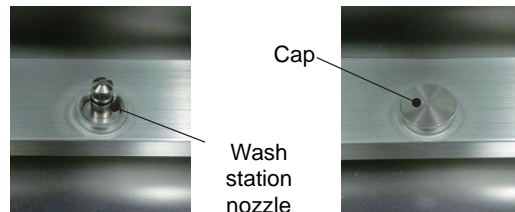

Cap

Wash station nozzle

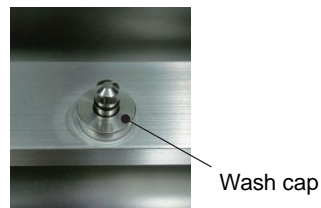


Wash cap

Installation Method

Installation of Accessories

- **Installing the activated carbon filter**

1. Remove the tape securing the fume control unit door, and open the door [1].
2. Set the attached activated carbon filters, and close the door.

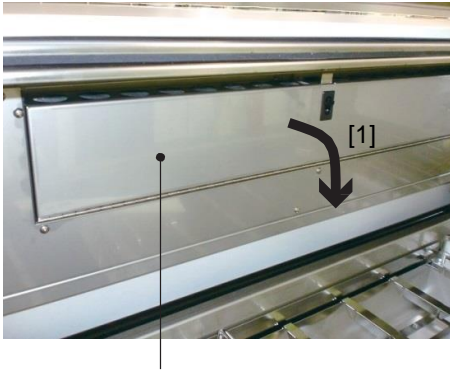

Fume control unit door





Activated carbon filter (2 pcs)

- **Installing the arm holder (attachment for standard solution reservoir)**

1. Align the grooves on both sides of the tab on the arm with the grooves provided on the inside of the arm holder, and install the arm holder from above.
2. The arm holder has been securely installed when the lock makes a "click" sound.
3. To remove the arm holder, raise the arm holder [2] while pulling the lock toward you [1].




Caution The holding part of the basket adapter is sharp, so exercise caution.

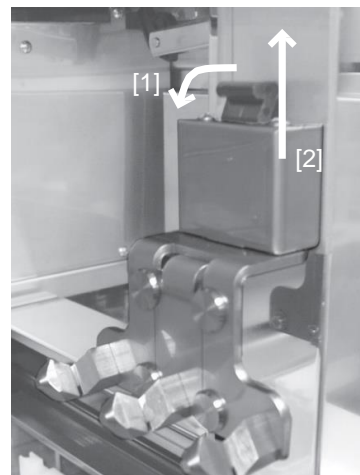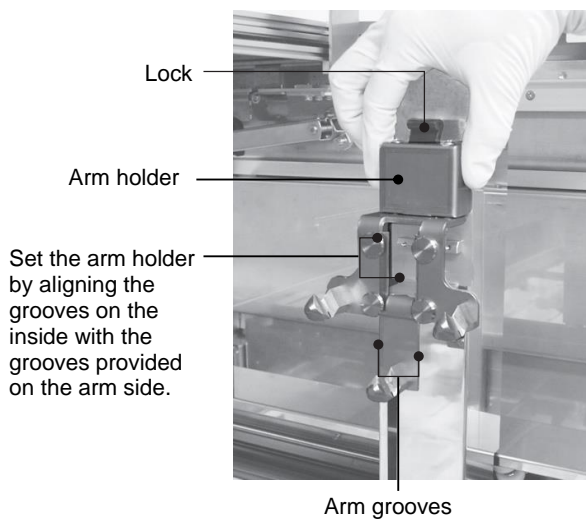

Installation Method

- **Installing each tray**

Set each tray with the two holes provided on the outer side of the handles on both sides of the tray (see the figure below) facing the rear. The position of each tray is shown below. Trays cannot be placed in the positions filled with gray.

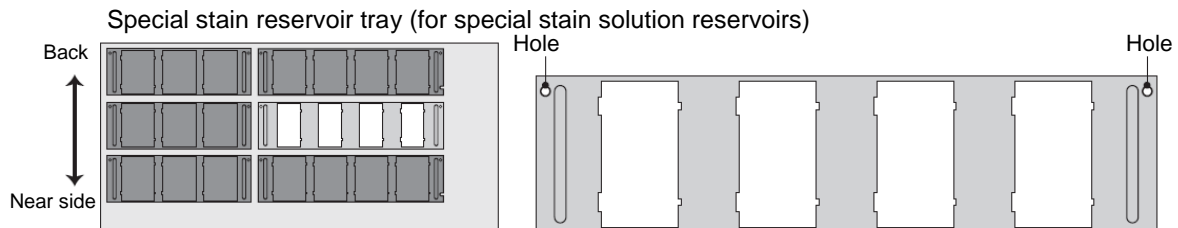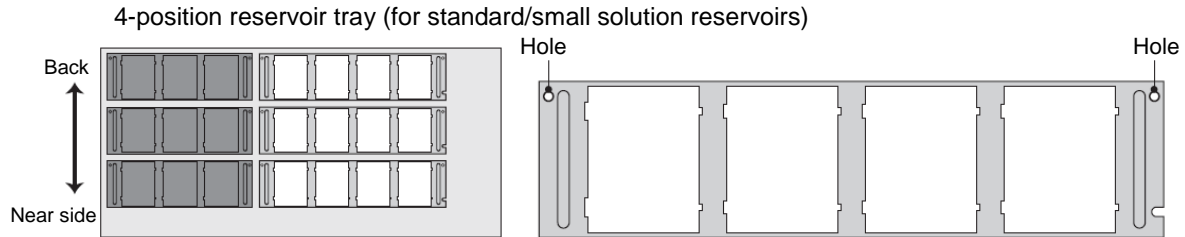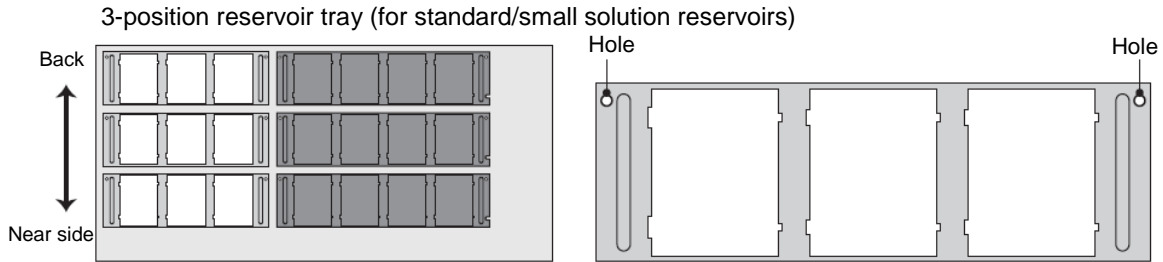

Installation Method

- **Installing the solution reservoirs**


Set the solution reservoirs in each tray. There are three types of solution reservoirs: standard solution reservoir, small solution reservoir and special stain solution reservoir. The type of applicable tray and setting orientation are different for each solution reservoir type.

- **Type**

A 3-position or 4-position reservoir tray contains standard solution reservoirs and small solution reservoirs. A special stain reservoir tray contains only special stain solution reservoirs.

- **Direction**

Orient each solution reservoir so that the "solution name tab" on top of the reservoir faces the front side of the instrument, and set the reservoir by aligning the "grooves" on both sides of the reservoir with the grooves in the tray. Pay attention to the widths of the grooves provided on both sides of the solution reservoir. When the solution reservoir is set in the tray, the wider grooves should come to the left, while the narrower grooves should come to the right.

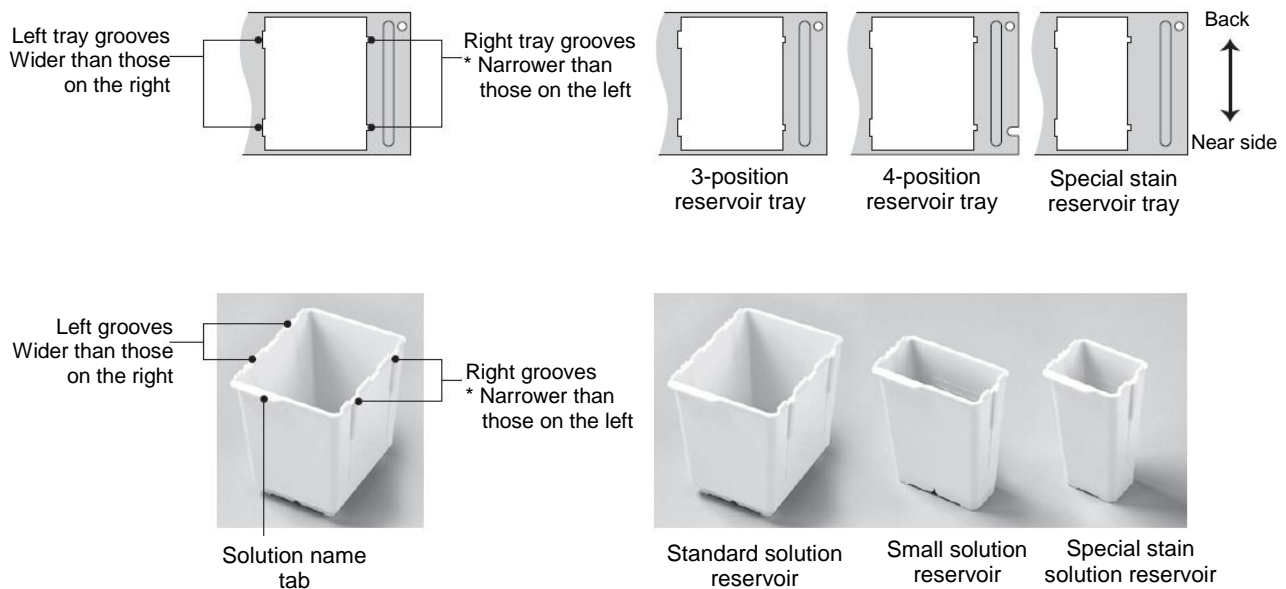

Turning on the Power



Use a power cord appropriate for the power-supply voltage. Be sure to connect the power cord to a power outlet with grounding bar.

1. Confirm that input voltage has been set according to the installation environment.
2. Connect the power cord to the power-supply inlet on the right side face of the instrument at the bottom.
3. Confirm that the power switch at the bottom right on the front face of the instrument is turned off, and then connect the plug on the power cord to the power outlet on the facility side.
4. Turn on the power switch to start the instrument.

Specification "Prisma-P-AD/AS"

General name		Automated slide stainer
Brand name		Tissue-Tek Prisma®Plus
Type	Prisma-P-AD	Prisma-P-AS
Product code	6170	6171
Manufacturing license number		20B2X00014000034
Dimensions		1250 (W) x 713 (D) x 650 (H) mm (excluding the control monitor)
Weight		Approximately 150 kg
Rated power supply		Single-phase 115 VAC \pm 10%, 50/60 Hz, 11 Amps
Power consumption		1000 VA
Electric shock protection		Class I, overvoltage category II
Pollution degree		Pollution degree I
Capabilities	Stain process	Batch mode: 60, 20 or 10 slides Continuous mode: Up to 11 runs can be set in the system.
	Staining programs	Up to 50 programs can be stored. One program can consist of up to 50 steps.
	Standard solution name	Up to 100 solution names can be stored.
	Solution configuration name	Up to 50 solution configurations can be stored.
Drying station		Temperature setting range: 30 to 65°C
Heating station		- Temperature setting range: 30 to 70°C
Required facilities	Power supply	Voltage = Rated voltage \pm 10%, 50/60 Hz Current capacity = Rated current min. Grounding = Comes with grounding bar conforming to class D grounding
	Water supply	Water = Tap water Pressure Dynamic water = 0.098 to 0.441 MPa Maximum static pressure = 0.74 MPa Temperature = 30°C max. (non-freezing) Nominal pipe diameter = 15A (1/2 in) min.
	Drain	Method = Indirect draining Drain capacity = 15 L per minute min. Nominal pipe diameter = 40A (1-1/2 in) min.
Operating environment	Ambient temperature	10 to 40°C
	Relative humidity	30 to 85% (non-condensing)
Storage environment	Ambient temperature	-10 to 65°C (instrument interior must be drained.)
	Relative humidity	20 to 90% (non-condensing)
Vibration/drop		Drop test JISZ0200:2013 (Level 1) Vibration test JISZ0232:2004 (Standard of Truck Transportation)
Noise		65 dB or less (A characteristics)
Law and conforming standard	Law	FDA Class 1
	Electrical safety evaluation	UL 61010-1:2012 CAN/CSA-C22.2 No. 61010-1-12: 2012 CAN/CSA-C22.2 No. 61010-2-010-04: 2014 CAN/CSA-C22.2 No. 61010-2-101-04: 2014
	Risk management	ISO 14971: 2007 IEC 62366: 2007
	Medical equipment software	IEC 62304: 2006
	Quality management	ISO 13485: 2003
	Electromagnetic compatibility	IEC61326-2-6:2005


Specification "Prisma-P-ED/ES"

General name		Automated slide stainer
Brand name		Tissue-Tek Prisma®Plus
Model		Prisma-P-ED Prisma-P-ES
Product code		6172 6173
Manufacturing license number		20B2X00014000034
Dimensions		1250 (W) x 713 (D) x 650 (H) mm (excluding the control monitor)
Weight		Approximately 150 kg
Rated power supply		Single-phase 230 VAC ± 10%, 50/60 Hz, 5.5 Amps
Power consumption		1000 VA
Electric shock protection		Class I, overvoltage category II
Pollution degree		Pollution degree I
Capabilities	Stain process	Batch mode: 60, 20 or 10 slides Continuous mode: Up to 11 runs can be set in the instrument.
	Staining programs	Up to 50 programs can be stored. One program can consist of up to 50 steps.
	Standard solution name	Up to 100 solution names can be stored.
	Solution configuration name	Up to 50 solution configurations can be stored.
Drying station		Temperature setting range: 30 to 65°C
Heating station		- Temperature setting range: 30 to 70°C
Required facilities	Power supply	Voltage = Rated voltage ± 10%, 50/60 Hz Current capacity = Rated current min. Grounding = Comes with grounding bar conforming to class D grounding
	Water supply	Water = Tap water Pressure Dynamic water = 0.098 to 0.441 MPa Maximum static pressure = 0.74 MPa Temperature = 30°C max. (non-freezing) Nominal pipe diameter = 15A (1/2 in) min.
	Drain	Method = Indirect draining Drain capacity = 15 L per minute min. Nominal pipe diameter = 40A (1-1/2 in) min.
Operating environment	Ambient temperature	10 to 40°C
	Relative humidity	30 to 85% (non-condensing)
Storage environment	Ambient temperature	-10 to 65°C (instrument interior must be drained.)
	Relative humidity	20 to 90% (non-condensing)
Vibration/drop		Drop test JISZ0200:2013 (Level 1) Vibration test JISZ0232: 2004 (Standard of Truck Transportation)
Noise		65 dB or less (A characteristics)
Law and conforming standard	Law	CE Marking IVD Directive (98/79/EC) RoHS Directive (2011/65/EU)
	Electrical safety evaluation	CE Marking & IVD Directive EN 61010-1: 2010 EN 61010-2-010: 2003 EN 61010-2-101: 2002
	Risk management	EN 14971: 2012 EN 62366: 2008
	Medical equipment software	EN 62304: 2006
	Quality management	ISO 13485: 2004
	Electromagnetic compatibility	EN 61326-2-6: 2006

Specification "DRS-Prisma-P-JD/JS" "DRS-Prisma-P-JCS"


General name		Automated slide stainer	
Brand name		Tissue-Tek Prisma®Plus	
Model		Prisma-P-JD	Prisma-P-JS
Product code		6174	6175
Manufacturing license number		20B2X00014000034	
Dimensions		1250 (W) x 713 (D) x 650 (H) mm (excluding the control monitor)	
Weight		Approximately 150 kg	
Rated power supply		Single-phase 100 VAC ± 10%, 50/60 Hz, 10 Amps Single-phase 230 VAC ± 10%, 50/60 Hz, 5.5 Amps	
Power consumption		1000 VA	
Electric shock protection		Class I, overvoltage category II	
Pollution degree		Pollution degree I	
Capabilities	Stain process	Batch mode: 60, 20 or 10 slides Continuous mode: Up to 11 runs can be set in the instrument .	
	Staining programs	Up to 50 programs can be stored. One program can consist of up to 50 steps.	
	Standard solution name	Up to 100 solution names can be stored.	
	Solution configuration name	Up to 50 solution configurations can be stored.	
Drying station		Temperature setting range: 30 to 65°C	
Heating station		-	Temperature setting range: 30 to 70°C
Required facilities	Power supply	Voltage = Rated voltage ± 10%, 50/60 Hz Current capacity = Rated current min. Grounding = Comes with grounding bar conforming to class D grounding	
	Water supply	Water = Tap water Pressure Dynamic water = 0.098 to 0.441 MPa Maximum static pressure = 0.74 MPa Temperature = 30°C max. (non-freezing) Nominal pipe diameter = 15A (1/2 in) min.	
	Drain	Method = Indirect draining Drain capacity = 15 L per minute min. Nominal pipe diameter = 40A (1-1/2 in) min.	
Operating environment	Ambient temperature	10 to 40°C	
	Relative humidity	30 to 85% (non-condensing)	
Storage environment	Ambient temperature	-10 to 65°C (instrument interior must be drained.)	
	Relative humidity	20 to 90% (non-condensing)	
Vibration/drop		Drop test JISZ0200:2013 (Level 1) Vibration test JIS Z0232: 2004 (Standard of Truck Transportation)	
Noise		65 dB or less (A characteristics)	
Law and conforming standard	Applicable instruments	DRS-Prisma-P-JD/JS	DRS-Prisma-P-JCS
	Law	Law Regarding Ensuring of Quality, Effectiveness and Safety of Drugs, Medical Equipment, Etc., and Other	Pharmaceutical and Medical Device Act (Manufacturing and Sales Notification, Export Notification), CFDA, Ordinance on Supervision and Control of Medical Devices, Class 1 Equipment, Administrative Regulation Pertaining to Limitations of Use of Harmful Substances in Electrical and Electronic Products
	Electrical safety evaluation	Law Regarding Ensuring of Quality, Effectiveness and Safety of Drugs, Medical Equipment, Etc., and Other JISC 1010-1: 2014 JISC 1010-2-101: 2013	CFDA, Ordinance on Supervision and Control of Medical Devices, Class 1 Equipment GB 4793.1-2007 (IEC 61010-1: 2010) GB 4793.6-2006 (IEC 61010-2-010: 2003) YY 0648-2008 (IEC 61010-2-101: 2002)
	Risk management	JIST 14971: 2012	
	Medical equipment software	JIST 2304: 2012	
	Quality management	ISO 13485: 2005	
	Electromagnetic compatibility	JISC 1806-2-6: 2012	GB/T 18268.1-2010 (IEC 61326-1: 2005) GB/T 18268.26-2010 (IEC 61326-2-6: 2005)

Name of Each Part

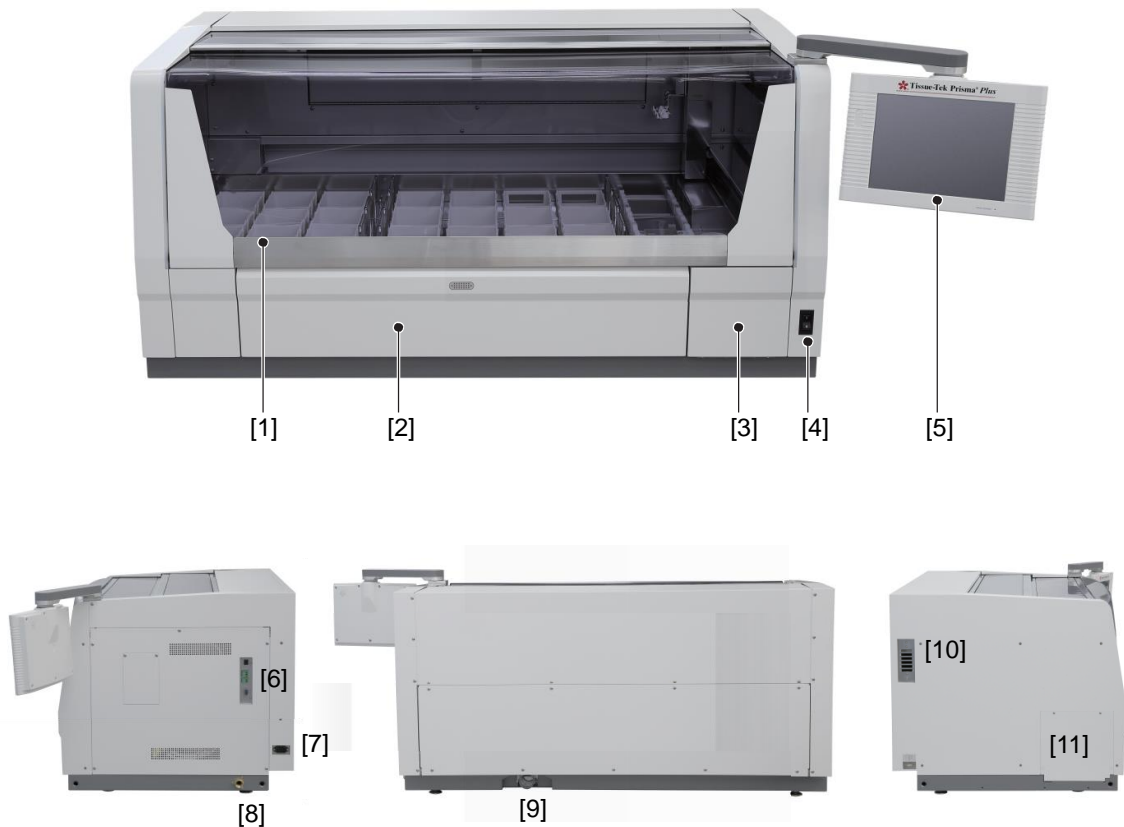

- [1] Cover**
The cover protects the operator against exposure to dangerous chemicals without obstructing his/her view.
- [2] Door**
Open this door when setting or removing tissues.
- [3] Water supply strainer door**
Access this area when maintaining the water supply circuit.
- [4] Power switch**
- [5] Control panel display**
A touch-panel type display for operating the instrument.
- [6] External interface**
It can be connected to an UPS (uninterruptible power supply), 2D bar code reader, etc.
- [7] Power-supply inlet**
- [8] Water supply port**
Tap water is supplied to this port.
- [9] Drain port**
- [10] Exhaust outlet**
This outlet can be connected to an external exhaust duct via a special attachment.
- [11] Automated coverslipper link port**
Remove the port cover to connect an automated coverslipper for automated operation encompassing staining through coverslipping.

Name of Each Part

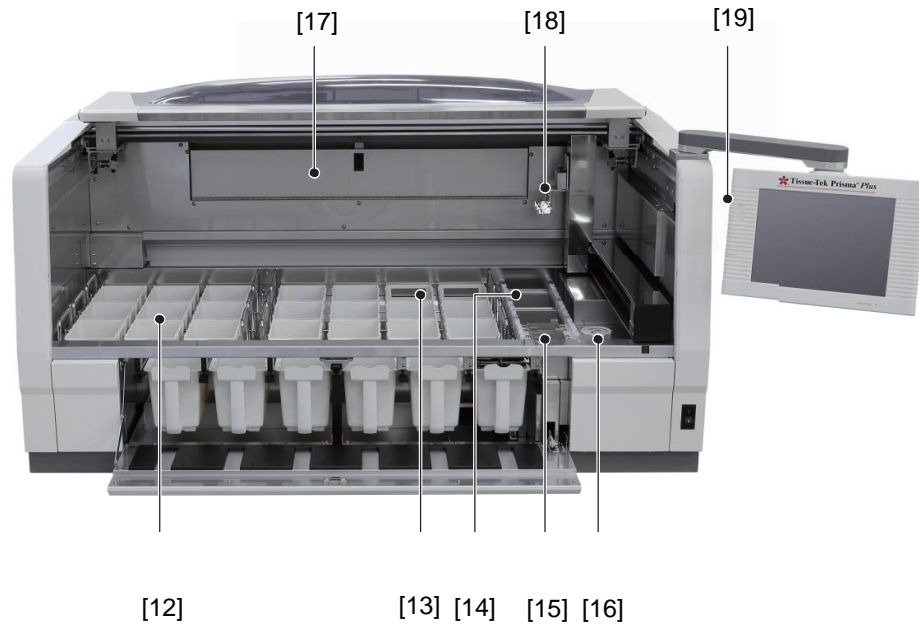

[12] Solution reservoir

[13] Wash station

Up to 4 wash stations.

[14] Drying station

[15] Heating station

Heating stations permit special stain runs that require heating.

[16] Water supply handle

Use this handle to adjust the water flow rate.

[17] Fume control unit

Activated carbon filters are part of the unit.


[18] Robotic arm

The function of robotic arms is to move slide basket.

[19] CF card slot

This sets a CF card in slot.

Standard Accessories/Options

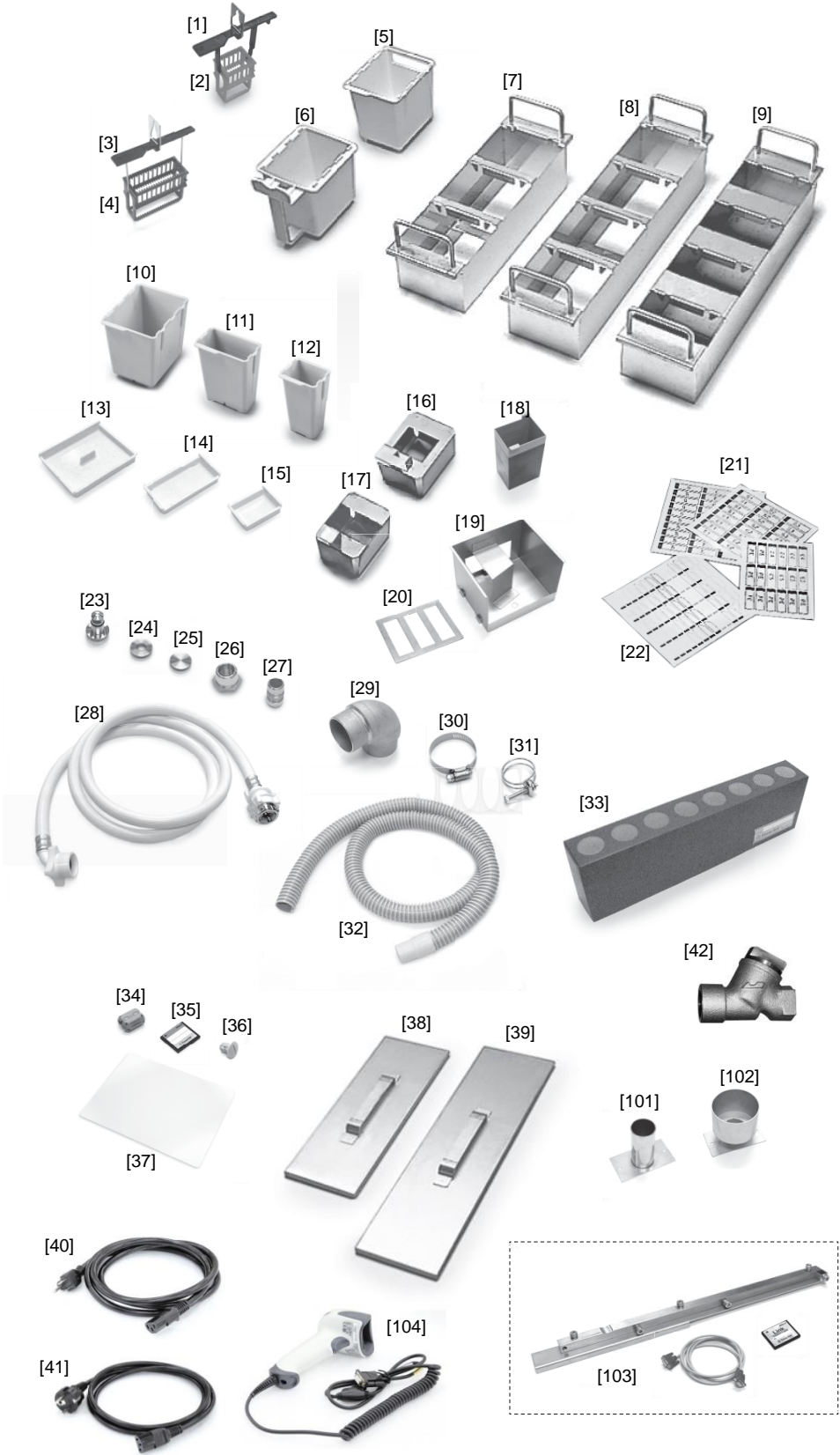

Standard Accessories/Options

No.	Product/part name	Product code	Product code	AD	AS	ED	ES	JD	JS	JCS		
				6170	6171	6172	6173	6174	6175	6177		
				Q'ty								
1	10-slide basket adapter	6138	K24-418-01	-	10	-	10	-	10	10		
2	10-slide basket	6137	K24-411-00	-	10	-	10	-	10	10		
3	20-slide basket adapter	6136	K24-408-01	10	10	10	10	10	10	10		
4	20-slide basket	4768	K16-500-08	10	10	10	10	10	10	10		
5	Wash reservoir	6149	K24-403-01	4	4	4	4	-	-	4		
6	Solution reservoir with handle	6148	K24-400-00	6	6	6	6	-	-	6		
7	3-position reservoir tray	6152	F6-02-7300	-	-	3 ⁽¹⁾	3 ⁽¹⁾	-	-	-		
8	4-position reservoir tray	6154	F6-02-7400	-	-	3 ⁽¹⁾	3 ⁽¹⁾	-	-	-		
9	Special stain reservoir tray	6156	K24-147-02	-	-	-	1	-	-	-		
10	Standard solution reservoir	6147	K24-401-01	22	22	22	22	-	-	22		
11	Small solution reservoir	-	K24-402-01	-	-	-	-	-	-	-		
12	Special stain solution reservoir	6140	K24-404-00	-	8	-	8	-	8	9		
13	Lid, standard solution reservoir	6151	K24-405-00	8	8	8	8	-	-	8		
14	Lid, small solution reservoir	6146	K24-406-00	-	-	-	-	-	-	-		
15	Lid, special stain reservoir	6144	K24-407-00	-	8	-	8	-	8	8		
16	10-slide basket load/unload adapter	6139	K24-488-03	-	3	-	3	-	3	3		
17	20-slide basket load adapter	6135	K24-491-01	-	-	-	-	-	-	-		
18	Heating solution reservoir	6141	K24-503-01	-	2	-	2	-	2	2		
19	Paraffin tray	-	K24-531-04	2	2	2	2	2	2	2		
20	Hot air outlet panel, drying station	-	K24-535-00	2	2	2	2	2	2	2		
21	Station labels (Start, End)	6161	F6-02-8400	1	1	1	1	1	1	1		
22	Basket adapter label	6162	K24-712-00	1	1	1	1	1	1	1		
23	Nozzle	-	K24-430-01	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾		
24	Wash cap	-	K24-433-00	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾		
25	Cap for covering water supply port	-	K24-437-00	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾	4 ⁽³⁾		
26	Nipple (1/2 inch)	-	A6-30-0040	1	1	2	2	1	1	2		
27	Bushing (1/2 x 3/4 inch)	-	A6-34-5160	-	-	1	1	1	1	1		
28	Water supply hose (faucet connection type)	-	B7-44-0030	-	-	1	1	1	1	1		
29	Drain elbow	-	K23-291-00	1	1	1	1	1	1	1		
30	Hose clamp	-	D9-00-0159	1	1	1	1	1	1	1		
31	Spiral wire band	-	D9-00-0146	1	1	1	1	1	1	1		
32	Drain hose unit (Ø38 mm x 2 m + cuffs)	-	K24-094-00	1	1	1	1	1	1	1		
33	Activated carbon filters	6160	K24-467-00	2	2	2	2	2	2	2		
34	Ferrite core	-	A3-33-6034	1	1	1	1	1	1	1		
35	CF card	6164	K24-718-00	1	1	1	1	1	1	1		
36	Bushing (bolt cap)	-	A4-07-0511	4	4	4	4	4	4	4		
37	Control panel protection sheet	6163	K24-640-00	1 ⁽²⁾	1 ⁽²⁾	1 ⁽²⁾	1 ⁽²⁾	1 ⁽²⁾	1 ⁽²⁾	1 ⁽²⁾		
38	Large lid, 3-position reservoir tray	6153	K24-423-03	-	-	3	3	-	-	-		
39	Large lid, 4-position reservoir tray	6155	K24-424-03	-	-	3	3	-	-	-		
40	Power cord	-	00000-0507	1	1	1	1	1	1	-		
41	Power cord	-	A4-01-0534	1	1	1	1	1	1	1		
42	Strainer	-	00000-3006	-	-	1	1	-	-	-		
-	Bushing (machined)	-	K23-440-01	1	1	-	-	-	-	-		
-	Street elbow (1/2 inch)	-	A6-31-3000	-	-	1	1	-	-	-		
-	Cap for LAN port	-	00000-0910	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾		
-	Cap for RS232C port	-	00000-0511	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾	1 ⁽⁵⁾		
-	3-position reservoir tray (2-phase stainless)	6180	A-MK25-U013-00-A	3 ⁽¹⁾	3 ⁽¹⁾			3 ⁽¹⁾	3 ⁽¹⁾	3 ⁽¹⁾		
-	4-position reservoir tray (2-phase stainless)	6182	A-MK25-U014-00-A	3 ⁽¹⁾	3 ⁽¹⁾			3 ⁽¹⁾	3 ⁽¹⁾	3 ⁽¹⁾		
-	Special stain reservoir tray (2-phase stainless)	6185	A-MK25-U020-00-A		1				1	1		
-	Large lid, 3-position reservoir tray (2-phase stainless)	6181	A-MK25-0620-01-A	3	3					3		
-	Large lid, 4-position reservoir tray (2-phase stainless)	6183	A-MK25-0622-01-A	3	3					3		
-	Installation Manual	-	-	1	1	1	1	1	1	1		
-	Operating Manual	-	-	1	1	1	1	1	1	1		
-	Package insert	-	-	-	-	-	-	1	1	-		
-	Warranty registration card	-	-	-	-	-	-	1	1	-		
No.	Optional or set	Product code	Product code	AD	AS	ED	ES	JD	JS	JCS		
38	Large lid, 3-position reservoir tray	6153	K24-423-03									
39	Large lid, 4-position reservoir tray	6155	K24-424-03									
101	Exhaust duct adapter, 38 mm dia.	-	K24-478-01									
102	Exhaust duct adapter, 75 mm dia.	-	K24-567-01									
103	Tissu-Tek Film link system	-	F6-02-8700									
104	2D bar code reader	-	1900GRS-1RS									
-	Large lid, 3-position reservoir tray (2-phase stainless)	6180	A-MK25-0620-01-A									
-	Large lid, 4-position reservoir tray (2-phase stainless)	6182	A-MK25-0622-01-A									
-	Tissu-Tek Glas-g2 link kit	-	K0000-1189									
-	Standard solution reservoir set	6157	F6-02-7500					1 ⁽⁴⁾	1 ⁽⁴⁾			
-	Small solution reservoir set	6158	F6-02-7600					1 ⁽⁴⁾	1 ⁽⁴⁾			
-	Special stain solution reservoir set	6159	F6-02-7700									
-	Special stain solution reservoir set (2-phase stainless)	6184	A-MK25-U015-00-A									

- * (1) Preinstalled inside the instrument.
(2) Pre-attached on the control panel screen.
(3) Of the four pieces, two are preinstalled on the instrument.
(4) Select either the standard solution reservoir set or small solution reservoir set.
(5) Pre-installed on each port.

Basics

Basics on This Instrument

This section explains the Basics of operating the instrument. For the details of each item, refer to the page indicated in the text.

Instrument Overview

- This instrument automatically stains tissues and cell smear samples on glass slides for microscopy, as part of histological studies and tests conducted in the fields of pathology, anatomy, clinical pathology, etc.
The instrument supports various types of staining such as H&E, Papanicolaou etc.
- This instrument is operated from the touch-panel type display located on the right side. This display allows the operator to easily operate the instrument according to visual cues by touching the necessary keys displayed on the screen. Staining data can also be output from a PC by using a special compact flash card (CF card).
- Two types of baskets can be used for storing slides: the 20-slide type for general staining and the 10-slide type for special staining
- Solution reservoirs are largely classified into three types: standard solution reservoir, small solution reservoir and special stain solution reservoir. The basket capacity and required solution volume of each solution reservoir are summarized below.

	Standard solution reservoir	Small solution reservoir	Special stain reservoir
20-slide basket	3 baskets	1 basket	-
10-slide basket	3 baskets	1 basket	1 basket
Required solution volume	680 ~ 820 mL	255 ~ 285 mL	160 ~ 180 mL

Stain Process Modes

This instrument performs stain process in the "continuous mode" and "batch mode." The two staining modes are explained below.

- **Continuous mode**
A different staining run can be started during the current staining run, with up to 11 staining runs performed simultaneously.
- **Batch mode**
In this mode, each stain process must be completed before the next sample can be stained. The start step and end step can be specified to start a staining run.

Refer to **P. 1-33** on how to set each mode.

User Management by Password

The instrument can limit the operating functions permitted to each user according to the user's password. Two types of passwords are provided. The "Administrator Password" is a higher-level password that permits all operations, while the "User Password" is a lower-level password that permits only the specific operations pre-defined for each user. Refer to **P. 1-39** for details on how to set a password, etc.

Basics

Logon/Logoff Function

The user enters his/her pre-assigned password to log on to the instrument, upon which the instrument becomes operational. Only the user who has logged on can operate the instrument. The operations that are permitted to the logged-on user vary depending on the level of the user's password. When the user logs off from the instrument during operation, unauthorized instrument operation by a third person can be prevented. The stain process is not affected by the logoff, and will continue until the end. To log on to the instrument again, the user enters a password using the Log On key in the monitor screen. The logon password need not be the same as the one that was used to log off from the instrument. The user can log on using any password currently stored in the instrument. During any operation other than stain process, the user can also freely log off from the instrument when leaving from the instrument for an extended period of time. This function protects the instrument against unauthorized operation by a third person.

Safety Mechanism of the Instrument

As a safety mechanism, the instrument is designed so that the arm will stop moving if the cover is opened during stain process. Although the staining time counter keeps counting while the cover is open, the baskets do not move because the arm is stopped. Staining can be resumed by performing the resumption operation after closing the cover. However, the scheduled completion of stain process may be delayed by up to the period during which the cover has been open. The arm will also stop if the front door is opened when a solution station in the frontmost row is being accessed. In this case, stain process can also be resumed by performing the resumption operation after closing the front door. However, the scheduled completion of staining process may be delayed by up to the period during which the door has been open.

How to Open and Close the Cover/Door

- **How to open the cover**
Hold the stainless part at the center of the cover, and raise the cover while pulling it toward you. As the cover rises, it automatically tilts forward. Push the cover into the back of the instrument, and the cover will be stored into the ceiling of the instrument.
- **How to close the cover**
Hold the stainless part at the center of the cover and pull it toward you. Once the cover has been fully drawn out of the ceiling of the instrument, the stainless part starts tilting down. When the cover drops to the bottom, push the stainless part into the instrument to securely close the cover.
- **How to open the door**
Touch the center of the door once, and the door will open toward you.
- **How to close the door**
Raise the center of the door and push in the door until a click sound is heard in the reverse order of door opening.

Basics

Handling the Touch-panel Display


The touch-panel display used for operating the instrument is movable. To move the display, do so slowly by holding both ends of the display. Note that the moving range of the display is limited. Do not move the display beyond the specified range or apply any excessive force, as it may cause the display to break. A protection sheet is attached on the control panel. A protection sheet that has become dirty, damaged or peeled must be replaced.



Handling the CF Card Slot

A CF card slot for setting a CF card is provided on the left side of the touch-panel display used for operating this instrument. You can save staining and other data stored in the instrument to a PC, etc., using a CF card.

- Set a CF card into the CF card slot from the side with an arrow, with the other side on which a label is attached facing toward you.
- Be sure to wait for the confirmation message to appear on the screen before removing the CF card.
- Sakura does not warrant the integrity of data stored in CF cards regardless of the failure exhibited by the instrument or monetary damage suffered in connection with the data loss/damage. Sakura does not provide any service associated with the recovery or restoration of data in CF cards.
- CF cards other than those specified for the instrument may not be supported. CF cards must be FTP-formatted using Microsoft Windows.

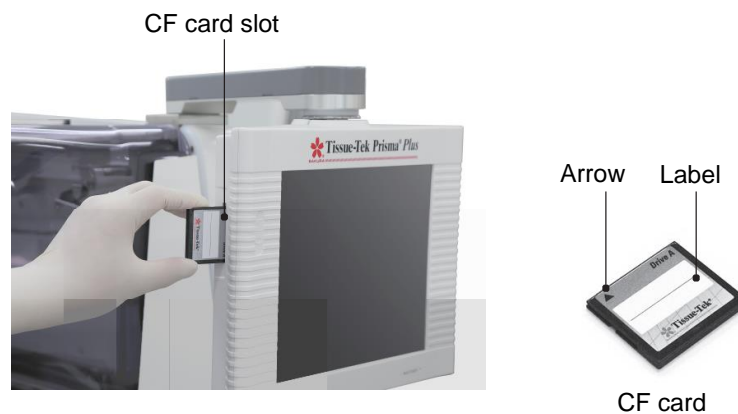

Basics

Various Gateways

External signal I/O connections for various gateways are provided on the right side of the instrument. You can connect the instrument to an uninterruptible power supply, alarm output, LAN, and bar code reader from these gateways. Each gateway is explained below.



- LAN connection
- External alarm
- External alarm output
- Power-outage detection signal input
- Bar code reader connection

- **Power-outage detection signal**

You can connect an uninterruptible power supply (UPS) to the instrument's power supply so that tissues can be preserved and evacuated in case of power outage. When installing an UPS, be sure to use a model with an output capacity equivalent to or greater than the input rating of the instrument.

Recommended UPS rating: Capable of outputting 15 kVA for 5 minutes

Some UPSs are not supported by the instrument. For details, contact the Sakura Finetek Technical Support representative or local representative. The instrument provides an UPS input signal to detect a power outage mode of the UPS connected to the power supply. Use an UPS equipped with a contact-output interface or open-collector output that remains open in normal state and shorts upon power outage. Always turn off the instrument power before connecting an UPS.


* When connecting the power-outage detection signal to an external circuit, loop the signal cable around the attached ferrite core at a point as close as possible to the instrument, as shown below. If the ferrite core is not installed, the instrument may malfunction due to external noise.

To the instrument's power-outage detection signal input

To the UPS's power-outage detection signal output



When the ferrite core is installed

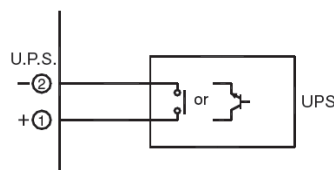


Example of power-outage signal connection

Basics

- **Alarm output**

This is a no-voltage contact output used to retrieve an alarm from the instrument in case of an error that stops the staining operation (error other than battery low, exhaust fan and heating/drying station errors). The rated voltage of the connected load must not exceed 30 VDC, with the rated current kept to 1 A or below. Always turn off the instrument power before connecting the alarm output. For details, contact the Sakura Finetek Technical Support representative or local representative.

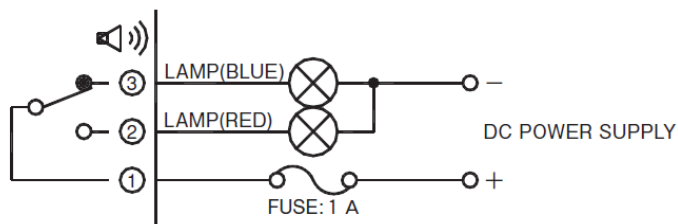

Example of external alarm connection

- **EMC**

This instrument conforms to the emission and immunity requirements specified in EN61326-2-6 or JISC1806-2-6.

- **Ethernet port (LAN connection)**

Special settings are needed to connect the instrument to a local area network. Should you require LAN connection, consult your system administrator. For details, contact the Sakura Finetek Technical Support representative or local representative.

Specification: 10BASE-T

- **2D bar code reader (optional)**

By connecting the 2D bar code reader, the instrument can read the bar codes on Sakura staining solutions*. Their expiration dates can be entered in the system based on the bar code information.

* Check the local Sakura Finetek website for availability of the Sakura staining solutions and staining kits.

Basics

Relationship of Solution Configurations and Station Numbers

This section explains the relationship of solution configurations and station numbers shown on the screen. The link station is available only when the instrument is connected to an automated coverslipper. It cannot be used if an automated coverslipper is not connected.

• Arrangement of small solution stations

The small solution stations can be arranged "without special solution stations" or "with special solution stations." In both cases, the solution configurations and number of stations are the same. However, the number of stations and station numbers vary depending on whether some stations are used as wash stations or solution stations.

Row 4	Station 1	Station 3	Station 5	Station 7	Station 8	Station 9	Station 10	Station 13
	Station 2	Station 4	Station 6					Station 14
Row 3	Station 15	Station 17	Station 19	Station 21	Station 23	Station 25	Station 27	Station 29
	Station 16	Station 18	Station 20	Station 22	Station 24	Station 26	Station 28	Station 30
Row 2	Station 31	Station 33	Station 35	Station 37	Station 39	Station 41	Station 43	Station 45
	Station 32	Station 34	Station 36	Station 38	Station 40	Station 42	Station 44	Station 46
Frontmost row	Station 53	Station 47	Station 48	Station 49	Station 50	Station 51	Station 52	

Figure Arrangement with/without Special Solution Stations

Fixed station numbers are assigned to the start stations (S1 to S3), end stations (linked) (E1-1 to E1-5), end stations (not linked) (E2-1 to E2-5), wash stations (W1 to W4), drying stations (D1A, D1B, D2A, D2B), heating stations (H1, H2) and link station (LINK). The tables below specify the identification number of each station attribute and the corresponding station number.

Start station	S1	S2	S3
Without special solution stations	Station 52	Station 51	Station 50
With special solution stations			

* S2 and S3 can also be set as solution stations.

End station (linked)	E1-1	E1-2	E1-3	E1-4	E1-5
Without special solution stations	Station 47	Station 48	Station 49	Station 50	Station 51
With special solution stations					

* E1-2 to E1-5 can also be set as solution stations.

PE station
The rightmost end station is selected as the PE station

End station (not linked)	E2-1	E2-2	E2-3	E2-4	E2-5
Without special solution stations	Station 47	Station 48	Station 49	Station 50	Station 51
With special solution stations					

* E2-2 to E2-5 can also be set as solution stations.

Link station	LINK
Without special solution stations	Station 53
With special solution stations	

* The link station can be set only when the instrument is connected to an automated coverslipper.

Wash station	W1	W2	W3	W4
Without special solution stations	Station 4	Station 5	Station 6	Station 7
With special solution stations				

* W1 to W4 can also be set as solution stations.

Drying station	D1A	D1B	D2A	D2B
Without special solution stations	Station 13	Station 14	Station 29	Station 30
With special solution stations				

Heating station	H1	H2
Without special solution stations	Station 45	Station 46
With special solution stations		

* Available with the DRS-Prisma-J0S only.

Special solution station	SP1	SP2	SP3	SP4	SP5	SP6	SP7	SP8
Without special solution stations	-	-	-	-	-	-	-	-
With special solution stations	Station 21	Station 22	Station 23	Station 24	Station 25	Station 26	Station 27	Station 28

Basics

- **Arrangement of standard solution stations (with drying stations)**

In the standard mode, the arrangement and number of stations vary depending on whether the solution stations are arranged "without special solution stations" or "with special solution stations."

Row 4	Station 1	Station 2	Station 3	Station 4	Station 5	Station 6	Station 7	Station 8
Row 3	Station 9	Station 10	Station 11	Station 12	Station 13	Station 14	Station 15	Station 16
Row 2	Station 17	Station 18	Station 19	Station 20	Station 21	Station 22	Station 23	Station 24
Frontmost row	Station 31	Station 25	Station 26	Station 27	Station 28	Station 29	Station 30	

Figure Arrangement without Special Solution Stations

Row 4	Station 1	Station 2	Station 3	Station 4	Station 5	Station 6	Station 7	Station 8
Row 3	Station 9	Station 10	Station 11	Station 12	Station 14	Station 16	Station 18	Station 20
Row 2	Station 21	Station 22	Station 23	Station 24	Station 25	Station 26	Station 27	Station 28
Frontmost row	Station 35	Station 29	Station 30	Station 31	Station 32	Station 33	Station 34	

Figure Arrangement with Special Solution Stations

Fixed station numbers are assigned to the start stations (S1 to S3), end stations (linked), (E1-1 to E1-5), end stations (not linked) (E2-1 to E2-5), wash stations (W1 to W4), drying stations (D1, D2) and link station (LINK). The tables below specify the identification number of each station attribute and the corresponding station number. The rightmost end station is selected as the PE station.

Start station	S1	S2	S3
Without special solution stations	Station 30	Station 29	Station 28
With special solution stations	Station 34	Station 33	Station 32

* S2 and S3 can also be set as solution stations.

End station	E1-1	E1-2	E1-3	E1-4	E1-5
Without special solution stations	Station 25	Station 26	Station 27	Station 28	Station 29
With special solution stations	Station 29	Station 30	Station 31	Station 32	Station 33

* E1-2 to E1-5 can also be set as solution stations.

End station (not linked)	E2-1	E2-2	E2-3	E2-4	E2-5
Without special solution stations	Station 25	Station 26	Station 27	Station 28	Station 29
With special solution stations	Station 29	Station 30	Station 31	Station 32	Station 33

* E2-2 to E2-5 can also be set as solution stations.

Link station	LINK
Without special solution stations	Station 31
With special solution stations	Station 35

* The link station can be set only when the instrument is connected to an automated coverslipper.

Wash station	W1	W2	W3	W4
Without special solution stations	Station 7	Station 6	Station 5	Station 4
With special solution stations	Station 7	Station 6	Station 5	Station 4

* W1 to W4 can also be set as solution stations.

Drying station	D1	D2
Without special solution stations	Station 8	Station 16
With special solution stations	Station 8	Station 20

* When a drying station unit is installed.

Special solution station	SP1	SP2	SP3	SP4	SP5	SP6	SP7	SP8
Without special solution stations	-	-	-	-	-	-	-	-
With special solution stations	Station 12	Station 13	Station 14	Station 15	Station 16	Station 17	Station 18	Station 19

Basics

- **Arrangement of standard solution stations (with drying stations)**

In the standard mode, the arrangement and number of stations vary depending on whether the solution stations are arranged "without special solution stations" or "with special solution stations."

Row 4	Station 1	Station 2	Station 3	Station 4	Station 5	Station 6	Station 7	Station 8
Row 3	Station 9	Station 10	Station 11	Station 12	Station 13	Station 14	Station 15	Station 16
Row 2	Station 17	Station 18	Station 19	Station 20	Station 21	Station 22	Station 23	Station 24
								Station 25
Frontmost row	Station 32	Station 26	Station 27	Station 28	Station 29	Station 30	Station 31	

Figure Arrangement without Special Solution Stations

Row 4	Station 1	Station 2	Station 3	Station 4	Station 5	Station 6	Station 7	Station 8
Row 3	Station 9	Station 10	Station 11	Station 12	Station 14	Station 16	Station 18	Station 20
				Station 13	Station 15	Station 17	Station 19	
Row 2	Station 21	Station 22	Station 23	Station 24	Station 25	Station 26	Station 27	Station 28
								Station 29
Frontmost row	Station 36	Station 30	Station 31	Station 32	Station 33	Station 34	Station 35	

Figure Arrangement with Special Solution Stations

Fixed station numbers are assigned to the start stations (S1 to S3), end stations (linked) (E1-1 to E1-5), end stations (not linked) (E2-1 to E2-5), wash stations (W1 to W4), drying stations (D1, D2), heating stations (H1, H2) and link station (LINK). The tables below specify the identification number of each station attribute and the corresponding station number. The rightmost end station is selected as the PE station.

Start station	S1	S2	S3
Without special solution stations	Station 31	Station 30	Station 29
With special solution stations	Station 35	Station 34	Station 33

* S2 and S3 can also be set as solution stations.

End station (linked)	E1-1	E1-2	E1-3	E1-4	E1-5
Without special solution stations	Station 26	Station 27	Station 28	Station 29	Station 30
With special solution stations	Station 30	Station 31	Station 32	Station 33	Station 34

* E1-2 to E1-5 can also be set as solution stations.

End station (not linked)	E2-1	E2-2	E2-3	E2-4	E2-5
Without special solution stations	Station 26	Station 27	Station 28	Station 29	Station 30
With special solution stations	Station 30	Station 31	Station 32	Station 33	Station 34

* E2-2 to E2-5 can also be set as solution stations.

Link station	LINK
Without special solution stations	Station 32
With special solution stations	Station 36

* The link station can be set only when the instrument is connected to an automated coverslipper.

Wash station	W1	W2	W3	W4
Without special solution stations	Station 7	Station 6	Station 5	Station 4
With special solution stations	Station 7	Station 6	Station 5	Station 4

* W1 to W4 can also be set as solution stations.


Drying station	D1	D2
Without special solution stations	Station 8	Station 16
With special solution stations	Station 8	Station 20

Heating station	H1	H2
Without special solution stations	Station 24	Station 25
With special solution stations	Station 28	Station 29


Special solution station	SP1	SP2	SP3	SP4	SP5	SP6	SP7	SP8
Without special solution stations	-	-	-	-	-	-	-	-
With special solution stations	Station 12	Station 13	Station 14	Station 15	Station 16	Station 17	Station 18	Station 19

Explanation of Menu Screens

Stain Process Menu

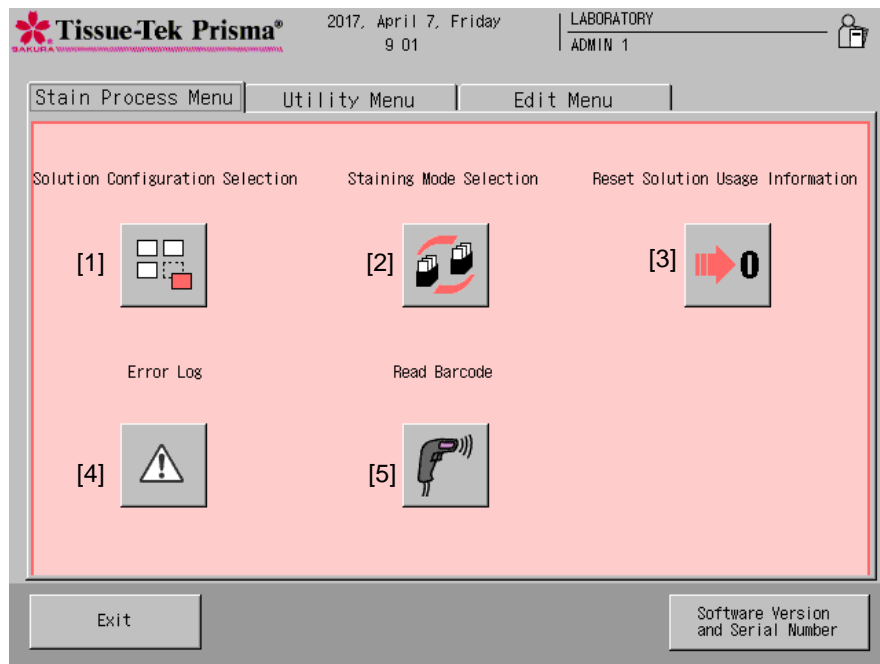

This menu screen appears while the instrument is standing by after logon.

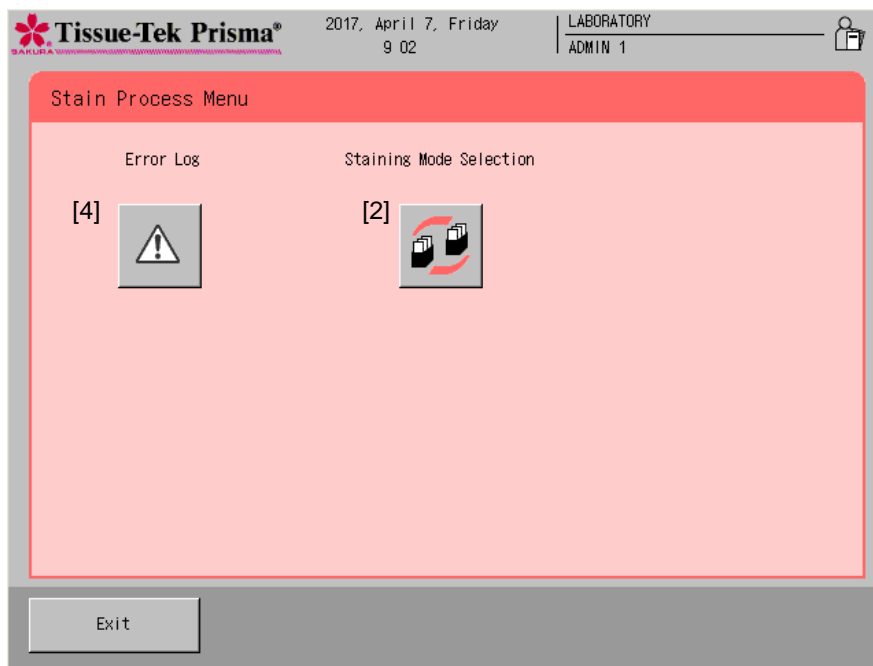

This menu screen appears during stain process.

Explanation of Menu Screens

Explanation of Icons in the Stain Process Menu

[1] Solution Configuration Selection

Touch this icon to switch the solution configuration used for staining. Select a desired solution configuration from the stored configurations and apply it to the instrument.

[2] Staining Mode Selection

Touch this icon to set the following items regarding the starting of stain process. Once set, the staining mode (continuous/batch) cannot be changed during stain process. Other settings can be changed during stain process.

- **Staining mode selection**
Set whether to perform stain process in the batch mode or continuous mode.
- **Initialization of Start Staining Screen**
Set when to show the start staining screen. Select **Opening the door** or **Touching START button**.
- **Initialization of Staining Process Start**
Set when to start staining. Select **Closing the door**, **Touching START button** or **Touching START button after closing the door**.



When the Closing the door is selected, be careful not to let stain process start inadvertently.

- **Start Confirmation**
Set whether or not to show a confirmation message at the start of staining.
- **Process Start Notice**
Set whether or not to notify the start of stain process by showing the start screen once staining process has started.
- **Heating Station Always Heated**
When heating stations are used, set whether or not to always heat the heating stations.
- **Solution Name Display**
Set whether or not to display the applicable solution name (abbreviation) on each station in the monitor screen showing a solution configuration.
- **Link with Coverslipper**
Set whether to connect or disconnect the communication link with the connected coverslipper when the link is set in the System Setup screen (**P. 1-36**). Even if the instrument is linked to a coverslipper, slides are not transferred to the coverslipper if "Not in Use" is shown here. This item is displayed only when the instrument is linked to an automated coverslipper.

[3] Reset Solution Usage Information

Touch this icon to reset the usage limit in runs/days/slides for each solution station whose usage information is managed.

[4] Error Log

Touch this icon to check the error history and error descriptions or clear errors.


[5] Read Barcode (Optional)

Use this setting to read the 2D bar codes on Sakura staining solutions* and enter their expirations in the instrument based on the 2D bar code information.

- * Check the local Sakura Finetek website for availability of the Sakura staining solutions and staining kits.

Explanation of Menu Screens

Utility Menu

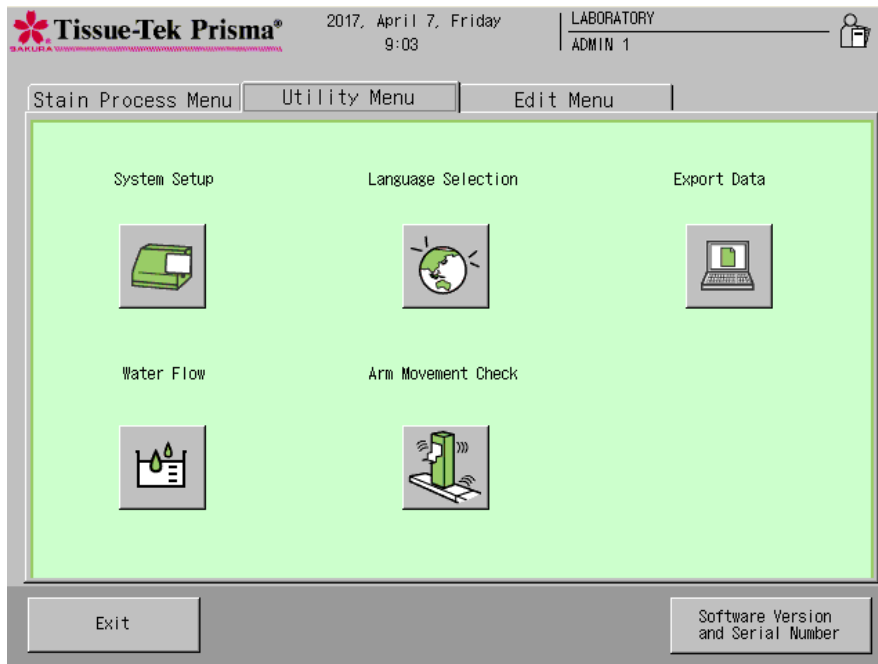

Explanation of Icons in the Utility Menu

[1] System Setup

Touch this key to set date/time, set date/time formats, change passwords, set filter use limits, set the institution name, set whether to enable or disable data export, adjust the alarm sound, and set whether or not to output a key input sound. Touching this key displays the System Setup screen. (P. 1-36)

[2] Language Selection

Touch this key to switch the display language.

[3] Export Data

Select this key to export process reports, solution configurations, staining programs, solution configuration name list, staining program name list, solution name list, system setup data and solution usage status. (P. 1-43)

[4] Water Flow


Select this key to adjust the water flow rate to the wash stations. (P. 1-42)

[5] Arm Movement Check

Arm movement check is an operation to check for obstacles inside the moving range of the arm. If the trays and solution reservoirs are not set correctly, the arm may contact them and cannot operate correctly. If a problem is found in this check, set the accessories again correctly.

Explanation of Menu Screens

Edit Menu

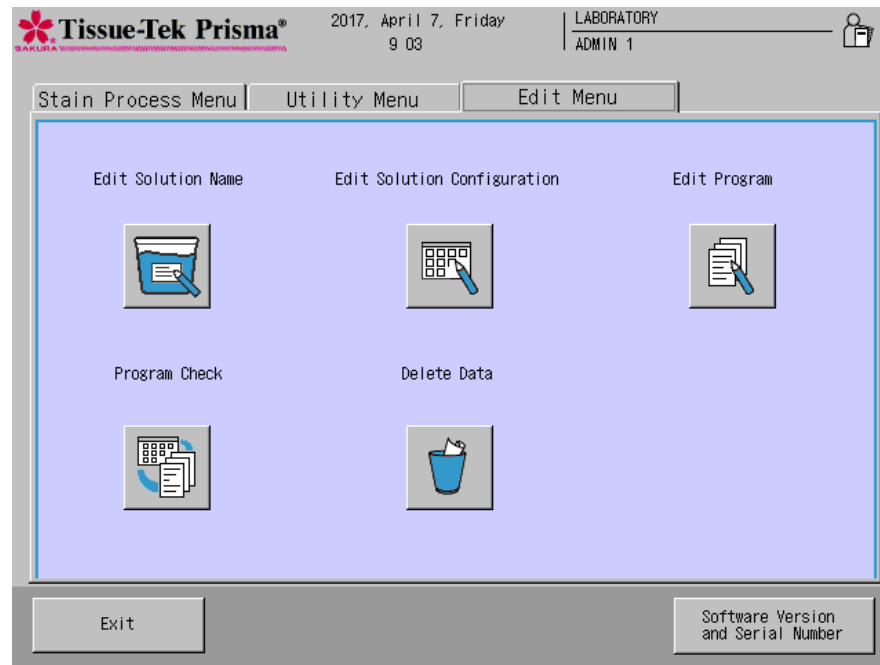

Explanation of Icons in the Edit Menu

[1] Edit Solution Name

Select this icon to create a new solution name or edit an existing solution name. (P. 2-23)

[2] Edit Solution Configuration

Select this icon to create a new solution configuration or edit an existing solution configuration. (P. 2-15)

[3] Edit Program

Select this icon to create a new staining program or edit an existing staining program. (P. 2-25)

[4] Program Check

Select this icon to check the solution configurations with which a given staining program can be run. (P. 2-33)


[5] Delete Data

Select this icon to delete solution configurations, solution names, staining programs and process reports. (P. 2-44)

System Setup

System Setup

In the System Setup screen accessible from the Utility Menu, you can define the basic settings needed to use the instrument.

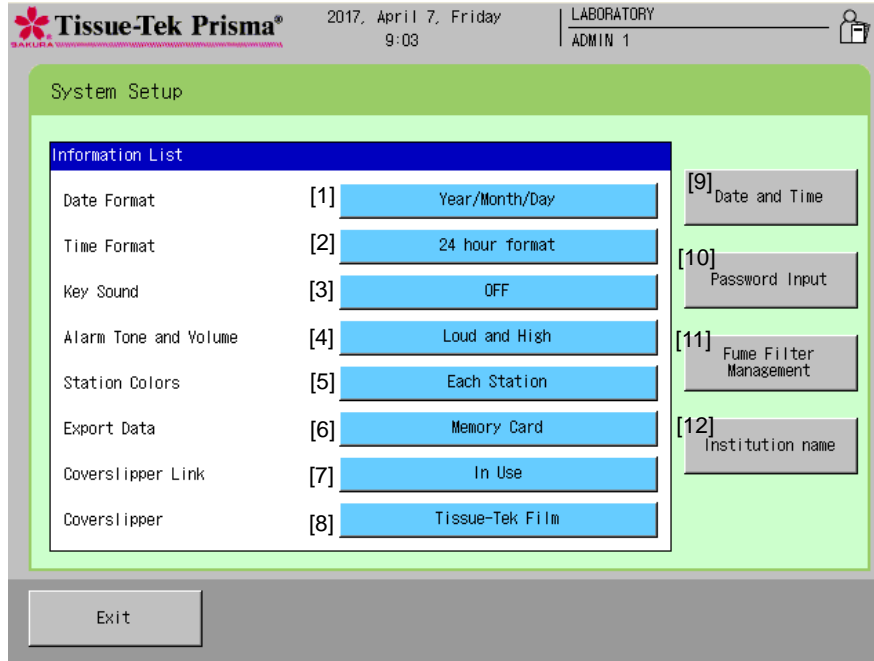

System Setup

[1] Date Format

This sets the format for the date display. Touching the option button toggles the display format in the sequence of "Year/Month/Day," "Month/Day/Year" and "Day/Month/Year."

[2] Time Format

This sets the format for the time display. Touching the option button switches the time format between "12 hour format" and "24 hour format."

[3] Key Sound

This selects whether to turn "ON" or "OFF" the sound output when a key is touched.

[4] Alarm Tone and Volume

This sets the tone and volume for system alarms. Touching the button toggles label on the button "Loud and High," "Loud and Low," "Soft and High" and "Soft and Low."

[5] Extended Station Colors

Touching the button toggles label on the button between "Each Solution Name" and "Each Station".

[6] Export Data

This sets whether or not exporting data is possible and the destination path for data export. Touching the button toggles label on the button between "File" and "OFF."

[7] Coverslipper Link

This button is used to set whether or not to enable the Coverslipper link.
This item is displayed only when the instrument is linked to an automated coverslipper.

[8] Coverslipper

If the Coverslipper link is installed, Coverslipper can be selected.

[9] Date and Time

You can set date and time. (P. 1-38)

[10] Password Input

You can manage user IDs and passwords. (P. 1-39)

[11] Fume Filter Management

You can manage filter use limits. (P. 1-41)

[12] Institution Name

You can set the name of the institution where the instrument is used.
Touching the **Institution Name** key displays the simple keyboard. Enter the applicable institution name and then touch the **Save** key to save the name. An institution name can be entered using up to 20 characters. Touch the **Caps Lock** key to enter characters in uppercase. Touch the key again to enter characters in lowercase.

System Setup

Setting Date and Time

You can set date and time by touching **Date and Time** from the System Setup screen accessible from the Utility Menu.

Entering Year

Touching the **Modify** button on the right of "Year" displays the numeric keypad. Enter the current year using the numeric keypad. You can enter a year in the range of 2000 to 2099.

Entering Month

Every time the ▲ or ▼ key is touched on the right of "Month," the selection changes to the next month (▲) or previous month (▼).

Entering Day

Select a desired day by touching the corresponding number in the calendar.

Entering Hours and Minutes

You can enter hours and minutes using the numeric keypad displayed by touching the **Modify** button below each item. Follow the time format selected by touching option button of "Time Format" (12 hour format/24 hour format) from the System Setup screen.

am/pm

You can select am or pm only when the 12 hour format is currently selected.

After all items have been entered, touching the **Save** key to apply the settings.

Tissue-Tek Prisma® 2017, April 7, Friday 9:03 LABORATORY ADMIN 1

Date and Time

Date Setup		Time Setup					
Year:	2017 <input type="button" value="Modify"/>	9	: 03				
Month:	April ▲ ▼	<input type="button" value="Modify"/>	<input type="button" value="Modify"/> <input type="button" value="am/pm"/>				
SUN	MON	TUE	WED	THU	FRI	SAT	
28	1	2	3	4	5	6	
7	8	9	10	11	12	13	
14	15	16	17	18	19	20	
21	22	23	24	25	26	27	
28	29	30	28	28	28	28	
28	28	28	28	28	28	28	

System Setup

Setting User IDs, Passwords and Accessible Functions

You can set user IDs and passwords as well as the functions accessible by each user by selecting **Password Input** from the System Setup screen accessible from the Utility Menu. To set each item, select the applicable user and perform the corresponding operation explained below.

Setting an ID


You can change the existing ID of a user. Touching the **User's ID** key displays the simple keyboard. Enter a new user ID, and then touch the **Save** key to save the ID. A user ID can be entered using up to 20 characters. Touch the **Caps Lock** key to enter characters in uppercase. Touch the key again to enter characters in lowercase.

Setting a Password

You can change a password only when you have logged on as an Administrator. Select the user whose password you want to set, touch the **Password Setup** key, and then enter a new password consisting of one to six digits on the numeric keypad. The entered password is shown in asterisks (*) in the password field. After the password has been entered, touch the **Enter** key. When the screen prompts you to enter the password again for confirmation, enter the same password and touch the **Enter** key again to save the password.

Up to four "Administrator" passwords and 20 "User" passwords can be saved. The factory settings are shown below.


Level	Factory-set IDs	Factory-set passwords
Administrator	[ADMIN 1] ~ [ADMIN 4]	[100000] ~ [400000]
User	[OPERATOR 1] ~ [OPERATOR	[000001] ~ [000020]

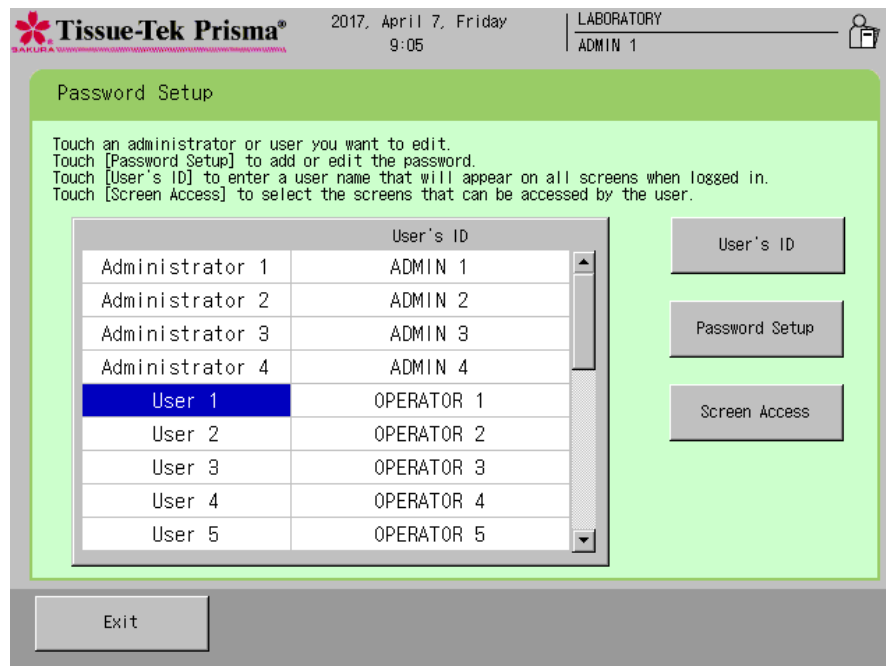

System Setup

Setting Accessible Functions

You can set functions accessible by each user. This operation cannot be performed for "Administrators."

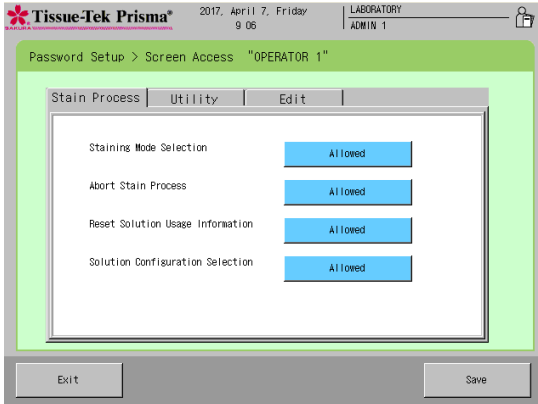

Stain Process

■ Staining Mode Selection

Touching this function changes permissions for "Staining Mode Selection" between **Allowed** and **Not Allowed**.

■ Abort Stain Process


Touching this function changes permissions for "Abort Stain Process" between **Allowed** and **Not Allowed**.

■ Reset Solution Usage Information

Touching this function changes permissions for "Reset Solution Usage Information" between **Allowed** and **Not Allowed**.

■ Solutions Configuration Selection

Touching this function changes permissions for "Solutions Configuration Selection" between **Allowed** and **Not Allowed**.

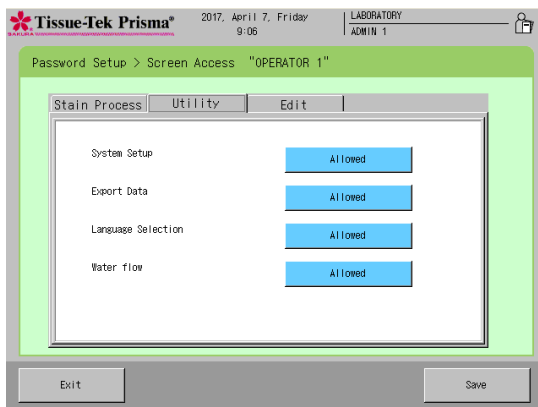

Utility

■ System Setup

Touching this function changes permissions for "System Setup" between **Allowed** and **Not Allowed**.

■ Export Data


Touching this function changes permissions for "Export Data" between **Allowed** and **Not Allowed**.

■ Language Selection

Touching this function changes permissions for "Language Selection" between **Allowed** to **Not Allowed**.

■ Water Flow

Touching this function changes permission for "Water Flow" between **Allowed** to **Not Allowed**.

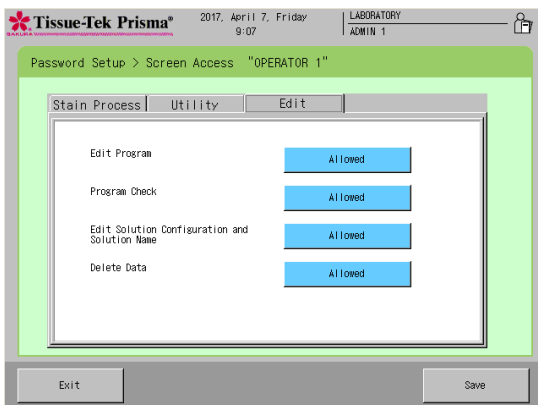

Edit

■ Edit Program

Touching this function changes permissions for "Edit Program" between **Allowed** to **Not Allowed**.

■ Program Check

Touching this function changes permissions for "Program Check" between **Allowed** to **Not Allowed**.

■ Edit Solution Configuration and Solution Name

Touching this function changes permissions for "Edit Solution Configuration and Solution Name" between **Allowed** to **Not Allowed**.

■ Delete Data

Touching this function changes permissions for "Delete Data" between **Allowed** to **Not Allowed**.

System Setup

Fume Filter Management

You can set items relating to fume filter management by selecting **Fume Filter Management** in the System Setup screen accessible from the Utility Menu.

Whether or Not to Use Fume Filter Management

You can set whether or not to manage the use of fume filters. Touch the button to change between "In Use" and "Not in Use."

Setting a Time Limit

Selecting "In Use" enables to touch the **Set Time Limit** key. When the **Set Time Limit** key is touched, you can set a time limit for fume filters. Touching this key displays the numeric keypad, so enter a desired time limit. To reset the entered time limit, touch the **Clear** key. A time limit can be set in the range of 1 to 999 hours in units of "hours." The default is "240" hours. After you have entered the time limit, touch the **Enter** key to save the entry.

To maintain a clean work environment, it is recommended that the activated carbon filters (each set consists of two filters) be replaced at least once every two weeks.

Resetting the Actual Time

Touch the **Reset Actual Time** button to initialize the present counter value of filter usage time. Touching the button resets the actual time currently stored.

When all items have been set, touch the **Exit** key to return to the System Setup screen.

The screenshot shows the 'Fume Filter Management Setup' screen. At the top, there is a header with the Tissue-Tek Prisma logo, the date '2017, April 7, Friday 9 07', and the user 'LABORATORY ADMIN 1'. The main area is titled 'Fume Filter Management' and contains a 'Fume Filter Management Setup' section. This section has a blue bar with 'Not In Use' and a 'Set Time Limit' button. Below this, there are two input fields: 'Usage Time Limit' set to '0:00' and 'Actual Time' set to '59:38'. To the right of these fields are 'Set Time Limit' and 'Reset Actual Time' buttons. At the bottom of the screen is an 'Exit' button.

System Setup


Adjusting the Water Flow

You can change the run-up time for water supply to the wash stations by selecting **Water Flow** from the Utility Menu.



- **Water supply run-up time** --- The time at which water supply to the wash stations is started during stain process, prior to the transfer of tissues into the wash stations.

Change the setting whenever necessary before the start of staining, after changing the solution configuration or staining program, and so on. The water supply run-up time must be changed after the wash solution configuration has been changed. The water flow can be adjusted with the cover open.

1. Confirm that water has been drained from all wash stations.
2. Open the water supply handle.
3. Touch the **Start** key and wait for water to overflow from the wash stations.
4. Once water starts to overflow from the wash stations, touch the **Stop** key. Adjust the water supply handle so that water will overflow within 20 to 30 seconds.
5. The water supply run-up time you have just set is displayed and the save screen appears. Touch **Save** if you want to use this run-up time. Touching **Exit** returns to the Water Flow screen.

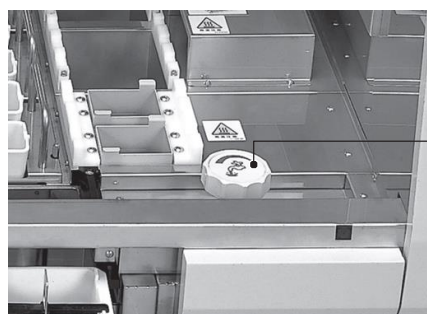


Water supply handle

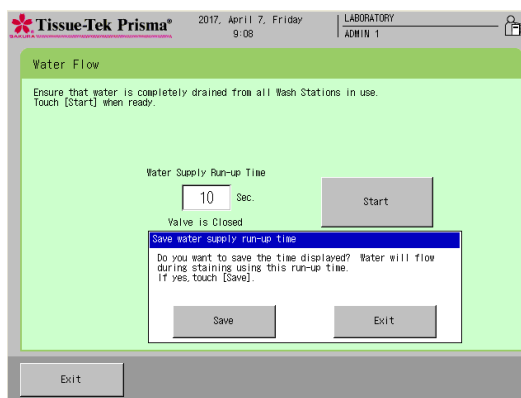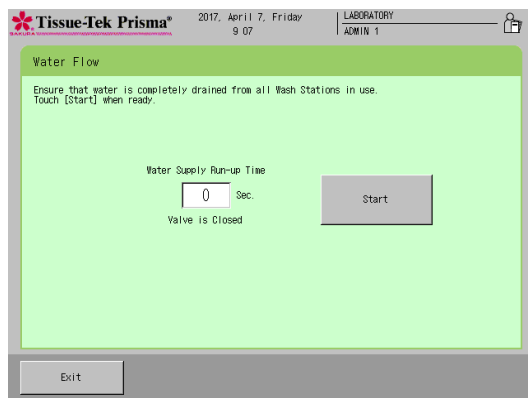

Export Data

About Exporting Data

You can export the following items by selecting **Export Data** from the **Utility Menu**. If the output destination is set to "OFF," export operation is disabled.

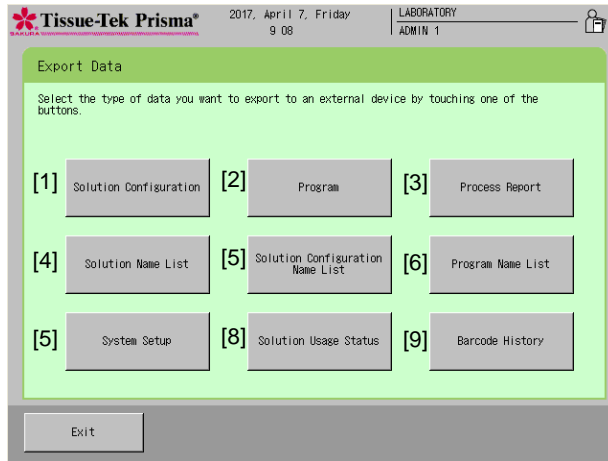

[1] Solution Configuration (Solution Configuration List)

Touch this button to export the details of solution configurations created by the user.

[2] Stain Program (Stain Program List)

Touch this button to export the details of staining programs created by the user.

[3] Process Result (Process Report)

Touch this button to export the results of staining runs. If an error or other disturbance occurred during stain process, information of the applicable error is also exported.

[4] Solution Name List

Touch this button to export a list of all solution names created by the user.

[5] Solution Configuration Name List

Touch this button to export a list of names of all solution configurations created by the user.

[6] Stain Program Name List

Touch this button to export a list of names of all staining programs created by the user.

[7] System Setup (System Setup List)

Touch this button to export the details of system setup saved by the user.

[8] Solution Usage Status (Solution Usage List)

Touch this button to export the solution usage status for the stations managed by the user.

[9] Barcode History

A list of up to 1,000 sets of bar code information read by the user can be exported to an external file.

Note An example of data export is shown on P. 2-40. The heading in the parenthesis is the title when exported..

Names of Files Exported to a CF Card

If the export destination is set to File, each set of output data is saved to a CF card as a file. The files of each exported item are named in the applicable format shown below.

Exported item	File name
Solution configuration	CL****.txt
Staining program	PL****.txt
Process report	REP****.txt
Solution name	SNL****.txt
Solution configuration name	CNL****.txt
Staining program name	PNL****.txt
System setup	ISL****.txt
Solution usage status	SUL****.txt
Bar code history	BCH****.txt



Chapter 2
Operating Procedure



Operating Procedure

Flow of Stain Process Operation

This section explains a series of operating procedures.

Flow of Stain Process Operation

A basic flow of normal operation is shown below.

• Preparation for Operation Standby

Refer to P. 2-2 for details.

- Start the water supply.
- Turn on the power and log on to the instrument.



• Preparation for Staining

Refer to P. 2-3 for details.

- Open the lid of each solution reservoir to check the solution level.
- Select a solution configuration.



• Start of Staining

Refer to P. 2-5 for details.

- Select a staining program.
- Set staining baskets in the start station.



To perform another staining run consecutively, add staining baskets.
* Applicable only in the continuous mode.
(P. 2-7)

Staining
Operations permitted during staining (P. 2-7)



• End of Staining

Refer to P. 2-12 for details.

- Remove the stained baskets from the end station.



• End of Operation

Refer to P. 2-12 for details.

- Turn off the power and stop the water supply.
- Close the lid of each solution reservoir.

Operating Procedure

Preparation for Stain Process

Refer to P. 2-12 for details.

This section explains the preparation to be performed before the start of stain process.


Opening the Water Supply

1. Open the tap water faucet so that water can be supplied.
2. Hold the center of the cover and raise while pulling it toward you to open the cover and remove the lid of each solution reservoir.

Caution! Be sure to remove all lids. If any lid remains closed, tissues may be damaged when the baskets are transferred to the applicable station.

Logging On to the Instrument



1. The user must log on to the instrument to start instrument operation. Only one user can log on to the instrument at a time. When the power switch located at the bottom right of the instrument is turned on, the display shows a message indicating that the instrument is loading data. After a while, a confirmation screen on instrument use appears. After confirming the items, touch the **OK** button to return to the Stain Process Monitor screen.

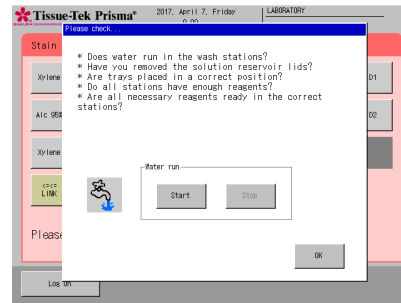

Note Touch the **Start** button under "Confirm Water Supply," and you can perform a water supply test. Touch the **Stop** button to stop the water supply.

2. When the Stain Process Monitor screen appears, touch the **Log On** key at the bottom left. When the Enter Password sub-window appears, enter a pre-assigned password using the numeric keypad (number keys 0 to 9) shown on the display. If you have made a mistake, touching the **Backspace** key deletes the last character entered. Alternatively, you can touch the **Cancel** key to cancel the password entry and return to the Stain Process Monitor screen. Refer to P. 1-39 for passwords.
3. After the password has been entered, touch the **Enter** key to complete the logon.

Note **Logging off**
To log off from the instrument, touch the **Log Off** key at the bottom left of the screen. You can log off from the instrument even after staining has started. Once the user logs off, the instrument only accepts the operation inputs to check the staining status, which prevents inadvertent operations by different users. To perform other operations, the user must log on to the instrument again.

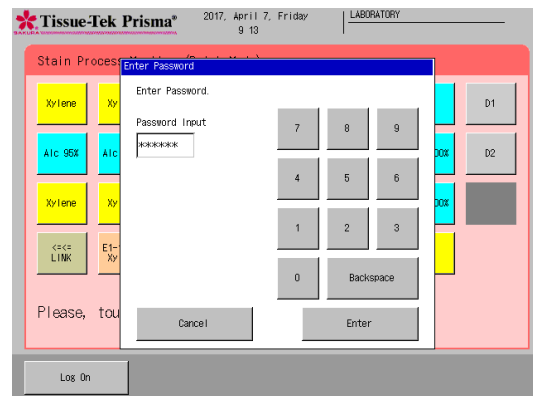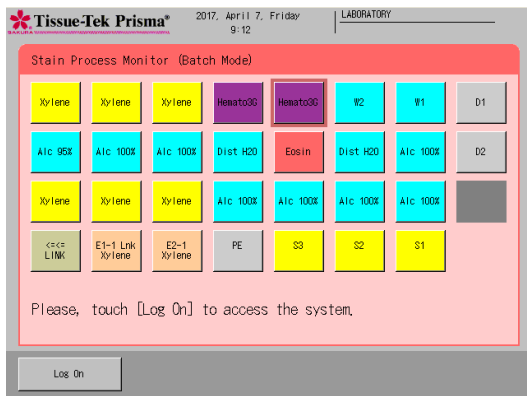

Operating Procedure

Selecting and Saving a Solution Configuration and Setting Solutions

The instrument can store multiple solution configurations and staining programs. Accordingly, you must select a solution configuration appropriate for the staining program you want to run and display the selected configuration on the monitor screen before starting the program. This section explains how to select and save a solution configuration that will be displayed on the monitor screen.

1. Touch the **Menu** key shown at the bottom right of the Stain Process Monitor screen.
2. Select Stain Process Menu from the Stain Process Menu, Utility Menu or Edit Menu, and then touch the **Solution Configuration Selection** key.
3. Select and touch a desired solution configuration name under Solution Configuration List, and then touch the **Select** key. For creating a new station configuration or editing an existing station configuration, refer to "Setting a Solution Configuration" (P. 2-15).





Caution!

"Selecting a Solution Configuration" If any of the following conditions applies to a solution station for which a solution management option has been set, the solution management information will not be transferred.

- The station position where solution management is performed changes when the solution configuration changes.
 - The solution at the station position where solution management is performed changes to a different solution when the solution configuration changes.
 - The management method for the station where solution management is performed changes when the solution configuration changes.
- If the solution used, station position (station number) and management method do not change after the solution configuration is changed, the solution management information will be transferred.

Note

Touch the Check Program key to check the staining programs that can be operated with the solution configuration you have selected. To check a given solution configuration, select the applicable solution configuration and touch the View key to display the Solution Configuration View screen showing the applicable station arrangement. You can also check the reservoir arrangement of the next configuration by touching the Change View key at the bottom right. If the current solution configuration is appropriate, touch the Select key. To check other solution configuration, touch the Exit key and select a desired solution configuration again.

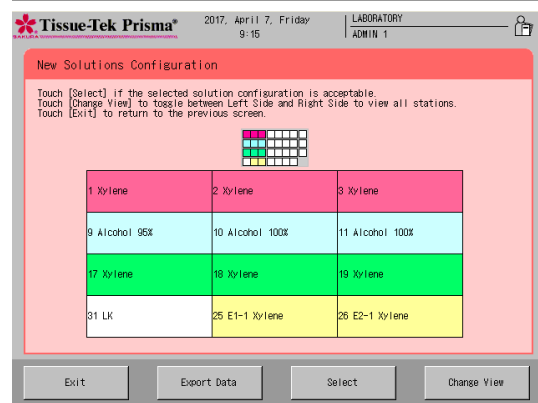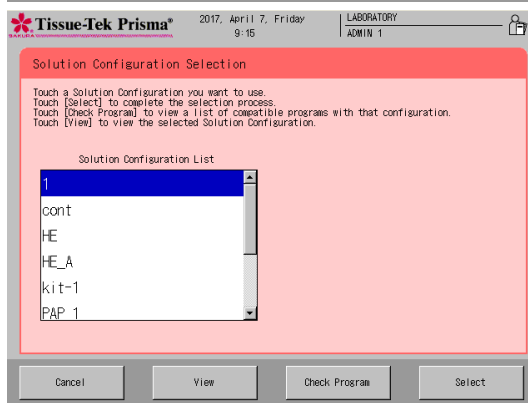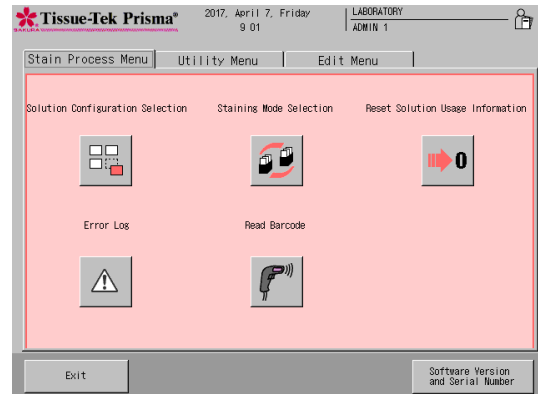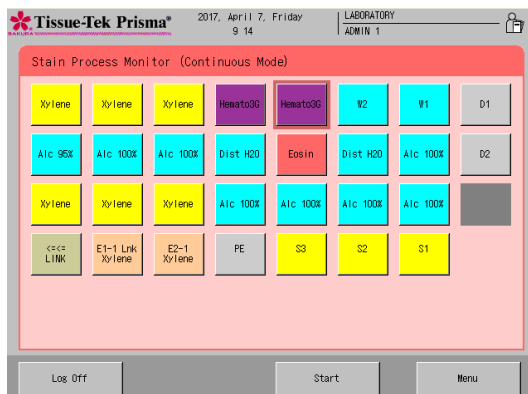

Operating Procedure

4. Touching the **Select** key displays the New Solution Configuration screen. If the new solution configuration you have selected requires changing the arm holder from the 3-position type to 1-position type, or from the 1-position type to 3-position type, the Change Arm Holder window appears with the message asking if the arm holder has been changed. Change the arm holder according to the message, and then touch **Yes**. Refer to **P. 1-14** for details on how to change the arm holder.
5. Open the cover and set solutions according to the screen display. First, set solution reservoirs by paying attention to the different reservoir types; i.e., standard solution reservoir, small solution reservoir and special stain solution reservoir.
6. Next, fill solutions into the reservoirs. Solutions should be filled into the solution reservoirs beforehand on a flat, stable surface, or directly into the reservoirs already set in the instrument. If solutions are filled outside the instrument, transport the filled reservoirs into the instrument using special solution trays to facilitate the operation.
7. After all solutions have been set, close the cover and touch the **Exit** key to save the selected solution configuration.

To perform other operation, you can return to the Stain Process Menu screen by touching the **Cancel** key.


Setting Slides into Baskets

This section explains how to set slides into baskets.

1. Prepare slides to be stained. Check the solution configuration and prepare baskets (20-slide or 10-slide baskets) together with appropriate basket adapters.
2. Attach a basket adapter to each basket by aligning the "UP SIDE" mark on top of the adapter with the "UP SIDE" mark on one side of the basket. After confirming that both "UP SIDE" marks are facing the same direction, tilt the basket adapter along the groove.
3. Next, store the slide into the basket one by one. Set the slides in the correct orientation so that their tissue side aligns with the "UP SIDE" marks. When all slides have been set, return the tilted basket adapter to the original position.

Note Do not use a deteriorated, worn, damaged or deformed basket/basket adapter, as it may cause problems. Replace any such basket/basket adapter with a new one. Store the slides evenly in the basket.

Caution! If the slides are stored unevenly, basket movements are affected during the staining operation. Particularly when there are fewer slides, be careful not to let the basket tilt when storing the slides.

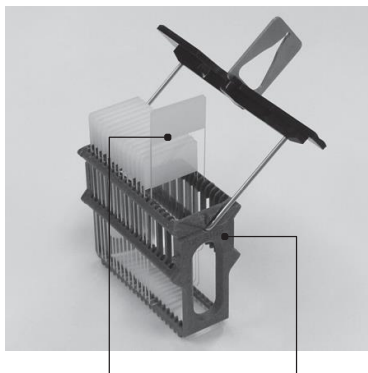

Tissue side "UP SIDE" mark




"UP SIDE" mark


Operating Procedure

Start of Stain Process

This section explains a normal procedure to start stain process. Refer to **P. 2-13** for the procedure to start stain process in the "batch mode" by specifying the start step and end step.

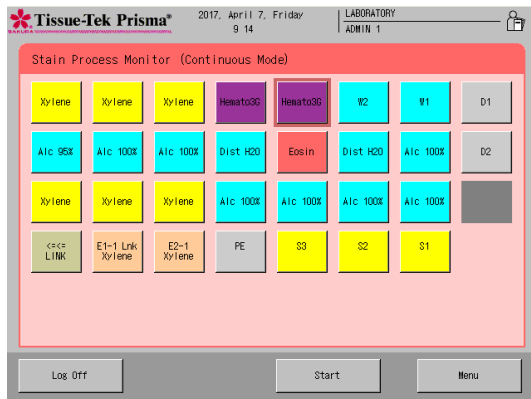


1. Touch the **Start** key at the bottom right of the Stain Process Monitor screen. If the Stain Process Monitor screen is not currently displayed, complete or abort the current operation and touch the **Exit** key or **Reset** key at the bottom right of the screen to return to the Stain Process Monitor screen.
2. Touch the **Start** key and select a desired staining program under Program List. To check the details of the selected program, touch **View Program** key at the bottom right.

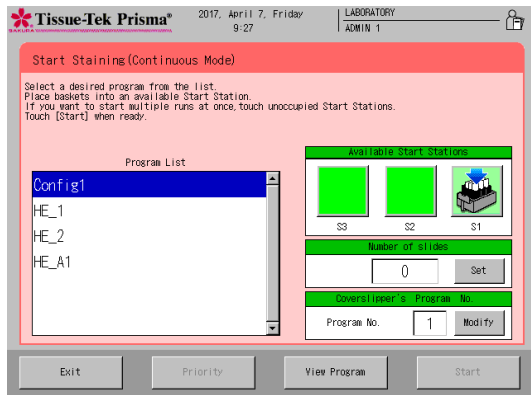


3. Next, select the place to set a basket in, from among the baskets shown under Available Start Stations on the right side of the screen. If all start stations are empty, "S1" is automatically selected. To change the start station, select the new station, and then touch the original station to cancel the selection. You can also select and set multiple start stations simultaneously for the same staining program. The number of start stations as well as the number and type of baskets that can be set in each start station are fixed depending on the selected "solution configuration." Check the applicable basket type from the basket icon shown on the screen.

Note If three start stations have been set, "S1," "S2" and "S3" are shown. Touching a desired icon on the screen adds the basket mark on the applicable station, thus setting it as the start station in which baskets are set.




For standard solution station
Up to three 20-slide baskets can be set.

Note Use the 3-position arm holder.



For small solution station
Only one 20-slide basket can be set.

Note Use the 1-position arm holder and attach a 20-slide basket load/unload adapter to the start station. The arm will not pick up baskets set in positions other than the center of the start station.



For special stain solution station
Only one 10-slide basket can be set.

Note Use the 1-position arm holder and attach a 10-slide basket load/unload adapter to the start station. Stain process cannot be started if a 10-slide basket load/unload adapter is not set.

4. If the program covers stations for which "Management by slides" has been set, the field in which to enter the number of slides is displayed. If this field is displayed, enter a desired number of slides. Touching the **Set** key displays the virtual numeric keypad for entering values. The range of values that can be entered varies depending on the program. If the program covers only standard solution stations, the range is 1 to 60. If the program covers only small solution stations, the range is 1 to 20. If the program covers special stain solution stations, the range is 1 to 10.

Note If the **Start** button is touched without entering a value, a warning window appears. If the program does not cover any station for which "Management by slides" has been set, the field in which to enter the number of slides is not displayed. If "Management by slides" is selected, processing can be started at only one station at a time.

Operating Procedure

5. Open the door at the bottom of the cover to set a basket into the start station. Push the top of the center of the door toward the back once opens the door toward you. When the door opens, draw out the reservoir in the start station and set by paying attention to the orientation of the basket in which slides have been set. Set the basket so that the "UP SIDE" mark on top of the basket adapter faces right. Align the grooves in the top surface of the start station with the grooves provided at both ends of the top surface of the basket adapter. After setting the basket, push the start station reservoir into the instrument, close the door, and then push the door until a click sound is heard.

Caution!

When setting a small solution reservoir or special stain solution reservoir, attach an applicable adapter to the start station. If not, stain process cannot be performed correctly. Also confirm that the arm holder has only one tab. If a wrong arm holder is set, stain process cannot be performed correctly.

6. Touching the **Start** key at the bottom right of the screen displays a confirmation message. Select **Start** to start staining.
7. Once staining starts, the Stain Process Monitor screen appears where you can check the current processing. Refer to **P. 2-7** for the operations permitted during stain process.

Caution!

The starting procedure explained above is only one example. If the instrument has been set to start staining upon closing of the door, you need not touch the Start key because closing the door automatically starts stain process. Refer to **P. 1-33** for the settings relating to the starting procedure.

Operating Procedure

Operations Permitted during Stain Process


The following operations are permitted during stain process. Addition of baskets and setting of priority start are enabled only in the continuous mode.

- Add a basket
- Priority start
- Check the end station status
- Check the staining status
- Menu operations permitted during stain process
- Pause and abort stain process
- Remove a basket


Adding a Basket

This section explains the procedure to add a basket during stain process. This procedure is basically the same as the normal staining start operation, but a basket can be added only in the continuous mode. Also note that no basket can be added in the following conditions:


- The instrument is performing stain process in the batch mode.
- 11 staining runs are being processed.
- The start station is full.
- An error (including a station full error) is present.

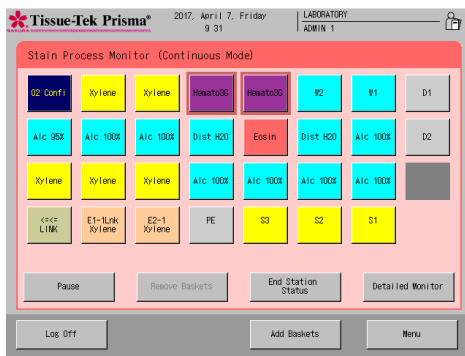

1. Touch the **Add Baskets** key at the bottom right of the Stain Process Monitor screen.
2. When the Stain Start Monitor screen appears, select a desired staining program under Program List. To check the details of the selected program, touch **View Program** key at the bottom right.
3. Next, select the location of the start station to set a basket in, from among the baskets shown under Available Start stations on the right side of the screen. Select **S1, S2 or S3**. Touching a desired station selects it as the start station, if the station is available. To change the start station, select the new station, and then touch the original station to cancel the selection. You can also select and set multiple start stations simultaneously for the same staining program. The number of start stations as well as the number and type of baskets that can be set in each station are fixed depending on the selected "solution configuration." Check the applicable basket type from the basket icon shown on the screen.
4. If the program covers stations for which "Management by slides" has been set, the field in which to enter the number of slides is displayed. If this field is displayed, enter a desired number of slides.

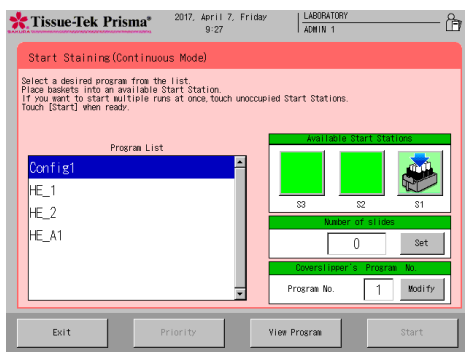

Start screen with the Glas g2 connected

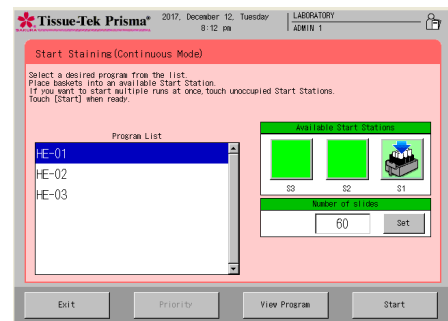

Start screen with the Film connected

Note If the **Start** button is touched without entering a value, a warning window appears. If the program does not cover any station for which "Management by slides" has been set, the field in which to enter the number of slides is not displayed. When using a coverslipper, enter an applicable program number in the coverslipper program number entry field.

Operating Procedure

5. Open the door at the bottom of the cover to set a basket into the start station. Confirm that the arm is not on a solution station in the frontmost row. If the arm is in the frontmost row when the door is opened, an error will generate and the operation will pause, which may extend the staining time. Open the door, draw out the start station reservoir, and then carefully set a basket in which slides have been set, by paying attention to the orientation of the basket. Set the basket so that the "UP SIDE" mark on top surface of the basket adapter faces right. After setting the basket, push the start station reservoir into the instrument, close the door, and then push the door until a click sound is heard.



If you are using a 20-slide basket in a small solution station, attach a 20-slide basket load/unload adapter to the start station. If not, staining process may not be performed correctly. If you are using a special stain solution station, attach a 10-slide basket load/unload adapter to the start station. If not, staining process will not be started.

6. Touching the **Start** key at the bottom right of the screen displays a confirmation message. Select Start to start staining. If the instrument has been set to start stain process upon closing of the door, you need not touch the **Start** key because closing the door automatically starts stain process. Refer to **P. 2-19** for the applicable settings.
7. Once staining starts, the Stain Process Monitor screen appears.

Setting Priority Start

Priority start is a function that allows tissues set in the start station later to be transferred and stained before other tissues that are ready and standing by in the start station. The end time cannot be forwarded. This function does not work if no basket is set in the start station.

1. Confirm that the target basket is set in the start station. Then, touch the **Start** key and select a desired staining program from Program List in the Stain Process Monitor screen, just like when you are adding a basket.
2. Touching the **Priority** key activates the priority start function and displays the Stain Process Monitor screen.

Checking the End Station Status

You can check the statuses of all end stations currently defined, by touching the **End Station Status** key in the Stain Process Monitor screen during stain process. The end stations are numbered E1-1, E1-2, and so on, from the left, when the instrument is linked to a coverslipper, or E2-1, E2-2, and so on, from the left, when the instrument is not linked to a coverslipper.



The end stations indicated by an asterisk (*) have no basket inside.

Touch the **Exit** key to return to the Stain Process Monitor screen.

Operating Procedure

Checking the Staining Status

You can check the status of the current staining run by touching the **Detailed Monitor** key in the Stain Process Monitor screen during staining process. The scheduled end time can also be checked in the Detailed Monitor. To return to the previous screen, touch the **Overview Monitor** key. You can log off from either screen, and both screens function as the Stain Process Monitor screen.

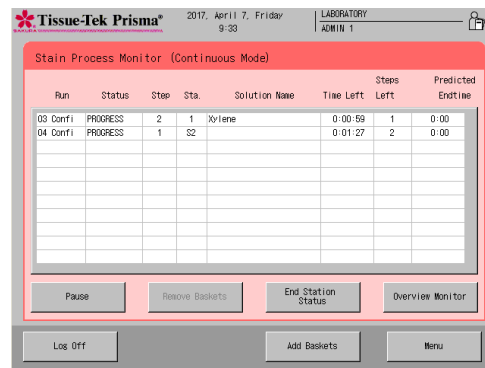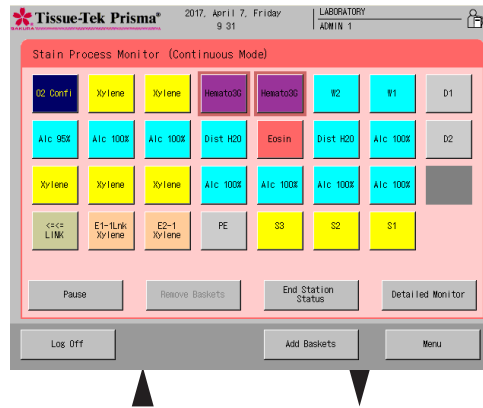


Note The staining run numbers and abbreviated staining program names shown in the "No." field are also shown in the Overview Monitor, so the current step can be visually identified at a glance. Also, you can touch the button corresponding to a desired staining run number (No.) in Overview Monitor to check the details of the applicable staining program. Touch the Exit key to return to the Overview Monitor.

Operating Procedure

Menu Operations Permitted during Staining Process

You can perform certain menu operations during stain process by touching the **Menu** key in the Stain Process Monitor screen.

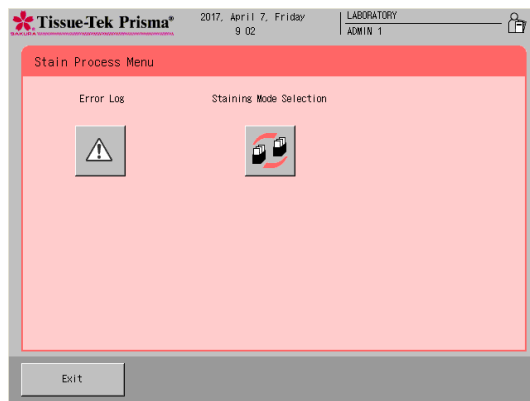

- **Error display**

Up to 99 errors that have occurred since the start of staining process are displayed, from the latest error shown on top.

You can touch the ◀ ▶ keys on the side of the screen to scroll the list to show the hidden columns. Touching **Error Description** displays a screen explaining error numbers and corresponding error descriptions. You can also touch **Clear** to clear the saved error data.

- **Setting a staining mode**

Once set, the staining mode (continuous/batch) cannot be changed during stain process. Other settings can be changed during stain process. (P.1-33)

Operating Procedure


Pausing and Aborting Stain Process

This section explains the procedures to pause and abort stain process. The abort operation can be performed only after the pause operation.

- **Pause operation**

To pause staining process, touch the **Pause** key on the left of the Stain Process Monitor screen. Touching the **Pause** key displays the pausing window. The arm will continue to move until a position where it can stop safely, after which all staining runs will stop.


To resume stain touch the **Resume** key. Note that the pause operation may make it no longer possible to complete the process by the scheduled end time. In this case, the scheduled end time must be extended at the time of resumption by up to the period during which stain process has been paused.

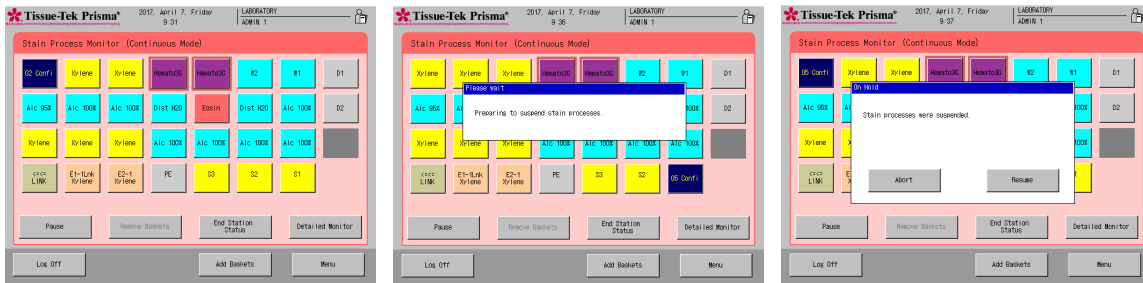

- **Abort operation**

To cancel all or part of the current staining runs, touch the **Pause** key in the Stain Process Monitor screen and then touch the **Abort** key. When the abort window appears, touch the **All Runs** key to cancel all staining runs. To cancel certain staining runs, touch the applicable staining run numbers to add a check mark to the "Selected" field of each staining run.

When the **Abort** key is touched, a confirmation screen appears and asks if you want to abort the selected staining runs. Selecting **Yes** displays the abort completion window. Open the cover and remove the aborted baskets. Once the baskets have been removed and the cover closed, the **Exit** key is displayed. Touch this key to complete the abort operation.

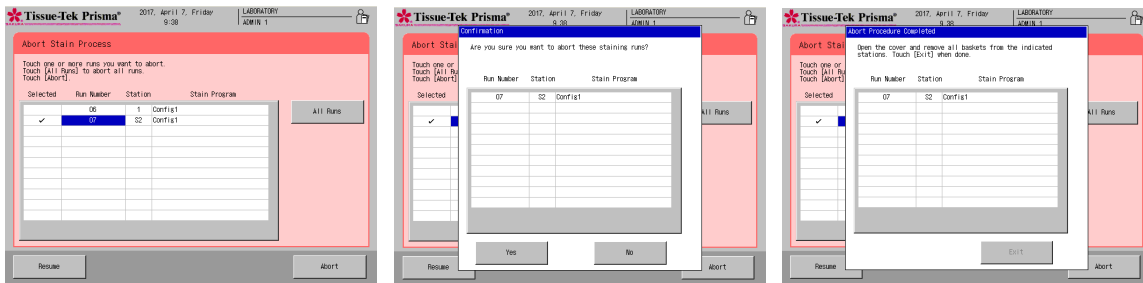

Operating Procedure

Removing a Basket during Stain Process

The following operation is performed when removing a basket from the end station or PE station during stain process or after completion of stain process. It cannot be performed if the end station or PE station contains no basket.

1. Confirm that the arm is not on a solution station in the frontmost row, and then open the door and remove the basket.
2. Once the basket has been removed and the door closed, the basket removal confirmation window appears. Touch the **Yes** key in this window. If you have not yet removed the basket, touch the **No** key.



End of Stain Process

Stained baskets are stored in the specified end station or PE station. (If the instrument is linked to the Coverslipper, stained baskets are stored in the link station. If the link station is full, stained baskets are temporarily stored in the end station and wait there until they can be stored in the link station.)

Removing Stained Baskets

1. A window notifying the end of stain process appears and automatically disappears in around 15 seconds.
2. Confirm that the arm is not on a solution station in the frontmost row, and then open the door and remove the basket.
3. Once the basket has been removed and the door closed, the basket removal confirmation window appears. Touch the **Yes** key in this window. If you have not yet removed the basket, touch the **No** key.

Note After the door is opened, remove all stained baskets from the end station and PE station.

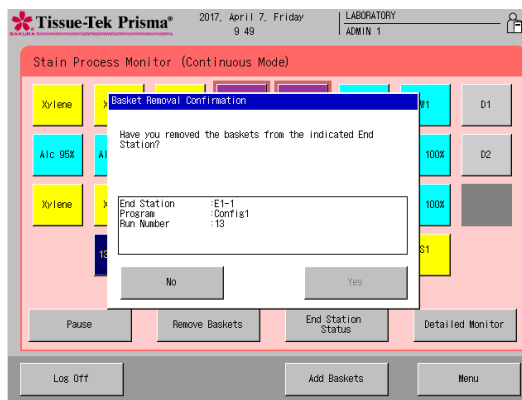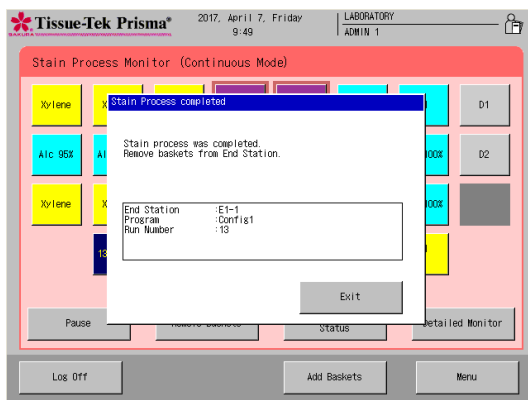

End of Operation

This section explains the procedure to complete the stain process and end the operation.

1. After all operations have completed, turn off the power switch located at the bottom right of the instrument, and then stop the water supply.
2. Close the lids on all solution reservoirs, and then close the cover.

This completes the series of operations.

Operating Procedure



How to Start Stain Process by Specifying the Start and End Steps

You can start stain process by specifying the start and end steps only in the batch mode. If a desired step is specified as the start step, the staining program will start from the specified step. If a desired step is specified as the end step, staining process will end at the specified step and the baskets will be transferred to the end station specified in the applicable staining program.

Note The start and end steps only represent temporary changes in the start and end stations. The settings will become ineffective once the series of staining runs are completed or aborted. The start and end steps cannot be set on the stations shown in the table below.

If the start step is set on a drying station, the drying station heater is turned on at the start of staining. If the start step is set on a heating station, the baskets are held in the start station until the heating station reaches the specified temperature.

Stations that cannot be set as the start step	Stations that cannot be set as the end step
End stations, PE station	Start stations, End stations, PE station

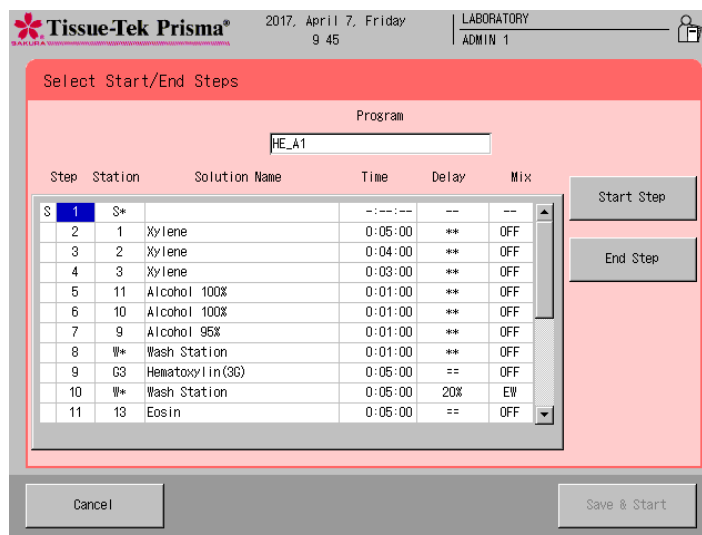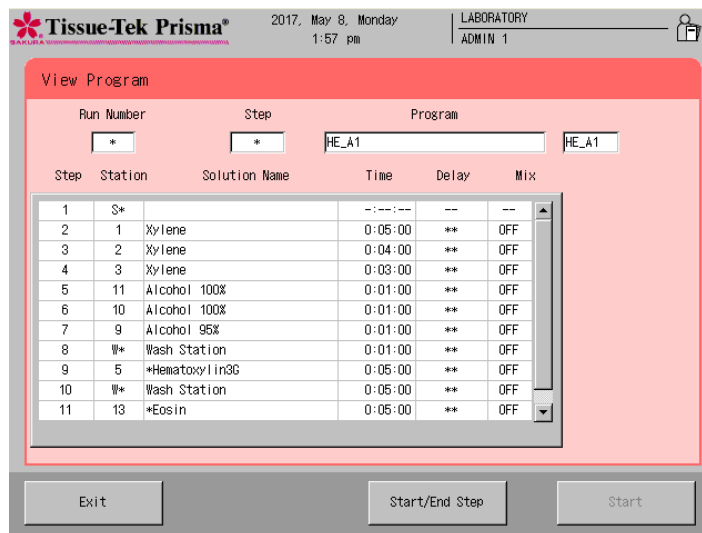

Operating Procedure

1. Touch the **Start** key at the bottom right of the Stain Process Monitor screen. If the Stain Process Monitor screen is not currently displayed, complete or abort the current operation and touch the **Exit** key or **Reset** key at the bottom right of the screen to return to the Stain Process Monitor screen.
2. Touch the **Start** key and select a desired staining program under Program List.
3. Next, set a basket and select a start station under Available Start stations. Refer to **P. 2-4** for details on how to set a basket and select a start station. Once the basket has been set and start station selected, touch the **View Program** key at the bottom right.
4. The View Program screen appears and the details of the currently selected staining program are displayed. Touch the **Start/End Step** key at the bottom right.
5. The Select Start/End Steps screen appears.
6. To set a start step, select a step number you want to specify as the start step, and then touch the **Start Step** key to the right. When the key is touched, "S" appears in front of the step number to indicate that it has been set as the start step.
7. To set an end step, select a step number you want to specify as the end step, and then touch the **End Step** key to the right. When the key is touched, "E" appears in front of the step number to indicate that it has been set as the end step.
8. Touching the **Save & Start** key starts stain process. Touching the **Cancel** key cancels the start/end step settings and returns to the View Program screen.

Setting a Solution Configuration



From "Solution Configuration," you can set a configuration of necessary solution stations and other items that are needed to run the staining program smoothly.

Creating a New Solution Configuration

This section explains the procedure to create a new solution configuration. Up to 50 solution configurations can be saved.

Saving a Configuration Name


1. Touch **Menu** at the bottom right of the Stain Process Monitor screen, and then select **Edit Solution Configuration** from the Edit Menu.
2. Touch **Add Configuration**, and then enter a solution configuration name (using of up to 20 characters) from the simple keyboard shown in the Create Solution Configuration Name screen.
After the solution configuration name has been entered, touch the **Save** key to save the name.
3. Touching the **Save** key displays the Solution Configuration Setup window (**P. 2-16**).

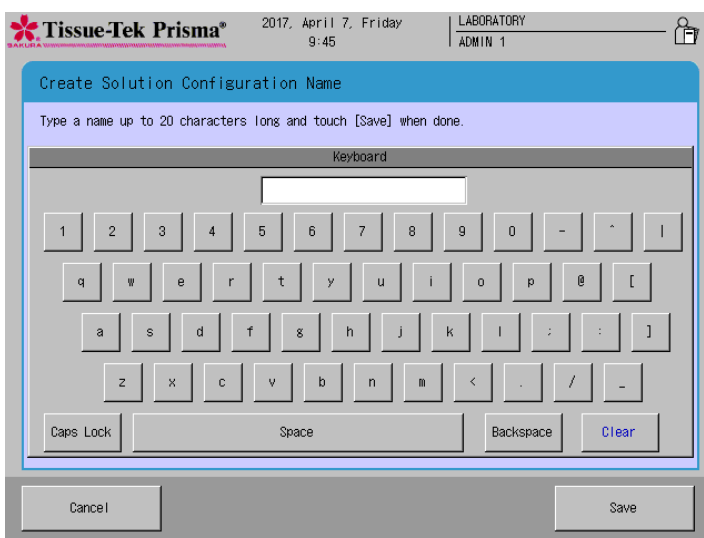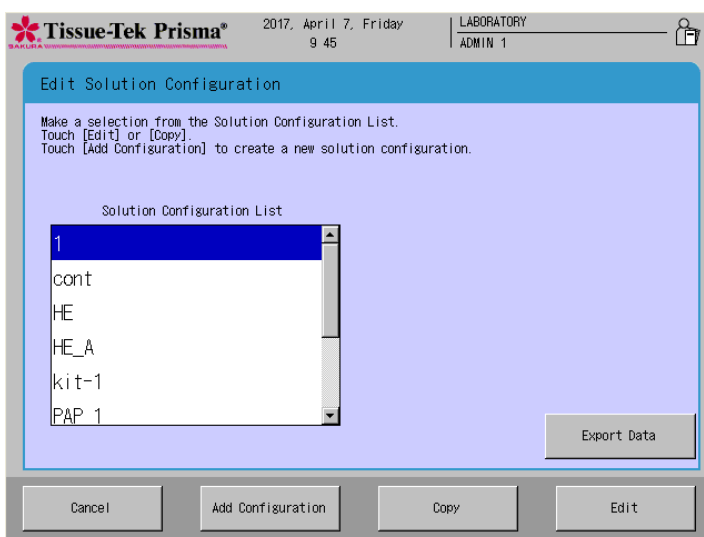

Setting a Solution Configuration

Setting Up a Solution Configuration

Setting a solution configuration name displays the Solution Configuration Setup window. In this screen, the entire solution configuration can be set. The setting of each item is explained below.

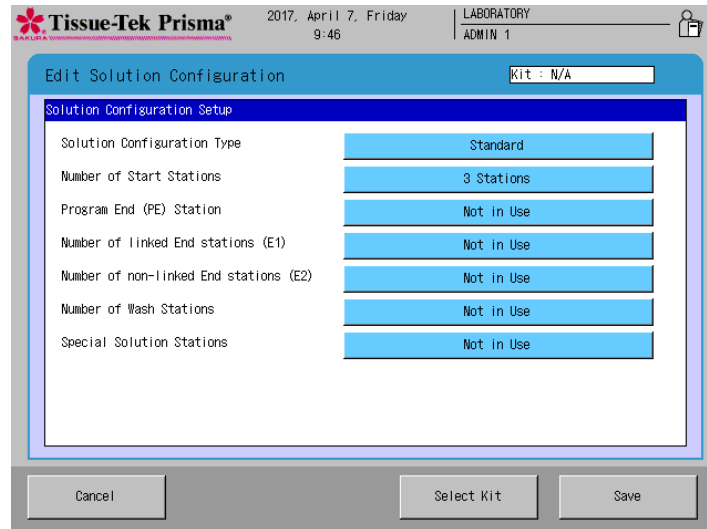

- **Solution Configuration Type**

You can select whether to use standard solution reservoirs (Standard) or small solution reservoirs (Small). Touching this function toggles the selection between Standard and Small.

- **Number of Start Stations**

Determine the number of start stations. Up to three start stations can be set. Touching this function toggles in the sequence of #1 Station, #2 Stations and #3 Stations.

- **Program End (PE) Station**

Select whether or not to use a PE station. If a PE station is used, the PE station is set on the rightmost end station. Touching this function toggles the selection between **In Use** and **Not In Use**.

- **Setting Linked End Stations (E1)**

If the instrument is linked to a coverslipper, the processing moves to the linked end stations when the staining ends. Up to five such stations can be set. Touching this function toggles the selection in the sequence of **Not In Use**, **1 Station**, **2 Stations**, **3 Stations**, **4 Stations** and **5 Stations**.

- **Setting Unlinked End Stations (E2)**

Determine the number of end stations that are not linked to the coverslipper. Up to five such stations can be set. Touching this function toggles the selection in the sequence of **Not In Use**, **1 Station**, **2 Stations**, **3 Stations**, **4 Stations** and **5 Stations**.

Note Set no more than six stations including the start station, PE station, linked end stations (E1) and unlinked end stations (E2).

Setting a Solution Configuration

- **Number of Wash Stations**

Determine how many wash stations to be used or no wash station will be used. If used, up to four wash stations can be set. Touching this button toggles in the sequence of **Not In Use**, **1 Station**, **2 Stations**, **3 Stations** and **4 Stations**.

- **Special Solution Stations**

Determine whether or not to use special solution stations. Touching this function toggles the selection between **In Use** and **Not In Use**.

- **Heating Stations**

Determine whether or not to use heating stations. If **Not In Use (No Heating)** is selected, the heating stations do not perform heating control and are treated as normal special stain solution stations. Touching this function toggles the selection in the sequence of **In Use**, **Not In Use** and **Not In Use (No Heating)**.

Touching the **Save** key displays the next setup screen, or Edit Solution Configuration screen, where you can set each solution station.

Setting an Edit Solution Configuration

Setting the items in the Solution Configuration Setup screen and then touching the **Save** key displays the Edit Solution Configuration screen. In this screen, you can set detailed settings for individual solution stations. The available settings are explained below. After the necessary items have been set, touch the **Save** key to save the completed configuration.

- **Solution Configuration Layout**

You can touch each station to set a solution name for the specified solution station as well as management method of the solution. These items can be set only for colored solution stations. Those filled with gray do not permit these settings.

- **List**

You can display the solution configuration in a list format.

- **Properties**



Touching **Properties** in the Edit Solution Configuration screen opens another screen where you can set display colors for the selected solution configuration as well as items relating to staining control.

...**Setting Station Colors (P.2-19)**



Change the display color of each station group.

...**How to Set Staining Control Data (P.2-20)**

You can set detailed controls such as "Mix," "Enhanced Wash," "Basket Lifting Speed," "Water Wash Cycle Time" and "Temperature Settings."

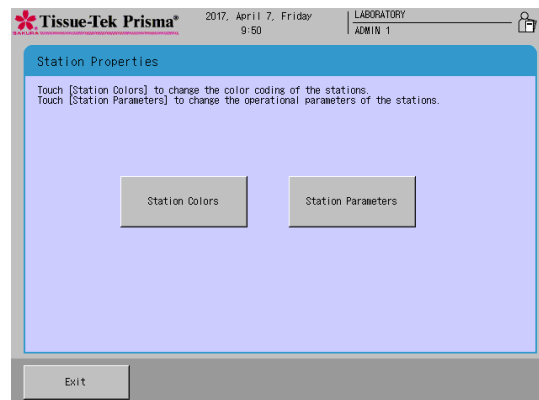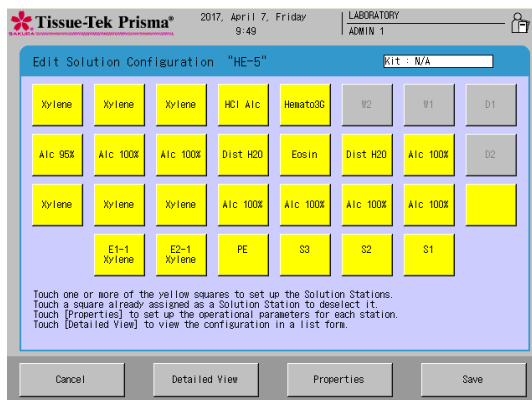

Setting a Solution Configuration

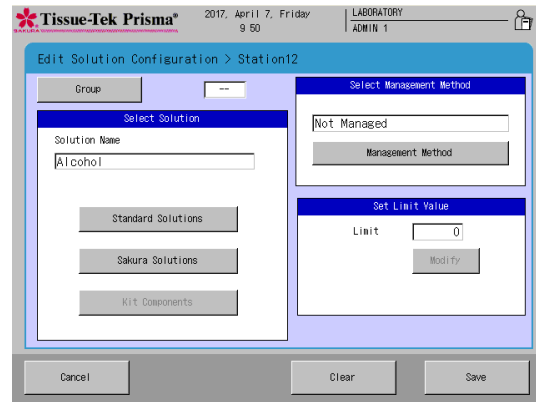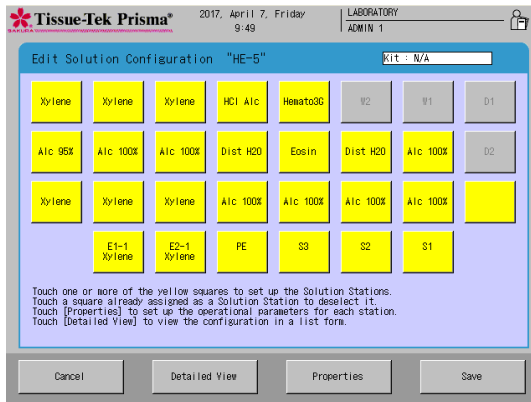

- **How to Set Solution Name and Management Method**

You can touch each station to set a solution name for the specified solution station as well as management method of the solution. These items can be set only for colored solution stations. Those filled with gray do not permit these settings.

1. In the Edit Solution Configuration screen, touch the station you want to set a solution for. Select a desired solution name from "Standard Solutions," "Sakura Solutions * " and "Kit Solutions * " in the solution setting field.

* Check the local Sakura Finetek website for availability of the Sakura staining solutions and staining kits.

Note Selecting "Standard Solutions" displays solution names of some standard solutions and solution names registered by user. Refer to P.2-23 for how to create or edit solution names.
 Selecting "Sakura solutions" displays solution names of Sakura staining solutions managed by bar code.
 Selecting "Kit solutions" displays the solutions that constitute the Sakura staining kits managed by bar code.

2. After selecting the solution name, set the solution management method. Touching the **Management Method** toggles in the sequence of "Runs: Management by runs done/Days: Management by days/Slides: Management by slides/NONE: Not in Use."
3. Next, determine a specific usage limit. Touching the **Set** button in the entry field displays the virtual numeric keypad, so enter a setting value using the numeric keypad.

Note **Runs:** Manages the number of runs. You can enter a value between 0 and 99.
Days: Manages the number of days. You can enter a value between 0 and 99.
Slides: Manages the number of slides. You can enter a value between 1 and 9999.
NONE: Not in Use. The entry field is grayed out.

4. Touching the **Save** key saves the settings and returns to the Edit Solution Configuration screen.



- **How to Set Station Groups**

Select solution names and touch the **Group** button, and you can assign the stations to groups G1 to G5. The stations in a group can have a colored border.

- **Switching to List View**

You can check the current solution configuration in a list format by touching Detailed View in the Edit Solution Configuration screen.

Setting a Solution Configuration

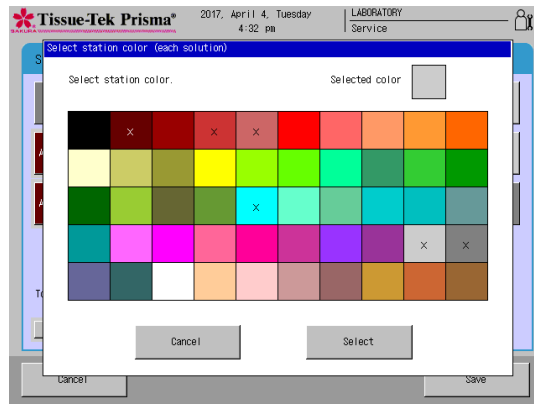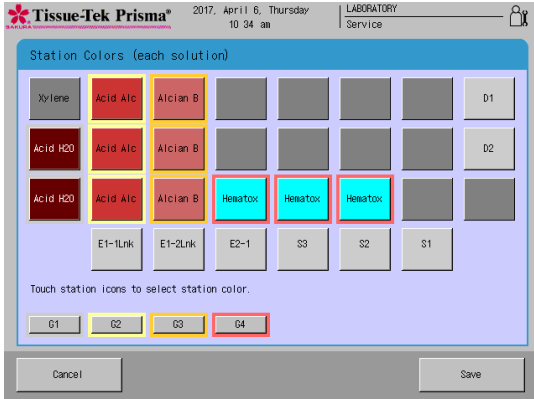

• How to Set Station and Group Colors

Touching **Properties** in the Edit Solution


Configuration screen opens another screen where you can set display colors for the selected solution configuration as well as items relating to staining control.

• Setting Station Colors

You can assign colors to the solution stations, heating stations, special solution stations, drying stations, wash stations, start stations, end stations, and PE station, except for the link stations. Fifty different colors are available. Different colors can be set for each solution name or station.

1. Touch the **Properties** button on the Edit Solution Configuration screen, and then touch the **Set Color** button on the Set Station Configuration Properties screen, to move to the Set Station Color screen.
2. Touch the button corresponding to the station you want to set a color for.
3. The Select Color window appears. Select one color, and then touch the "Select" button to apply the selection.
4. The Select Station Color screen closes and the color of the applicable station button changes to the selected color. Touch the **Save** button to save the setting.

Note If the color setting mode is different for each solution name, all stations of the same solution name have the same color. Note that this only applies to the solution stations, heating stations and special solution stations. The colors already selected are denoted by x.

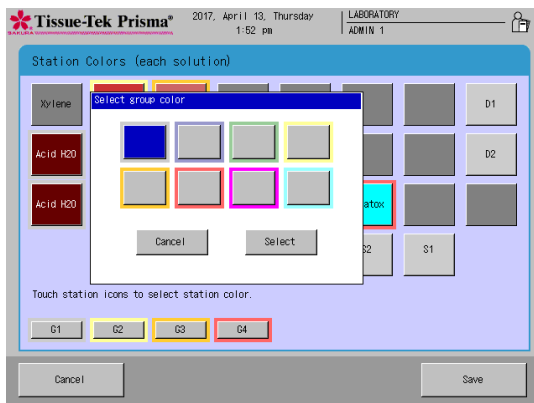

• How to Set Colors for Solution Name Groups


The color setting screen for a group of solution names has up to five buttons of G1 to G5. Eight colors are available.

1. Touch the **Properties** button on the Edit Solution Configuration screen, and then touch the **Set Color** button on the Set Solution Configuration Properties screen, to move to the Set Station Color screen.
2. Touch the button corresponding to the group whose color you want to change. Example) **G1** button
3. The Select Group Color window appears. Select one color, and then touch the "Select" button to apply the selection.
4. The selected color appears around the border of the stations buttons in the group. Touch the **Save** button to save the setting.

Setting a Solution Configuration

- **How to Set Staining Control Data**

At staining control data, you can set detailed staining controls by touching **Mix**, **Enhanced Wash**, **Basket Lifting Speed**, **Water Wash Cycle Time** and **Temperature Settings**. These items can be set separately for each solution configuration.

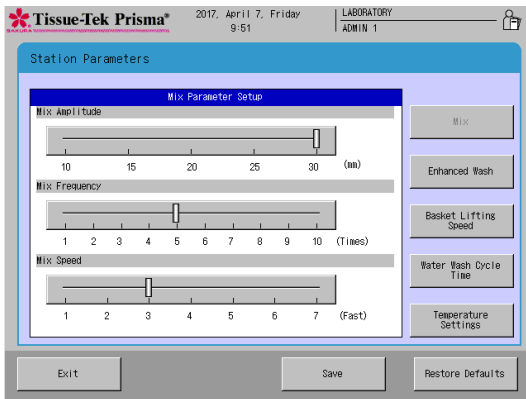

Mix Parameter Setup

- **Mix Amplitude**


You can set the amplitude of up/down basket movement in the solution. Select a desired slider and move it to right or left to change the setting. The adjustable range is "10 to 30" (mm) at 5-mm intervals. The default is "30" mm.

- **Mix Frequency**

You can set the mix frequency applicable to each mixing operation. Select a desired slider and move it to right or left to change the setting. The adjustable range is "1 to 10" (times). The default is "5."

- **Mix Speed**

You can set the speed of up/down basket movement during mixing. Select a desired slider and move it to right or left to change the setting. The adjustable range is "1 to 7," with "7" representing the maximum speed. The default is "3."

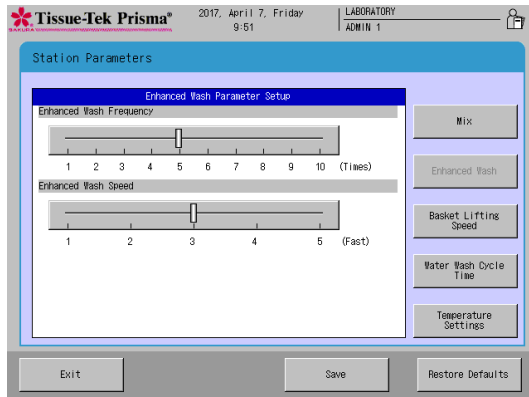

Enhanced Wash Parameter Setup


- **Enhanced Wash Frequency**

You can set the number of times the up/down basket movement out of the solution and back into the solution is repeated. Select a desired slider and move it to right or left to change the setting. The adjustable range is "1 to 10" (times). The default is "5."

- **Enhanced Wash Speed**

You can set the speed of up/down basket movement during enhanced wash. Select a desired slider and move it to right or left to change the setting. The adjustable range is "1 to 5," with "5" representing the maximum speed. The default is "3."

Setting a Solution Configuration

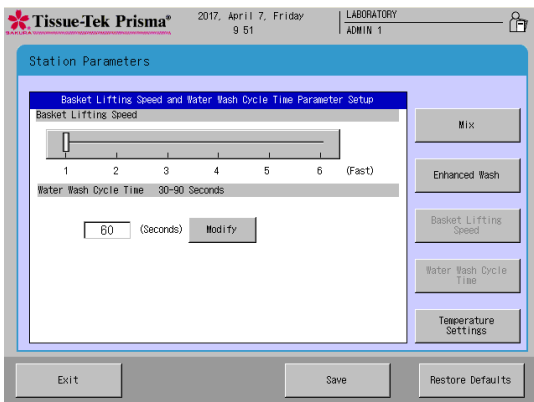

Basket Lifting Speed and Water Wash Cycle Time Parameter Setup

• Basket Lifting Speed


You can set the speed at which baskets are lifted. Select a desired slider and move it to right or left to change the setting. The adjustable range is "1 to 6," with "6" representing the maximum speed. The default is "1." Use "1" in normal conditions of use.

Caution! When the pull-up speed is increased, more solution will be brought into the next station and accelerate solution deterioration, resulting in lower staining quality. There are also risks of tissue contamination due to dropping solution.

• Water Wash Cycle Time

You can set the cycle at which the dirty water containing staining solution and other contaminants in the wash stations is changed. Set a value using the numeric keypad displayed by touching the **Modify** key.

The adjustable range is "30 to 90" (seconds). The default is "60" (seconds).

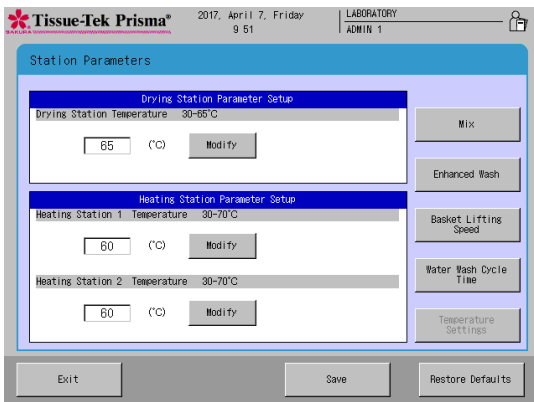

Temperature Settings

• Drying Station Temperature

You can set the temperature of drying stations. Set a value using the numeric keypad displayed by touching the **Modify** key. The adjustable range is "30 to 65" (°C). The default is "65" (°C). Note that the drying stations may not be controlled at low temperatures depending on the outside air temperature.

• Heating Station 1 Temperature

You can set the temperature of the heating station installed at the back of the instrument. Set a value using the numeric keypad displayed by touching the **Modify** key. The adjustable range is "30 to 70" (°C). The default is "60" (°C). Note that this heating station may not be controlled at low temperatures depending on the outside air temperature.

• Heating Station 2 Temperature

You can set the temperature of the heating station installed at the front of the instrument. Set a value using the numeric keypad displayed by touching the **Modify** key. The adjustable range is "30 to 70" (°C). The default is "60" (°C). Note that this heating station may not be controlled at low temperatures depending on the outside air temperature.

Setting a Solution Configuration

Editing a Solution Configuration

This section explains the procedure to edit an existing solution configuration.

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen.
2. Select **Edit Solution Configuration** in the Edit Menu.
3. Under Solution Configuration List in the Solution Configuration Selection screen, touch and select the name of the solution configuration you want to edit.
4. Next, touching the **Edit** key displays the Create Solution Configuration Name screen showing the simple keyboard. To change the solution configuration name currently saved, enter a new name from the onscreen keyboard (using of up to 20 characters). After the new solution configuration name has been entered, touch the **Save** key to save the name.
5. Touching the **Save** key displays the Solution Configuration Setup screen explained on **P. 2-16**.

Caution If a solution configuration affecting any saved staining program has been edited, the changes cannot be saved.

Caution "Editing a Solution Configuration"
If any of the following conditions applies to a solution station for which a solution management option has been set, editing the solution configuration resets the solution management information.


- The station position where solution management is performed is changed by editing the solution configuration.
- The solution at the station position where solution management is performed is changed to a different solution by editing the solution configuration.
- The management method for the station where solution management is performed is changed by editing the solution configuration.

Note that, so long as each station where solution management is performed is not edited by editing the solution configuration, and so long as the solution used, station position (station number) and management method remain unchanged, the solution management information will not be reset even when the solution configuration is edited.

Copying a Solution Configuration

This section explains the procedure to copy a solution configuration.

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen.
2. Select **Edit Solution Configuration** in the Edit Menu.
3. Under Solution Configuration List in the Solution Configuration Selection screen, touch and select the name of the solution configuration you want to copy and then touch the **Copy** key. The Copy Solution Configuration screen appears.
4. Next, select the copying destination. To create a new solution configuration to copy to, touch the **Add Configuration** key. To copy to an existing solution configuration, select the desired solution configuration from the Copy to window and then touch the **Copy** key.
5. Touching the **Add Configuration** or **Copy** key displays the Create Solution Configuration Name screen showing the simple keyboard. To change the solution configuration name, enter a new name from the onscreen keyboard (using up to 20 characters). After the new solution configuration name has been entered, touch the **Save** key to save the name.
6. Touching the **Save** key displays the Solution Configuration Setup screen explained on **P. 2-16**.

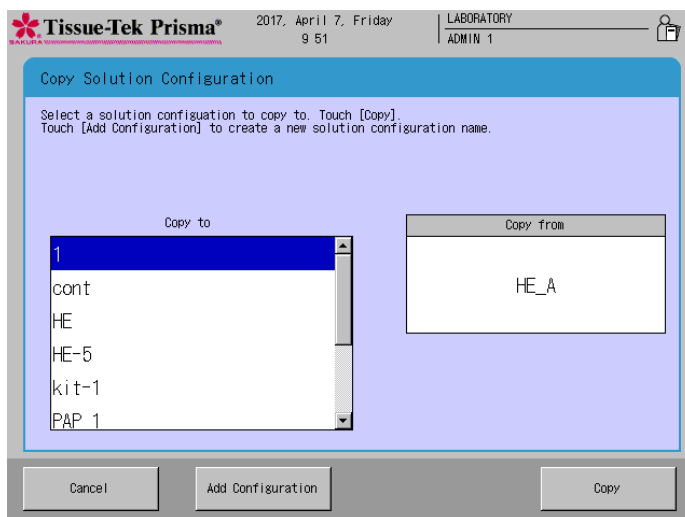



Setting a Solution Configuration

Editing a Solution Name

This section explains the procedure to set a solution name. Up to 100 solution names can be saved. Each solution name must consist of up to 20 alphanumeric characters.

Creating and Editing a Solution Name



1. Touch **Menu** at the bottom right of the Stain Process Monitor screen.
2. Select **Edit Solution Name** from the Edit Menu.
3. When the Edit Solution Name screen appears, touch the **Add Solution Name** key if you want to create a new solution name. To change an existing solution name, select the solution name you want to change under Solution List, and then touch the **Edit** key.
4. Touching the **Edit** key displays the Create Solution Name screen showing the simple keyboard, so enter a solution name (using up to 20 characters). Once the solution name has been entered, touch the **Save** key. When the screen prompts you to enter an abbreviated solution name, enter an abbreviation from the onscreen keyboard and then touch the **Save** key to save the abbreviation. (Enter the abbreviation using up to eight characters.)
5. Touching the **Save** key displays the Edit Solution Name screen, where the solution name you have just saved is shown under Solution List.

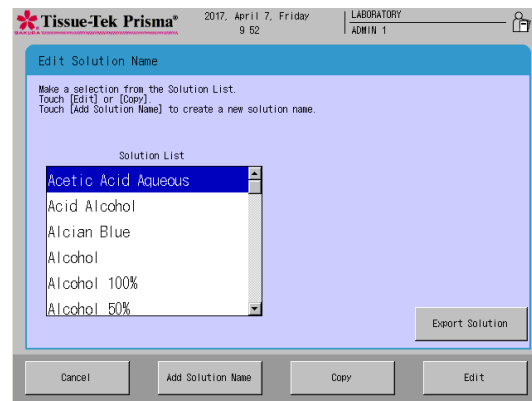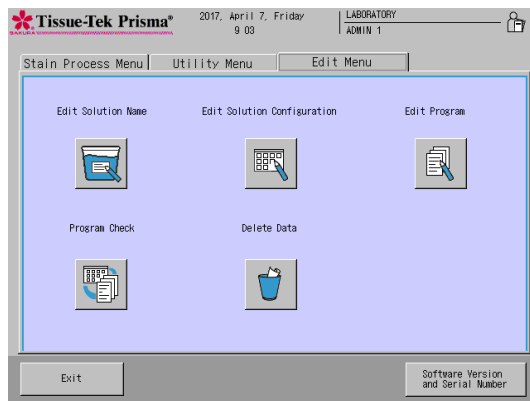

Setting a Solution Configuration

Copying a Solution Name

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen.
2. Select **Edit Solution Name** from the Edit Menu.
3. Under Solution List in the Edit Solution Name screen, touch and select the solution you want to copy and then touch the **Copy** key.
4. Touching the **Copy** key displays the Create Solution Name screen showing the simple keyboard. To change the solution name, enter a new name from the onscreen keyboard (using of up to 20 characters). Once the solution name has been entered, touch the **Save** key. When the screen prompts you to enter an abbreviated solution name, enter an abbreviation from the onscreen keyboard and then touch the **Save** key to save the abbreviation. (Enter the abbreviation using up to eight characters.)
5. Touching the **Save** key displays the Edit Solution Name screen, where the solution name you have just saved is shown under Solution Name List.

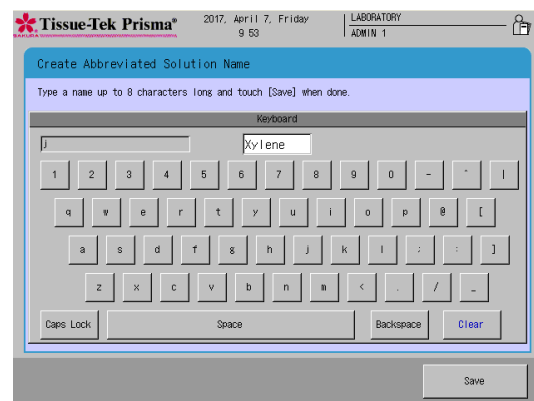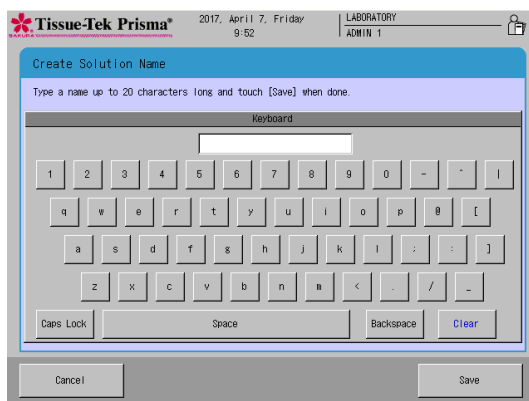

Setting a Staining Program

"Staining programs" are programs that automatically perform stain process based on preset "solution configurations." Refer to **P. 2-15** for "solution configurations."


Creating a New Staining Program

This section explains the procedure to create a new staining program from a preset "solution configuration." If there is no "solution configuration" appropriate for the staining program you are creating, refer to "Setting a Solution Configuration" on **P. 2-15**.

Up to 50 staining programs can be saved.

Saving a Staining Program Name


1. Touch **Menu** at the bottom right of the Stain Process Monitor screen.
2. Select **Edit Program** from the Edit Menu.
3. Touching **Change Solution Configuration** displays the Select Configuration window. In the list of preset solution configurations, touch the solution configuration you want to use for your new staining program and then touch the Select key.
4. Next, touch the **Add Program** key. When the Create Program Name screen showing the simple keyboard appears, enter a solution name (using up to 20 characters). Once the solution name has been entered, touch the **Save** key. When the screen prompts you to enter an abbreviated staining program name, enter an abbreviation from the onscreen keyboard and then touch the **Save** key to save the abbreviation. (Enter the abbreviation using up to five characters.)
5. Touching the **Save** key proceeds to the operation explained under "Creating a Staining Program" on **P. 2-26**.

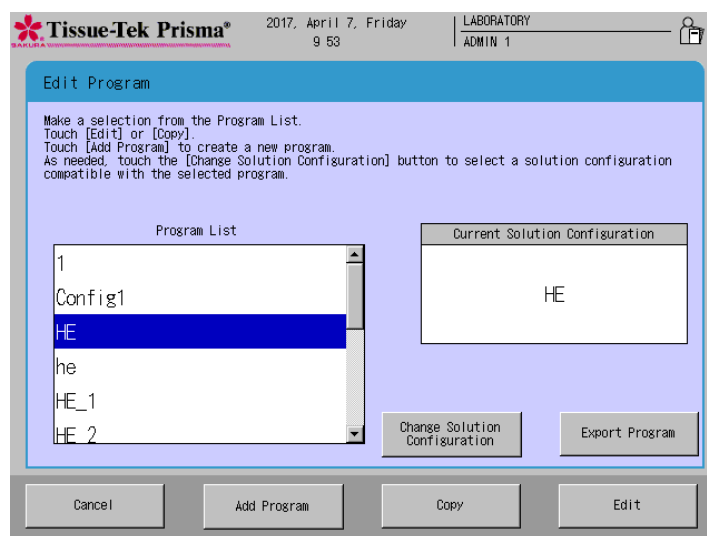

Setting a Staining Program

Creating a Staining Program

To create a staining program, touch a desired item or select a desired location using the cursor keys located at the bottom right. The procedure to create each item is explained below.

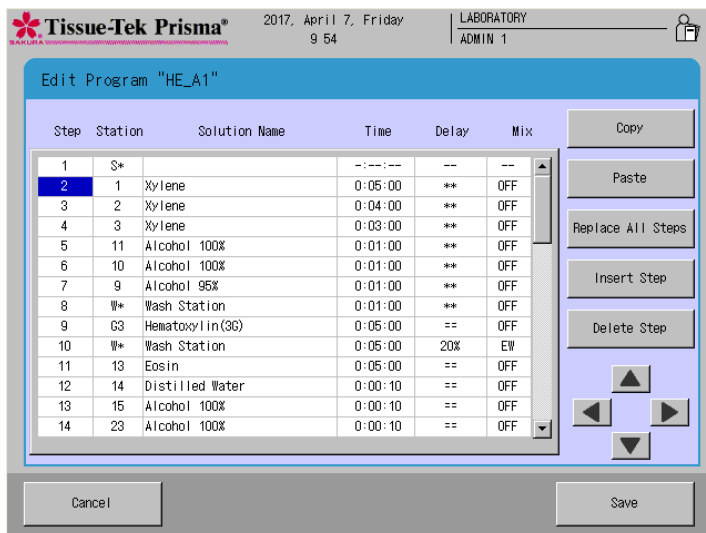

- **How to Operate the "Step" Field**

"Step" refers to the order of each staining step. You can touch and select a number(s) or blank field(s) under "Step" in the Edit Program screen and then select **Copy**, **Paste**, **Replace All Step**, **Insert Step** or **Delete Steps** to perform the applicable operation.

- **Copy**

Touching this button copies the line corresponding to the selected "Step" field. The data is not yet pasted.

- **Paste**

Touching this button pastes the data in the copied line to the line corresponding to the "Step" field currently selected. This operation inserts the source line immediately above the "Step" line currently selected by the cursor. Basically, only the "Time," "Delay" and "Mix" data are copied.

- **Replace All Steps**

Touching this button replaces the "Time," "Delay" and "Mix" data of all lines excluding the start station, end station and PE station lines, with the corresponding data in the copied line. The solution station names are not replaced.

- **Insert Step**


Touching this button inserts a new blank "Step" field directly below the line corresponding to the selected "Step" field.

- **Delete Step**

Touching this button deletes the line corresponding to the selected "Step" field.

Touching the **Save** key saves the applicable changes and closes this screen.

Setting a Staining Program

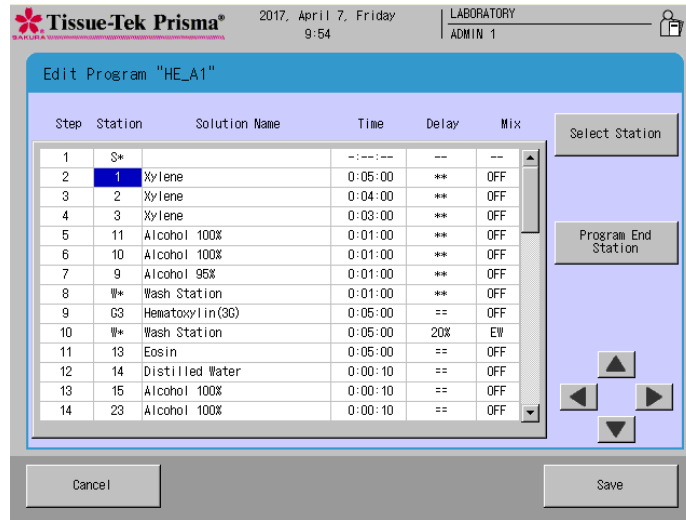

- **How to operate the "Station" field**
"Station" refers to each station number assigned to the currently selected solution configuration. You can touch and select each "Station" field in the Edit Program screen and then select **Select Station** or **Program End Station** to perform the applicable operation.

- **Select Station**


You can select the station to be assigned to the selected "Station" field. Touching the **Select Station** key displays the solution configuration layout. When a desired station is touched, the data of that station is displayed at the bottom of the screen to let you check the attribute and solution name for the station. Touching the **Modify** key reflects the selection in the "Station" field in the Edit Program screen, with the selected station number shown in the "Station" field and applicable solution name shown in the "Solution Name" field. If a non-solution station such as start station, end station, wash station or drying station is selected, the text displayed in the "Station" field conforms to the table below.

Note Set an end station or stations at the end of the staining program.

- **Program End Station**


You can insert a PE station as a standby station in the selected line. This operation adds "PE" to the "Station" field in the Edit Program screen. For your information, the PE station is automatically set to the rightmost end station.

Touching the **Save** key saves the applicable changes and closes this screen.

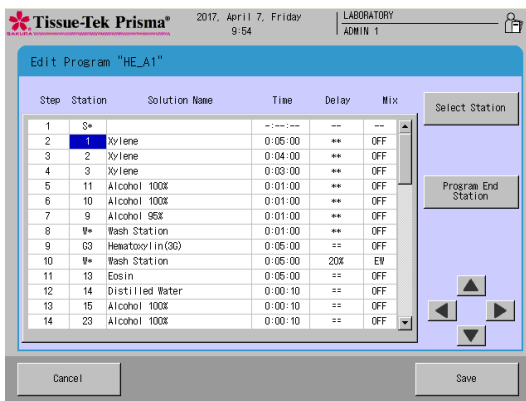

Station attribute	Text displayed in "Station" field	Text displayed in "Solution Name" field
Start stations	S *	(Solution name)
End stations (linked)	E1- *	(Solution name)
End stations (not linked)	E2- *	(Solution name)
Wash stations	W *	Wash Station
Drying stations	D *	Drying Station
PE station	PE	(Solution name)

Setting a Staining Program

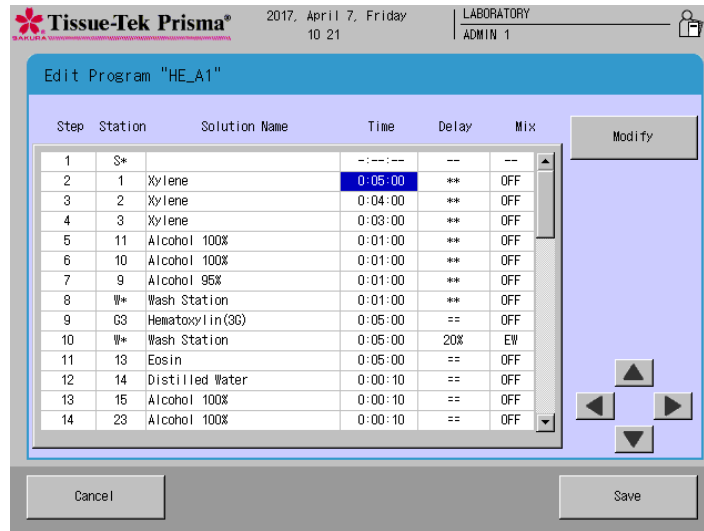

- **How to operate the "Time" field**

In the "Time" field, you can set a processing time for the currently selected solution. Set a desired time in the "Hours: Minutes: Seconds" format. The maximum time that can be entered is "99:59:59."


- **Modify**

Touch the **Modify** key to determine a processing time. Touching this key displays the numeric keypad where you can set a desired time in the "Hours: Minutes: Seconds" format. The input range of each segment is shown below:


Hours: 0 to 99
 Minutes: 0 to 59
 Seconds: 0 to 59

Touching **Clear** clears the selected segment to "0." Touching the **Enter** key applies the entered processing time. You can enter "0:00:00" in some fields. But if "0:00:00" is set in all fields, you cannot save.

Touching the **Save** key saves the applicable changes and closes this screen.

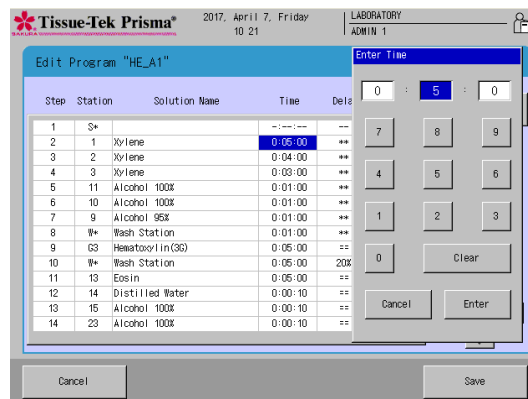

Setting a Staining Program

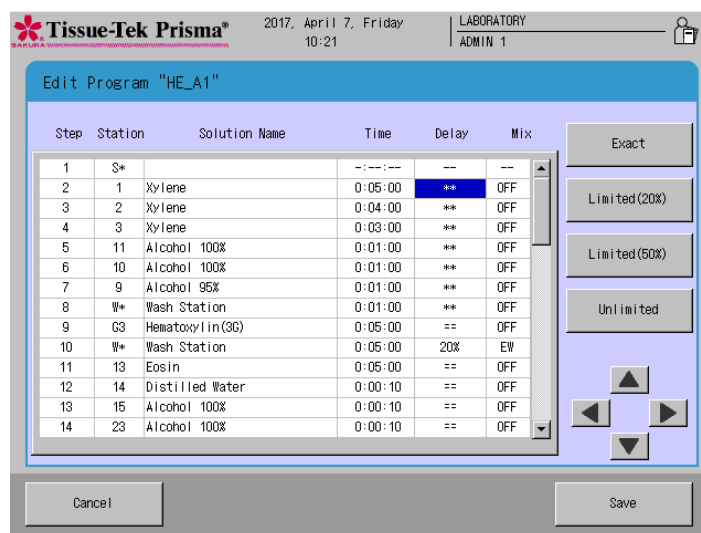

- **How to operate the "Delay" field**

"Delay" refers to an "allowable delay" to be applied to a staining run in order to run another staining run when multiple staining runs must be performed at the same time in the continuous mode while delaying other stain process.

In the "Delay" field, you can set this "allowable delay" as one of four levels.


- **Exact**
No delay is allowed and each staining run is performed exactly according to the specified time.
- **Limited (20%)**
A delay of up to 20% of the specified time is allowed.
- **Limited (50%)**
A delay of up to 50% of the specified time is allowed.
- **Unlimited**
An unlimited delay is allowed.

Each setting is displayed in the "Delay" field as follows.

Touching the **Save** key saves the applicable changes and closes this screen.

Delay setting	Text displayed in "Delay" field
Exact	= =
Limited (20%)	20%
Limited (50%)	50%
Unlimited	* *

Setting a Staining Program

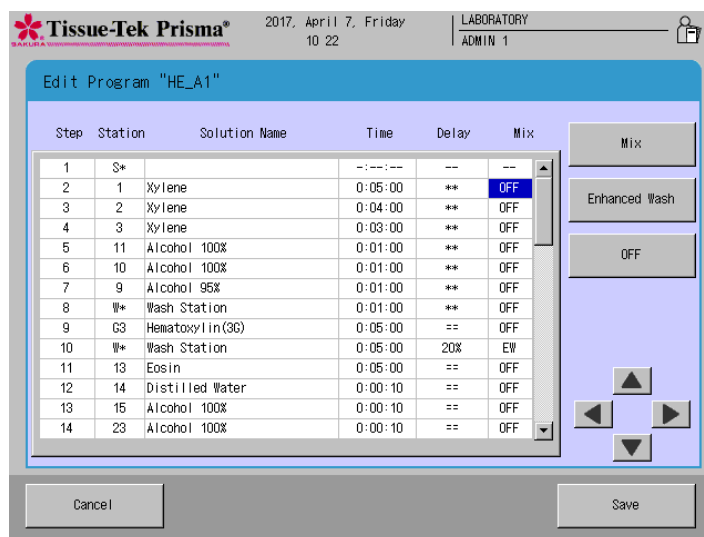

- How to operate the "Mix" field**
 In the "Mix" field, you can set whether to perform "mixing" and "enhanced wash" during staining. Use of these operations that closely simulate manual staining prevents mottled colors and co-staining to enhance the staining quality.
 - Mix**
 Touching the **Mix** key displays the screen where you can set mixing.
 - Enhanced Wash**
 Touching the **Enhanced Wash** key displays the screen where you can set enhanced wash.
 - OFF**
 Touching the **OFF** key disables both mixing and enhanced wash.

Each setting is displayed in the "Mix" field as follows.

Touching the **Save** key saves the applicable changes and closes this screen.


Mixing setting	Text displayed in "Mix" field
Mix	ON
Enhanced wash	EW
OFF	OFF

Setting a Staining Program

Editing a Staining Program

This section explains the procedure to edit an existing staining program.

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen.
2. Select **Edit Program** from the Edit Menu.
3. Under Program List in the Select Program screen, touch and select the name of the staining program you want to edit. The solution configuration used for the selected staining program is displayed on the right. To change this solution configuration, touch the **Change Solution Configuration** key at the bottom and select a new solution configuration under Solution Configuration List. To use the displayed solution configuration, proceed to the next step.
4. Next, touch the **Edit** key. The Create Solution Configuration Name screen with the simple keyboard appears. To change the staining program name currently saved, enter a new name from the onscreen keyboard (using up to 20 characters). Once the staining program name has been entered, touch the **Save** key. When the screen prompts you to enter an abbreviated staining program name, enter an abbreviation from the onscreen keyboard and then touch the **Save** key to save the abbreviation. (Enter the abbreviation using up to five characters.)
5. Touching the **Save** key proceeds to the operation explained under "Creating a Staining Program" on P. 2-26.

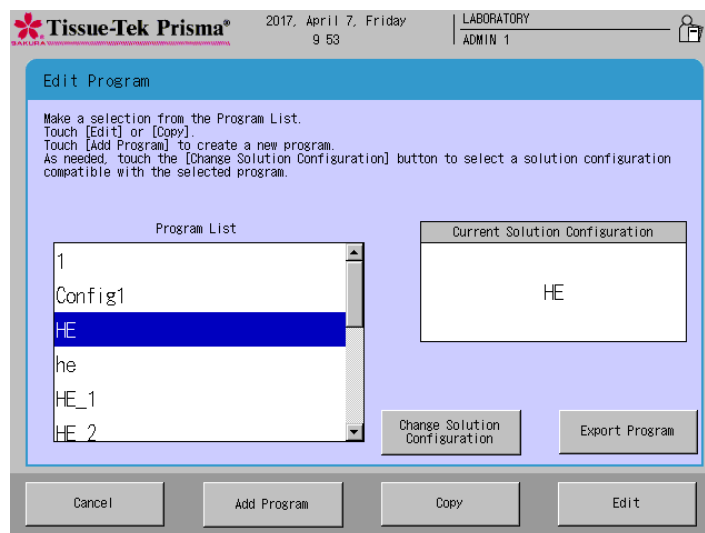


Setting a Staining Program

Copying a Staining Program

This section explains the procedure to copy a staining program.

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen.
2. Select **Edit Program** from the Edit Menu.
3. Under Program List in the Select Program screen, touch and select the name of the staining program you want to copy, and then touch the **Copy** key. The Copy Program screen appears.
4. Next, select the copying destination. To create a new solution configuration to copy to, touch the **Add Configuration** key. To copy to an existing program, select the desired program from the Copy to window and then touch the **Copy** key.
5. Touching the **Add Program** or **Copy** key displays the Create Solution Configuration Name screen showing the simple keyboard. To change the staining program name, enter a new name from the onscreen keyboard (using up to 20 characters). Once the staining program name has been entered, touch the **Save** key. When the screen prompts you to enter an abbreviated staining program name, enter an abbreviation from the onscreen keyboard and then touch the **Save** key to save the abbreviation. (Enter the abbreviation using up to five characters.)
6. Touching the **Save** key proceeds to the operation explained under "Creating a Staining Program" on P. 2-26.


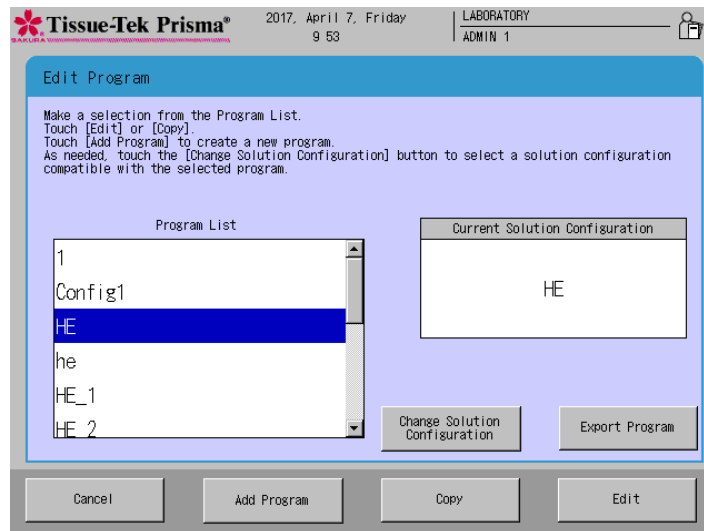


Setting a Staining Program

Checking a Staining Program

This section explains the procedure to check if a given combination of solution configuration and staining program is feasible.

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen.
2. Select **Program Check** from the Edit Menu.
3. When the Program Check screen appears, select the solution configuration you want to check under Solution Configuration List. Next, select the staining program you want to check under Program List.
4. After both the program and solution configuration have been selected, touch the **Check** key at the bottom right. The "Program Check Result" screen appears.
5. This screen shows problems with the selected combination, if any. If necessary, change the staining program and/or solution configuration by referring to the displayed check result to correct the problems.

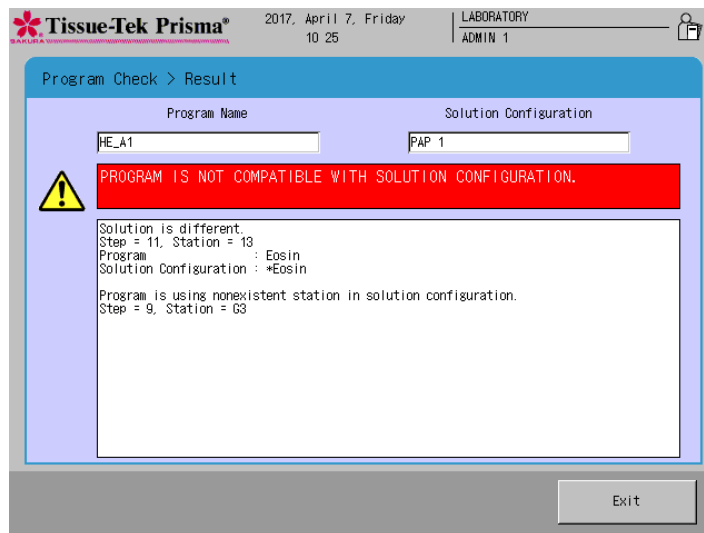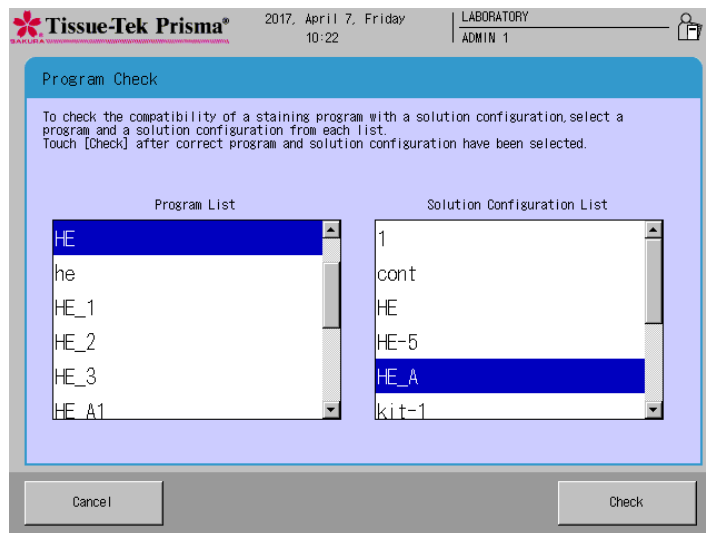

How to Use the Bar Code Reader, and It's Function (Options)

Overview

Use the bar code reader connected to the instrument to read the bar codes, so that the expiration dates and lot numbers of the staining solutions* will be entered in the instrument and reports of such solution usage information, etc., will be generated. Also, bar code management of solution information enables the following items:

- Store the solution usage information in the bar code of each solution used, so that an alarm is issued when the unopened solution expires or the opened solution expires, whichever occurs first.
- An alarm is issued at the end of operation, even on the expiration date.
(The open date is the day when the bar code is read.)

Applicable Solutions


- Sakura staining solutions*
- Staining kits*

* Check the local Sakura Finetek website for availability of the Sakura staining solutions and staining kits.


How to Use Sakura Staining Solutions

Editing a Solution Configuration

1. When a special solution is used, select **Edit Solution Configuration** [1] on the Edit Menu, to move to the Solution Configuration Setup screen.
2. On the Solution Configuration Setup screen, click the **Save** button [2] to move to the Solution Configuration screen.

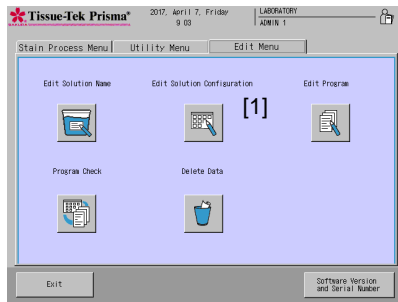



Edit Menu Screen

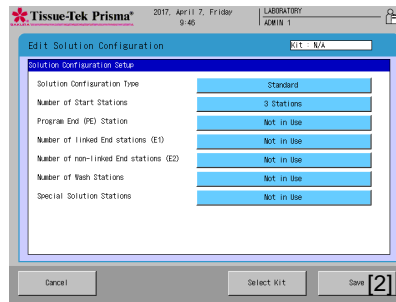

Solution Configuration Setup Screen

3. Select the station where the special solution will be used, and select **Sakura Staining Solutions** [3]. When the Select Solution window [4] appears, select the special solution to be used. When the solution is selected, the station is recognized as a Sakura special solution station.
(The details of each category are shown in the table below.)



Category	Description
Standard solutions	Solutions other than Sakura staining solutions (including user-defined solutions)
Sakura solutions	Sakura staining solutions managed by bar code * "H*" is attached at the beginning of the solution name.
Kit solutions	Solutions that constitute the Sakura staining kits managed by bar code * "H/" is attached at the beginning of the solution name.

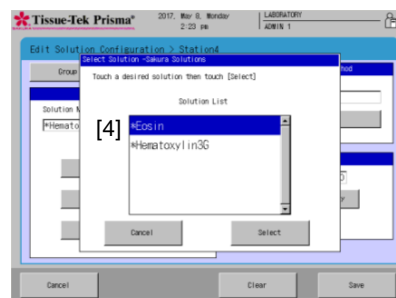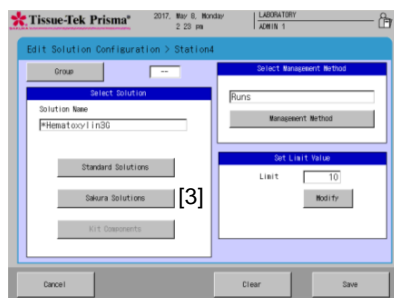


How to Use the Bar Code Reader, and It's Function (Options)

How to Enter Sakura Solution Bar Code Data

1. Prepare the solution to be set to the instrument.
2. Touch the **Read Barcode** button on the Stain Process Menu.
3. Move to the Read Barcode screen, and confirm the station for which the special solution is set. Only the stations for which bar code can be entered are displayed. The bar code reading procedure and solution state vary depending on the station/text color, so use the table below as reference.

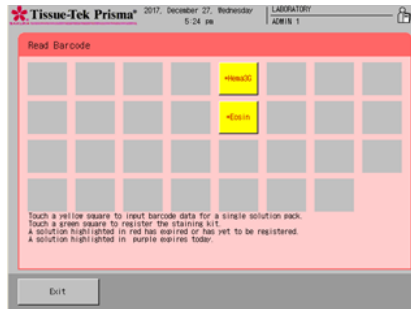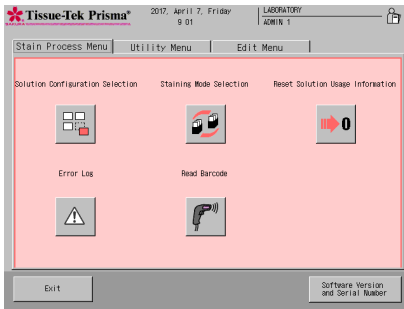

Station color	
Yellow	Sakura staining solution station
Green	Kit staining solution station
Solution name color	
Black	Unexpired solution
Purple	Solution expiring that day
Red	Solution whose bar code is not yet entered, or expired solution

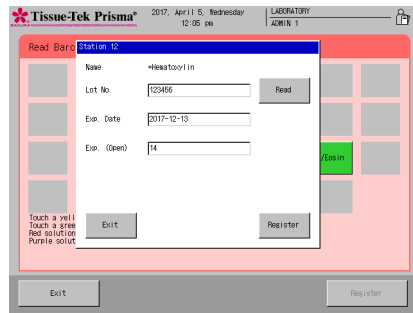


4. Select the applicable station and read the solution bar code.
5. Once the bar code of the special solution has been read, the information of the special solution is displayed on the screen.




6. If the information displayed on the screen is correct, set the special solution to the specified station.

Caution! The special solution pouch is easy to deform, so use one hand to orient its solution outlet toward the station. Also, squeezing the pouch causes the solution to gush out, so gently hold the pouch with the other hand.

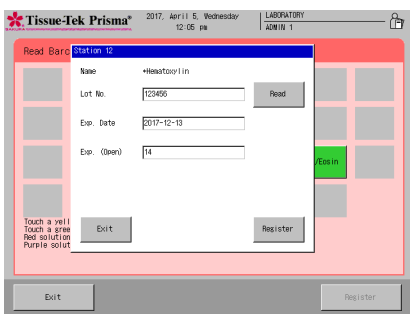

7. Once the solution has been set, touch the **Enter** button.
8. Touch the **Save** button and then touch the **Yes** button in the figure below, and the entered bar code information will be saved.

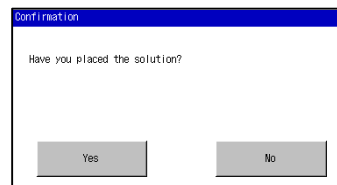


How to Use the Bar Code Reader, and It's Function (Options)


How to Use Sakura Staining Kit

Editing a Solution Configuration

1. When a staining kit is used, select **Edit Solution Configuration** [1] on the Edit Menu, to move to the Solution Configuration Setup screen.
2. On the Solution Configuration Setup screen, touch the **Select Kit** button [2] to display the Select Staining Kit window, and then set the staining kit used for the solution configuration.

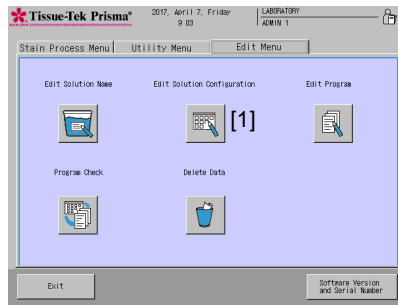

Edit Menu Screen


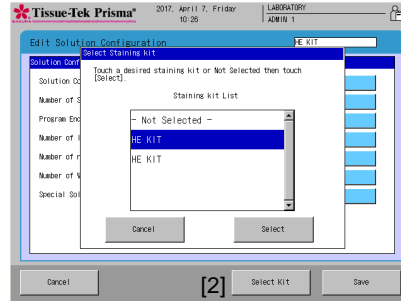


Solution Configuration Setup Screen

3. For the specified station, select **Kit Solutions** [4] from the categories shown in [3].
(The details of each category are shown in the table below.)

Category	Description
Standard solutions	Solutions other than Sakura staining solutions (including user-defined solutions)
Sakura solutions	Sakura staining solutions managed by bar code * "S" is attached at the beginning of the solution name.
Kit solutions	Solutions that constitute the Sakura staining kits managed by bar code * "/" is attached at the beginning of the solution name.



Touching the **Kit Components** button [4] displays the Select Solution Kit Components window [5], so select the solution you want to assign to the station. When the solution is selected, the station is recognized as a kit component solution station.

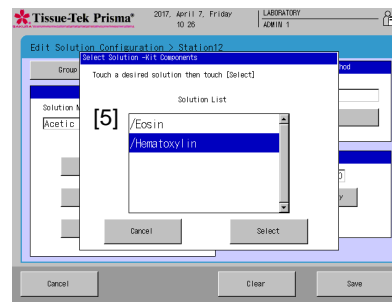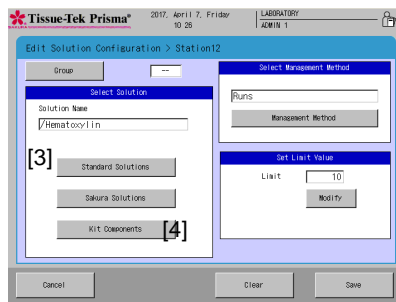


How to Use the Bar Code Reader, and It's Function (Options)

How to Enter Kit Component Solution Bar Code Data


1. Prepare the solution to be set to the instrument.
2. Touch the **Read Barcode** button on the Stain Process Menu.
3. Move to the Read Barcode screen, and confirm the station for which the special solution is set. Only the stations for which bar code can be entered are displayed. The bar code reading procedure and solution state vary depending on the station/text color, so use the table below as reference.

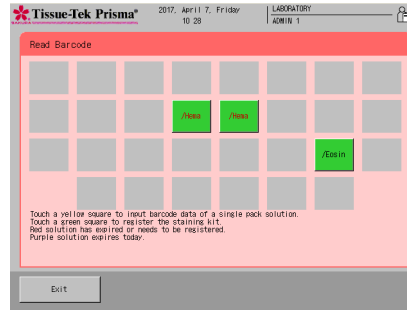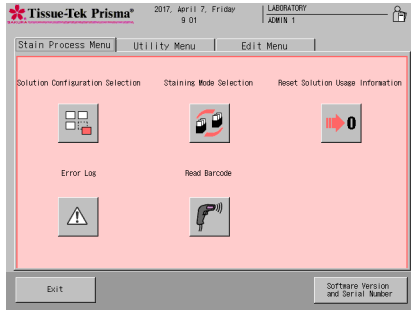

Station color	
Yellow	Sakura staining solution station
Green	Kit staining solution station
Solution name color	
Black	Unexpired solution
Purple	Solution expiring that day
Red	Solution whose bar code is not yet entered, or expired solution

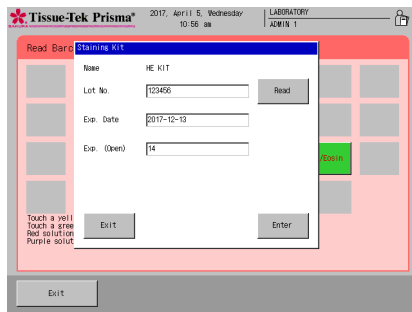


4. Select the station for which the kit is set, and read the bar code of the kit.
5. Once the kit has been read, the information of the kit is displayed on the screen. If the information displayed on the screen is correct, touch the **Enter** button.

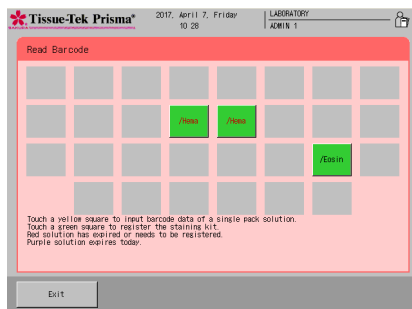

6. The bar code of the kit component solution is read.

How to Use the Bar Code Reader, and It's Function (Options)

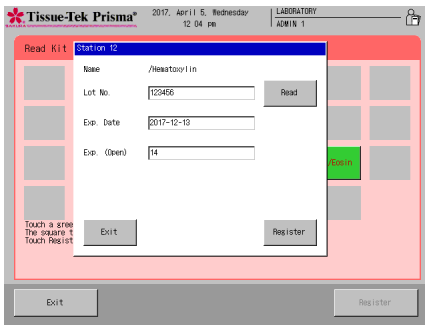


- Once the bar code of the solution has been read, the information of the solution is displayed on the screen, as with the kit.





- If the information displayed on the screen is correct, set the station containing the special solution as the specified station in the instrument.

Caution! The special solution pouch is easy to deform, so use one hand to orient its solution outlet toward the station. Also, squeezing the pouch causes the solution to gush out, so gently hold the pouch with the other hand.

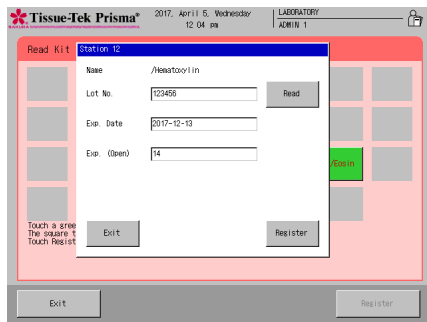

- Once the solution has been set, touch the **Register** button. Touch the **Register** button and then touch the **Yes** button in the figure below, and the entered bar code information will be saved.

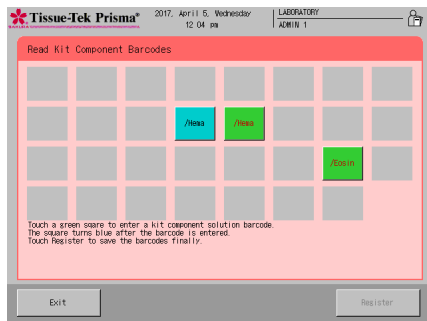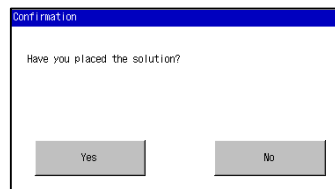

- When the **Yes** button is touched, the station to which the bar code information has been set turns blue. In the case of a kit, read all solutions constituting the kit and then touch the **Register** button in the bottom right of the screen, to complete the solution saving process.



Caution! The Register button becomes selectable after all of the solution constituting the kit have been read. Returning to the previous screen without touching the Register button resets the saved information.

How to Use the Bar Code Reader, and It's Function (Options)


Resetting the Solution Usage Information

You can select the Reset Solution Usage Information screen from the Stain Process Menu and check for expired solutions and reset their usage information.

You can also move to the Read Bar Code screen from the Reset Solution Usage Information screen.

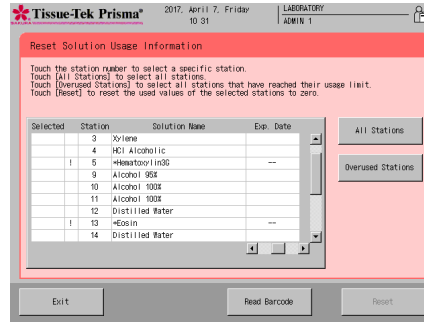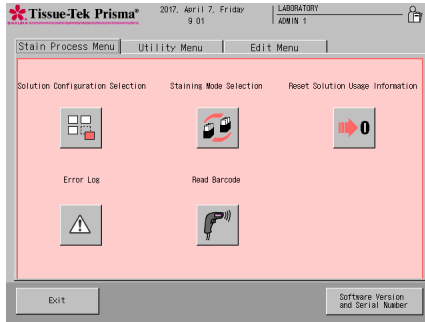

How to Use the Bar Code Reader, and It's Function (Options)

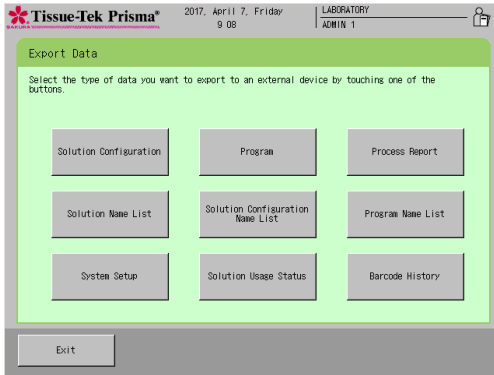

Outputting Process Report

Each report showing bar code information can be output from the Export screen.

Solution Usage Information Report

You can output this report by touching **Solution Usage Status** on the Export Data screen.

The report shows, among the bar code data of the currently used solutions and kits, only the lot numbers and the expiration dates of unopened and opened solutions/kits.

The Exp. (Use) field of the report shows the practical expiration date (unopened or opened, whichever comes first). Also, the date the bar code was read is shown as the Date Opened.

The report shows the expiration dates of individual kit solutions separately.

In addition to the bar code of each station, the bar code of each kit is also shown.

[Example]

Station	Solution	Count	Set	Management	Updated date
1	Xylene	--	--	--	--
2	Xylene	--	--	--	--
3	Xylene	--	--	--	--
4	HCl Alcoholic	--	--	--	--
5	/Hematoxylin	2	20	Runs	2017-03-08 13:05:20
6 (W2)	Wash Station	--	--	--	--
7 (W1)	Wash Station	--	--	--	--
8 (D1)	Drying Station	--	--	--	--
9	Alcohol 95%	1	10	Days	2017-03-08 13:05:20
10	Alcohol 100%	2	10	Days	2017-03-08 13:05:20
11	Alcohol 100%	--	--	--	--
12	Distilled Water	--	--	--	--
13	/Eosin	--	--	--	--
14	Distilled Water	--	--	--	--
15	Alcohol 95%	--	--	--	--
16 (D2)	Drying Station	--	--	--	--
17	Xylene	--	--	--	--
18	Xylene	--	--	--	--
19	Xylene	--	--	--	--
20	Alcohol 100%	--	--	--	--
21	Alcohol 100%	--	--	--	--
22	Alcohol 100%	--	--	--	--
23	Alcohol 95%	--	--	--	--
24(H1)	--	--	--	--	--
25(H2)	--	--	--	--	--
26(E1-1)	End Station1	--	--	--	--
27(E1-2)	End Station1	--	--	--	--
28(E1-3)	End Station1	--	--	--	--
29(S3)	Start Station	--	--	--	--
30(S2)	Start Station	--	--	--	--
31(S1)	Start Station	--	--	--	--

Station	Lot Number	Exp.(Unopened)	Exp. Time	Date Opened	Exp.(Use)
5	123456	2017-12-13	14	2017-03-08	2017-03-21
13	123456	2017-12-13	14	2017-03-08	2017-03-21

Kit information				
H&Ekit250	Lot Number	Exp.(Unopened)	Exp. Time	Exp.(Use)
	123456	2017-12-13	14	2017-03-08 2017-03-21

How to Use the Bar Code Reader, and It's Function (Options)

Process Report

You can output this report by touching **Process Report** on the Export screen. The report shows, among the bar code data of the currently used solutions and kits, only the lot numbers and the expiration dates of unopened and opened solutions/kits.

The Exp. (Use) field of the report shows the practical expiration date (unopened or opened, whichever comes first). Also, the date the bar code was read is shown as the Date Opened. The report shows the expiration dates of individual kit solutions separately. The solution usage status at the end of processing is shown. In addition to the bar code of each station, the bar code of each kit is also shown.

The Usage field shows the solution usage status. In the case of a group, the stations actually used, and the usage status of each station used, are shown.

[Example]

Process Report						
Exported Date: 2017-03-08 1:07:06 PM						
Start:2017-03-08 11:27:34						
End :2017-03-08 12:06:33						
Program Name: HE_kit						
Specimen No : 2						
Step	Station	Solution	Time	Result	Delay	Mix Usage
1	S*	Start Station	--:--:--	00:00:14		
2	1	Xylene	00:05:00	00:05:00	**	ON
3	2	Xylene	00:04:00	00:04:00	**	ON
4	3	Xylene	00:03:00	00:03:00	**	ON
5	11	Alcohol 100%	00:01:00	00:01:00	**	ON
6	10	Alcohol 100%	00:01:00	00:01:00	**	ON
7	9	Alcohol 95%	00:01:00	00:01:00	**	ON
8	W*	Wash Station	00:01:00	00:01:00	**	ON
9	5	/Hematoxylin	00:05:00	00:05:00	==	ON Runs
11/20						
10	W*	Wash Station	00:05:00	00:05:00	20%	EW
11	13	/Eosin	00:05:00	00:05:00	==	ON
12	14	Distilled Water	00:00:10	00:00:10	==	ON
13	15	Alcohol 95%	00:00:10	00:00:10	==	ON
Days 1/10						
14	23	Alcohol 95%	00:00:10	00:00:10	==	ON
Days 2/10						
15	22	Alcohol 100%	00:01:00	00:01:00	20%	ON
16	21	Alcohol 100%	00:01:00	00:01:00	20%	ON
17	20	Alcohol 100%	00:01:00	00:01:00	**	ON
18	19	Xylene	00:01:00	00:01:00	**	ON
19	18	Xylene	00:01:00	00:01:00	**	ON
20	17	Xylene	00:01:00	00:01:00	**	ON
21	E1-*	End Station 1	--:--:--	00:58:53		
Sakura solution information						
Station	Lot Number	Exp.(Unopened)	Exp. Time	Date Opened	Exp.(Use)	
5	123456	2017-12-13	14	2017-03-08	2017-03-21	
6	123456	2017-12-25	14	2017-01-05	2017-01-18	
13	123456	2017-12-13	14	2017-03-08	2017-03-21	
14	123456	2017-12-13	14	2017-01-05	2017-01-18	
Kit information						
H&Ekit250						
Lot Number	Exp.(Unopened)	Exp. Time	Date Opened	Exp.(Use)		
123456	2017-12-13	14	2017-03-08	2017-03-21		

How to Use the Bar Code Reader, and It's Function (Options)

Solution Configuration Report

You can output this report by touching **Solution Configuration List** on the Export Data screen. The solution configuration report also shows the staining kit name.

[Example]

Solution		Configuration		List
				Exported Date: 2017-03-08 13:06:46
				Created Date : 2017-01-27 10:40:34
				Revised Date : 2017-03-08 10:37:23
Configuration				
Name:HE_kit Kit				
Name:H&Ekit250				
Station	Solution	Set	Management	
1	Xylene	--	--	
2	Xylene	10	Runs	
3	Xylene	10	Runs	
4	HCl Alcoholic	--	--	
5	/Hematoxylin	--	--	
6 (W2)	Wash Station	--	--	
7 (W1)	Wash Station	--	--	
8 (D1)	Drying Station	--	--	
9	Alcohol 95%	--	-	
10	Alcohol 100%	--	--	
11	Alcohol 100%	--	--	
12	Distilled Water	--	--	
13	/Eosin	--	--	
14	Distilled Water	--	--	
15	Alcohol 95%	--	--	
16(D2)	Drying Station	--	--	
17	Xylene	--	--	
18	Xylene	--	--	
19	Xylene	--	--	
20	Alcohol 100%	--	--	
21	Alcohol 100%	--	--	
22	Alcohol 100%	--	--	
23	Alcohol 95%	--	--	
24(H1)	--	--	--	
25(H2)	--	--	--	
26(E1-1)	End Station 1	--	--	
27(E1-2)	End Station 1	--	--	
28(E1-3)	End Station 1	--	--	
29(S3)	Start Station	--	--	
30(S2)	Start Station	--	--	
31(S1)	Start Station	--	--	
		Color	Information	
G1	Green			
G2	Cream			
G3	Yellow			
G4	Pink			
G5	Magenta			
		Station	Parameters	
Mix Amplitude			30 mm	
Mix Frequency			5	
Mix Speed			3	
Enhanced Wash Frequency			5	
Enhanced Wash Speed			3	
Basket Lifting Speed			1	
Water Wash Cycle Time			60 seconds	
Drying Station Temperature			65 C.	
Heating Station(1) Temperature			60 C.	
Heating Station(2) Temperature			60 C.	
Heat up Heating Stations			Yes	

How to Use the Bar Code Reader, and It's Function (Options)

Barcode History Report

You can output this report by touching **Barcode History** on the Export Data screen. The barcode history shows the GTIN of each barcode that has been read, lot number, and the date the barcode was read, for up to 1,000 most recently read barcodes.


[Example]

Barcode History report		
Barcode	History	Exported Date: 2015-06-27 16:47:23
GTIN	Lot Number	Date Opened
04560133241122	316510	2015-06-19
04560133241105	216513	2015-06-19
04560133241105	216514	2015-06-19
04560133241207	116511	2015-06-19
04560133241115	516215	2015-06-19

Deleting Data


Deleting a Solution Configuration/Staining Program/Solution Name/Process Report

This section explains the procedure to delete each item (solution configuration/staining program/solution name/process report).

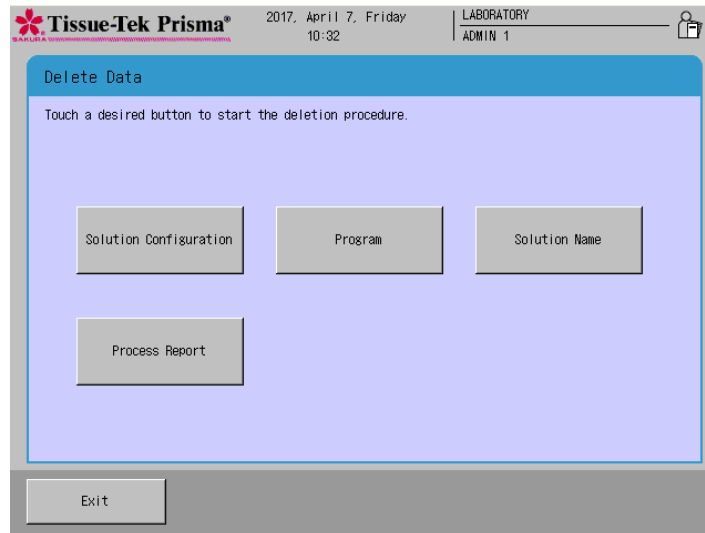


Deleting a Solution Configuration

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen, and then select **Delete Data** in the Edit Menu.
2. When the Delete Data screen appears, select **Solution Configuration**.
3. Touching the key displays the Deletion Procedure window. Under Solution Configuration List, select the solution configuration you want to delete. This adds a checkmark "✓" to the Check field. You can select and delete multiple solution configurations at once. You can also touch **All Data** to select all solution configurations in the list. Only solution configurations not used by the available staining programs are shown under Solution Configuration List. If the solution configuration you want to delete is not in the list, delete all staining programs that use the applicable solution configuration and then perform this operation again.
4. Once the target item has been selected, touch **Delete**. When the Confirmation screen appears, touch **Yes** to complete the deletion.

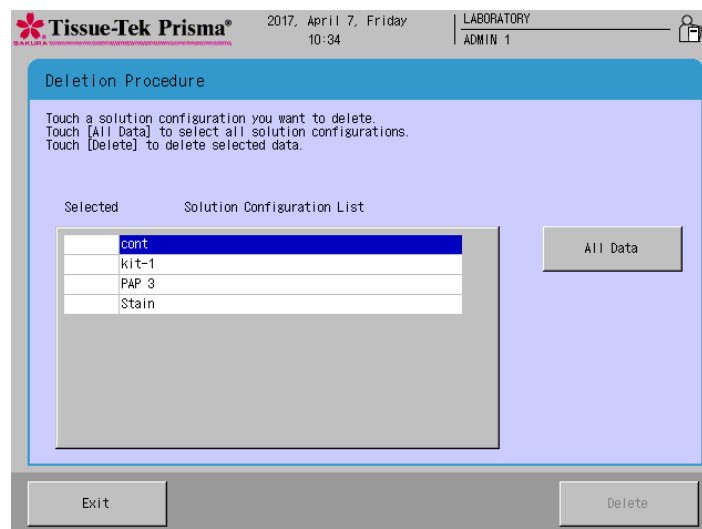


Deleting Data

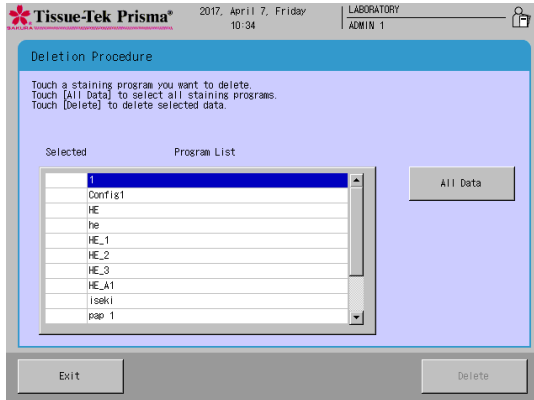


Deleting a Staining Program

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen, and then select **Delete Data** in the Edit Menu.
2. When the Delete Data screen appears, select **Program**.
3. Touching the key displays the Deletion Procedure window. Under Program List, select the staining program you want to delete. This adds a checkmark "✓" to the Check field. You can select and delete multiple staining programs at once. You can also touch **All Data** to select all solution configurations in the list.
4. Once the target item has been selected, touch **Delete**. When the Confirmation screen appears, touch **Yes** to complete the deletion.

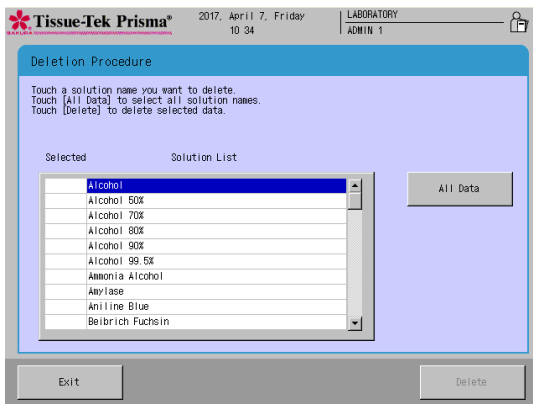

Deleting a Solution Name

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen, and then select **Delete Data** in the Edit Menu.
2. When the Delete Data screen appears, select **Solution Name**.
3. Touching the key displays the Deletion Procedure window. Under Solution List, select the solution name you want to delete. This adds a checkmark "✓" to the Check field. You can select and delete multiple solution names at once. You can also touch **All Data** to select all solution configurations in the list. Only solutions not used by the available staining programs are shown under Solution List. If the solution name you want to delete is not in the list, delete all staining programs that use the applicable solution and then perform this operation again.
4. Once the target item has been selected, touch **Delete**. When the Confirmation screen appears, touch **Yes** to complete the deletion.

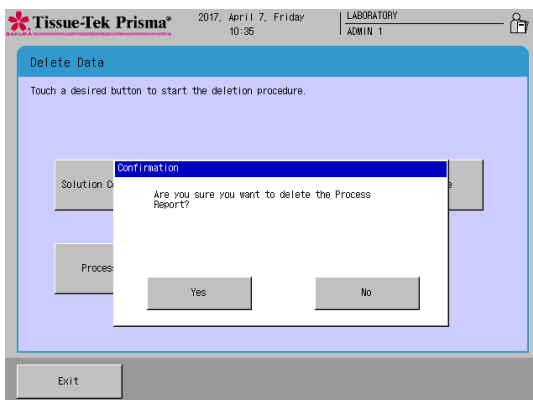

Deleting a Process Report

1. Touch **Menu** at the bottom right of the Stain Process Monitor screen, and then select **Delete Data** in the Edit Menu.
2. When the **Delete Data** screen appears, select **Process Report**.
3. Once the target item has been selected, touch **Delete**. When the Confirmation screen appears, touch **Yes** to complete the deletion.





Note Up to 50 process reports can be saved. If the 51st process report is saved, the oldest report will be automatically deleted to make room for the latest report.

Action to Be Taken upon Power Outage

Power Recovery Operation

This section explains the operation to be performed if a power outage occurred during the stain process. Once the power is restored, perform the power recovery operation explained below.

1. Once the power is restored, an error buzzer sounds and the Power Outage Alarm screen or Arm Error window appears on the control panel display to inform the operator that a power outage has occurred.

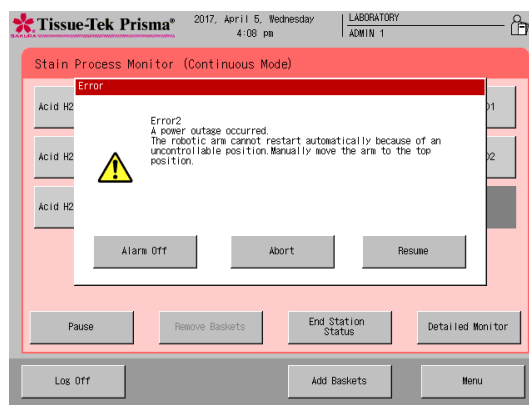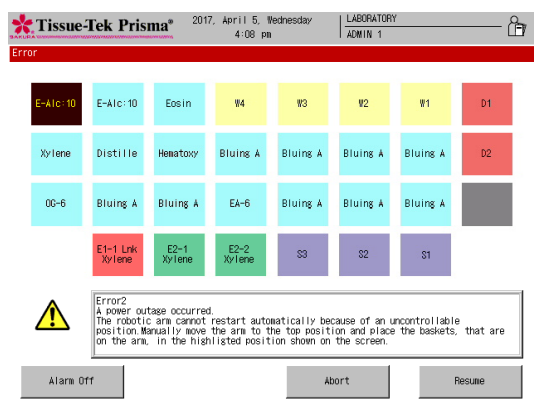

2. The operator must evacuate the arm to the initial position according to the onscreen instruction. If any basket is currently set on the arm, remove the basket from the arm according to the onscreen instruction and then place it in the station shown in brown.
3. To resume stain process, touch the **Resume** key. Because of the power outage, the staining may not end by the initially scheduled end time. In this case, the specified time may be delayed by up to the period after the power outage occurred until the **Resume** key was touched.
4. To abort stain process, touch the **Abort** key. Refer to **P. 2-11** for the method to abort staining.



A prolonged power outage may cause the tissues to dry up or result in staining problems. If you know that a power outage will occur, do not use the instrument. If a prolonged power outage has occurred due to an unexpected problem, take an appropriate action to prevent the tissues from drying up, and then manually perform the remainder of the staining process.

Action to Be Taken upon Error

Error Handling

During operation, the instrument may stop after displaying an error. This section explains the actions to be taken when errors occur. Most errors accompany a number. Some errors temporarily stop the series of operations such as staining runs, while other errors display a caution/confirmation window prompting the operator to check the applicable items. Representative error/caution/confirmation screens are shown below.

Errors with a Number







Most errors accompany a number and are displayed as follows. You can check the action to be taken against each error by referring to the section corresponding to the applicable number in Chapter 4, "Troubleshooting." If further assistance is needed regarding your particular problem, if the problem persists after taking the specified action, or if the specific problem you have encountered is not listed, contact the Sakura Finetek Technical Support representative or local representative.

Other Errors

If the error does not accompany a number, follow the onscreen instruction shown in the error window.



In the event of an unexpected problem for which you cannot find any solution, immediately turn off the power. Next, open the cover, remove the tissues, and protect them so as not to let them dry up. Then, contact the Sakura Finetek Technical Support representative or local representative.

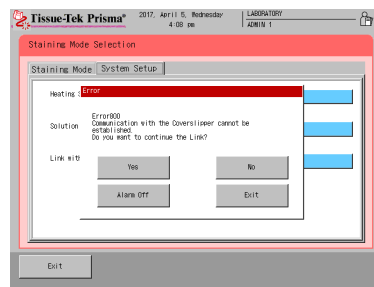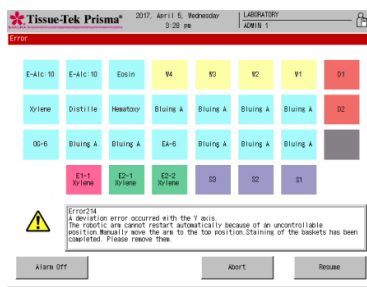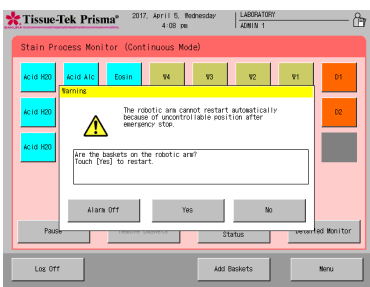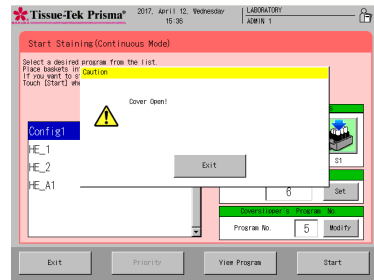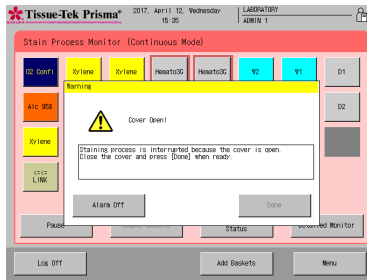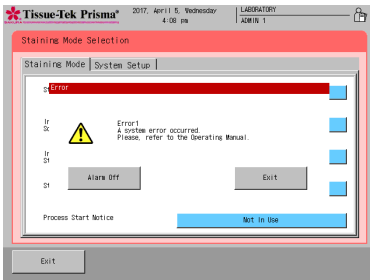



Chapter 3

Maintenance and Inspection



Daily Inspection and Cleaning

Inspection and Cleaning Methods

Inspect and clean the instrument periodically to prevent instrument malfunction and failure. Unless otherwise specified in this manual, perform inspection and cleaning with the power turned off.

Cleaning the Solution Reservoirs and Baskets


Clean solution reservoirs and baskets with water or neutral detergent. If solution has attached to a reservoir/basket and the color or stain does not come off easily, soak the reservoir/basket in a commercial chemical dye cleaner or bleach (diluted to the manufacturer's instruction), and then clean with neutral detergent. Sometimes cleaning does not fully remove built-up stains. To prevent stains from building up, it is recommended that the solution reservoirs/baskets be cleaned after each staining. Alternatively, use different solution reservoirs/baskets for each solution so that the reservoirs/baskets with solution stains can still be used.

Cleaning the Drying Stations


In the instrument, the two rightmost stations located at the back are drying stations. First, remove the two special attachments from the drying stations. Next, clean the stations using neutral detergent, and then wipe down with a dry cloth. Store the stations by following the removal steps in the reverse order. If the drying stations are not set properly, the drying function will not operate properly.



Attachment

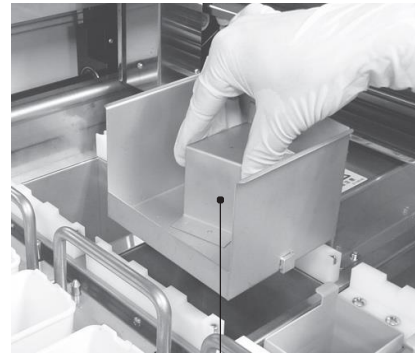

Paraffin tray

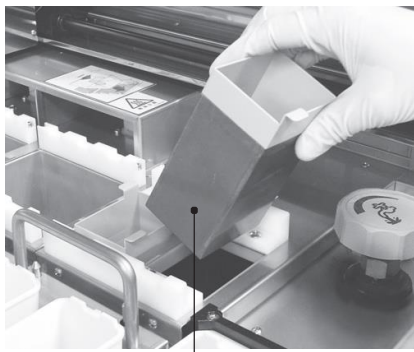

Special heating station reservoir


Cleaning the Heating Stations

In the instrument, the two rightmost stations located at the front are heating stations. Remove the heating stations by holding the two tongues at the edges of the heating station. Exercise due caution when handling the heating stations, because these stations remain very hot immediately after any operation in which the heating stations have been used. To clean the heating stations, clean with neutral detergent and then wipe down with a dry cloth. After cleaning, store the stations in the original positions. The interior of the heating stations is coated with Teflon. When cleaning, make sure the coating is not rubbed with sharp edges, as the coating will be scraped off.

Daily Inspection and Cleaning

Replacing Activated Carbon Filters

When to replace the activated carbon filters can be easily determined by setting a time limit using the fume filter management function (refer to **P. 1-41** for details). Always replace the two activated carbon filters in each filter pair together. If the above is not followed, the overall performance of the activated carbon filters may significantly decrease. To replace the activated carbon filters, find a black knob located in the filter storage area. Tilt down the knob while pulling it toward you to draw out the filter storage door. After the filters have been replaced, store the filter storage door into the original position. Push in the door until a click sound is heard. If the filter storage door is not set properly, the storage door may contact the arm and cause problems.

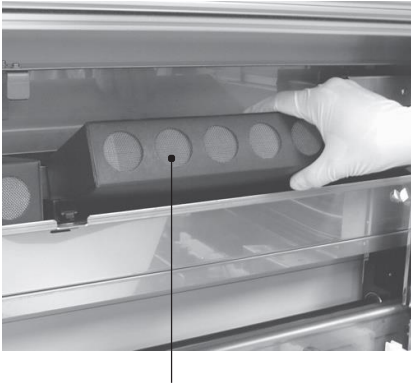

Activated carbon filter
* One filter is set on the right and left, respectively.




Activated carbon filter
Open/close knob

Cleaning the Arm Holder and Basket Adapter

The arm holder groove and the metal part of the basket adapter become dirty due to repeated staining operations. Use a cloth, etc., moistened with alcohol to clean the arm holder and basket adapter. Clean them at least once a month.

Basket adapter
metal part

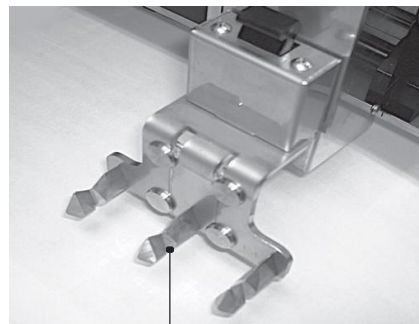


Arm holder groove

Daily Inspection and Cleaning


Cleaning the Water Supply Strainer (Excluding Prisma-P-ED, ES)

Close the main water valve. Open the water supply strainer door at the bottom right of the system. Opening the cover makes opening the door easier. Use a wrench of 17 mm across flats to turn the lid counterclockwise and remove the lid. Water in the pipe will come out, so use a container, etc., to catch the water. Wash the strainer with water using a brush, etc. After washing the strainer, dry and assemble the strainer, and screw in the lid. Open the water supply plug and confirm that no water leaks from the strainer. Clean the strainer once a month or so. After the cleaning, perform a water supply test by referring to

P. 1-42.

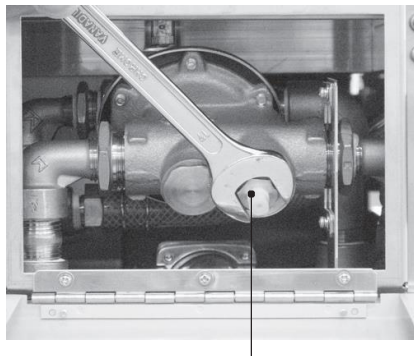

Water supply strainer lid

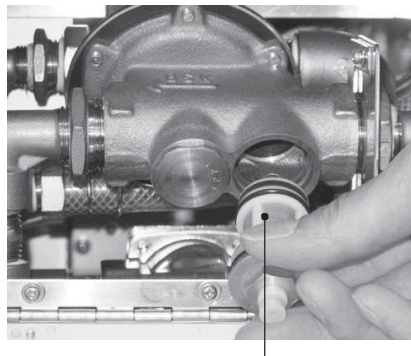

Water supply strainer mesh

Cleaning the Cabinet

Clean the entire storage space for the system and solution stations as well as the sink and pans at least once a month. Remove all trays from the cabinet and clean them using a cloth moistened with water. Also clean the trays with neutral detergent, and then wipe down with a dry cloth.



In addition to the interior of the instrument, also clean the cover, door and around the exhaust port regularly. At this time, be sure to turn off the power and disconnect the power plug from the power outlet. Failure to do so may result in electric shock.

<Corrosion Protection>

The Tissue-Tek Prisma uses corrosion-resistant stainless steel, but the stainless steel may rust depending on a combination of factors such as the environment (water, air), staining method (type of solution, concentration), use method (handling of solvent, maintenance), etc. Take note of the following points to protect the instrument from corrosion.

Keep the solution reservoir lids closed except during the staining process. Remove those solution reservoirs containing highly corrosive solutions, from the trays, and keep the solutions outside the instrument. If a highly corrosive solution spilled in the instrument, wipe it off immediately.

Remove all solution trays (including the solution reservoirs) from the instrument once a week or so, wipe down the trays using a cloth moistened with water, clean them with neutral detergent, and wipe down with a dry cloth.

Corrosive solutions

Acids [hydrochloric acid (HCL), hydrobromic acid (Hbr), hydrofluoric acid (HF), etc.] and staining solutions containing an acid

Reagent solutions [Schiff's reagent, cold Schiff, sulfurous acid solution, resorcin - fuchsin staining solution, etc.]

Oxidizing chlorides [ferric chloride (FeCl₃), sodium tiosulfate (Na₂S₂O₃), organic acid (HCOOH), acetic acid (CH₃COOH), bromic acid (C₂O₄H₂), lactic acid (CH₃CH(OH)COOH), etc.]

If this is not performed, the metal parts may corrode (rust, etc.) quickly. Contact the Sakura Finetek Technical Support representative or local representative for use of metal cleaners.

Daily Inspection and Cleaning

Cleaning the Wash Station Nozzle

Remove the wash station nozzle and clean each part using neutral detergent. Since this is a wash station, the nozzle need not be wiped down. However, perform a water supply test after installing the nozzle and setting the solution reservoir, to confirm that the wash station nozzle has been connected correctly. Refer to **P. 1-13** for the installation/removal of the wash station nozzle, or **P. 1-42** for the water supply test.

Cleaning the Drain Hose

If the drain hose at the back of the stainer is clogged, staining may be affected and unwanted problems may result. To prevent such problems, clean the drain hose at least once a month. Remove the drain hose from the drain elbow and clean the inside of the hose using tap water. If the hose is still clogged after the cleaning or is cracked and leaking water, etc., replace the hose with a new one. Refer to **P. 1-10** for the installation method of the drain hose.

If you have any question regarding cleaning, contact the Sakura Finetek Technical Support representative or local representative.



Chapter 4

Troubleshooting



Troubleshooting

Error Codes Displayed during System Operation

The troubleshooting chart below lists problems that may occur when operating the system. Each problem description is accompanied by possible causes and actions to be taken. If you have encountered a problem, use this chart to determine the instrument condition and take an appropriate action. If the problem persists after taking the specified action, or if the specific problem you have encountered is not listed in the chart, contact the Sakura Finetek Technical Support representative or local representative. The number in the table indicates the corresponding error number.

*** In the event of an unexpected problem for which you cannot find any solution, immediately turn off the power. Then, contact the Sakura Finetek Technical Support representative or local representative.**

Error number	Name	Cause	Action
1	System Error	The staining program does not end 30 seconds after the scheduled time.	Contact the Sakura Finetek Technical Support representative or local representative.
2	Power Outage	A power outage has occurred due to abnormal power supply from the facility.	Contact the Sakura Finetek Technical Support representative or local representative.
		The power switch was turned off during the staining process.	Turn on the power switch.
		The power cord came off from the power outlet during the staining process.	Connect the power cord.
3	Power Outage Recovery Failure	A power outage signal was sent to the instrument from the UPS.	This is normal when the facility power was cut off. Wait for the power to be restored.
		Faulty memory on the CPU board of the control panel. The voltage of the battery unit installed in the memory of the CPU board of the control panel dropped.	Contact the Sakura Finetek Technical Support representative or local representative.
10	Cover Open	The operator opened the cover during staining.	Contact the Sakura Finetek Technical Support representative or local representative.
		The cover deformed and created a gap between the detection sensor and the detection plate during the staining process.	
		The instrument malfunctioned during the staining process due to a cover open/close sensor error.	
		The instrument malfunctioned during the staining process due to a controller board input error.	
		The control panel connection harness connecting the CPU board of the control panel and the controller board was disconnected during the staining process.	
		The instrument malfunctioned during the staining process due to an input error of the CPU board of the control panel.	
		The instrument malfunctioned during the staining process due to a CPU error of the CPU board of the control panel.	
11	Cover Open (when not staining)	The operator opened the cover while the arm was operating other than during the staining process.	Close the cover and resume the process.
		The cover deformed and created a gap between the detection sensor and the detection plate while the arm was operating other than during the staining process.	Contact the Sakura Finetek Technical Support representative or local representative.
		The instrument malfunctioned due to a cover open/close sensor error while the arm was operating other than during the staining process.	
		The instrument malfunctioned due to a controller board input error while the arm was operating other than during the staining process.	
		The control panel connection harness connecting the CPU board of the control panel and the controller board shorted while the arm was operating other than during the staining process.	
		The instrument malfunctioned due to an input error of the CPU board of the control panel while the arm was operating other than during the staining process.	
		The instrument malfunctioned due to a CPU error of the CPU board of the control panel while the arm was operating other than during the staining process.	
20	Processing Halt	The operator paused the instrument during the staining process.	Resume the process.

Troubleshooting

Error number	Name	Cause	Action
21	Door Open	The operator opened a door while the arm was removing a basket from a station in the frontmost row during the staining process.	Close the door and resume the process.
		The operator opened a door while the arm was storing a basket into a station in the frontmost row during the staining process.	
		An incompletely latched door suddenly opened during the staining process.	Contact the Sakura Finetek Technical Support representative or local representative.
		The instrument malfunctioned during the staining process due to a door open/close sensor error.	
		The instrument malfunctioned during the staining process due to a controller board input error.	
		The control panel connection harness connecting the CPU board of the control panel and the controller board shorted during the staining process.	
		The instrument malfunctioned during the staining process due to an input error of the CPU board of the control panel.	
		The instrument malfunctioned during the staining process due to a CPU error of the CPU board of the control panel.	
45	Bar Code Reader Circuit Overcurrent	The bar code reader or connection cable is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
		The external I/F board is faulty.	
		The controller board is faulty.	
		The external I/F connection harness is faulty.	
50	Exhaust Fan Stop	The exhaust fan stopped due to an error of the fan.	Contact the Sakura Finetek Technical Support representative or local representative.
		Due to an error of the exhaust fan, the fan stop detection signal was output even when the fan was turning.	
		Due to a sensor input error of the controller board, the fan stop detection signal was not transmitted.	
		The wiring exhaust fan relay harness experienced a contact failure/disconnection.	
70	Battery Voltage Low	The battery unit reached its life.	Contact the Sakura Finetek Technical Support representative or local representative.
80	Motor Unit Communication Error	The control panel connection harness was disconnected.	Contact the Sakura Finetek Technical Support representative or local representative.
		The CPU board of the control panel is faulty.	
		The controller board is faulty.	
81	I/O Unit Communication Error	The control panel connection harness was disconnected.	Contact the Sakura Finetek Technical Support representative or local representative.
		The CPU board of the control panel is faulty.	
		The controller board is faulty.	
90	Vacuum Breaker Error	Water leaked and caused the float switch to operate.	Contact the Sakura Finetek Technical Support representative or local representative.
		The float switch malfunctioned.	
		The switch malfunctioned due to a sensor input error of the controller board.	
100	Robotic Arm Error	Deviation of each axis	Contact the Sakura Finetek Technical Support representative or local representative.
		Program error	
111	Home Position Sensor Error (X axis)	The X-axis slow-down sensor is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
		The X-axis home sensor is faulty.	
		The controller board is faulty.	
112	Wire Slack (X axis)	The X-axis wire assembly is loose.	Contact the Sakura Finetek Technical Support representative or local representative.
114	Deviation Error (X axis)	Movement of the X-axis is inhibited by an obstacle or faulty wire drive.	Contact the Sakura Finetek Technical Support representative or local representative.
		The X-axis home sensor is faulty.	
		The X-axis encoder is faulty.	
		The X-axis motor is faulty.	
		The controller board is faulty.	
115	Wire Cut Off (X axis)	The X-axis wire broke or came off from the pulley, etc.	Contact the Sakura Finetek Technical Support representative or local representative.
		The X-axis wire breakage sensor is faulty.	
		The controller board is faulty.	
211	Home Position Sensor Error (Y axis)	The Y-axis slow-down sensor is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
		The Y-axis home sensor is faulty.	
		The controller board is faulty.	
212	Wire Slack (Y axis)	The Y-axis wire assembly is loose.	Contact the Sakura Finetek Technical Support representative or local representative.

Troubleshooting

Error number	Name	Cause	Action
214	Deviation Error (Y axis)	Movement of the Y-axis is inhibited by an obstacle or faulty wire drive. The Y-axis home sensor is faulty. The Y-axis encoder is faulty. The Y-axis motor is faulty. The controller board is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
215	Wire Cut Off (Y axis)	The Y-axis wire broke or came off from the pulley, etc. The Y-axis wire breakage sensor is faulty. The controller board is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
311	Home Position Sensor Error (Z axis)	The Z-axis home sensor is faulty. The controller board is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
312	Wire slack (Z axis)	The Z-axis wire assembly is loose.	Contact the Sakura Finetek Technical Support representative or local representative.
314	Deviation Error (Z axis)	Movement of the Z-axis is inhibited by an obstacle or faulty wire drive. The Z-axis home sensor is faulty. The Z-axis encoder is faulty. The Z-axis motor is faulty. The controller board is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
315	Wire Cut Off (Z axis)	The Z-axis wire broke or came off from the pulley, etc. The Z-axis wire breakage sensor is faulty. The controller board is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
430	Sensor Opened (Drying Station 1)	An abnormal value was input to the signal amplification part of the controller board due to a disconnected temperature sensor. The drying station connection harness connecting the temperature controller board and the controller board was disconnected. A disconnected harness was recognized by CPU1 due to a signal amplification error of the controller board. Normal judgment was disabled due to a CPU1 error of the controller board.	Contact the Sakura Finetek Technical Support representative or local representative.
431	Sensor Shorted (Drying Station 1)	An abnormal value was input to the signal amplification part of the controller board due to a shorted temperature sensor. The drying station connection harness connecting the temperature controller board and the controller board shorted. A shorted harness was recognized by CPU1 due to a signal amplification error of the controller board. Normal judgment was disabled due to a CPU1 error of the controller board.	Contact the Sakura Finetek Technical Support representative or local representative.
432	Temperature Low (Drying Station 1)	Power could not be supplied to the heater due to a disconnection or connection failure of the heater. Power could not be supplied to the heater due to a SSR error of the temperature controller board. Power could not be supplied to the drying heater due to a disconnection of the drying station connection harness connecting the temperature controller board and the controller board. Power could not be supplied to the heater due to a CPU1 error of the controller board. The heater voltage selector switch is set incorrectly. The power-supply voltage selector switch is set incorrectly.	Contact the Sakura Finetek Technical Support representative or local representative.
433	Thermal Fuse (Drying Station 1)	The SSR3 on the temperature controller board experienced a short-circuit failure. The heater remained on due to a driver error of the controller board. A false detection occurred due to an error of the SSR1 for power-supply voltage detection. The drying station connection harness connecting the temperature controller board and the controller board was disconnected. The instrument malfunctioned due to an over-temperature protection input error of the controller board. The heater voltage selector switch is set incorrectly. The power-supply voltage selector switch is set incorrectly.	Contact the Sakura Finetek Technical Support representative or local representative.

Troubleshooting

Error number	Name	Cause	Action
434	Fan Stop (Drying Station 1)	The fan is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
		An operation signal was not output due to a controller board error.	
		The connection cable connecting the temperature controller board and the controller board was disconnected.	
		The fan could not be driven due to a driver error of the temperature controller board.	
		The connection cable connecting the temperature controller board and the controller board was disconnected.	
		A stopped fan was recognized by CPU1 due to a signal input error of the controller board.	
530	Sensor Opened (Drying Station 2)	Normal judgment was disabled due to a CPU1 error of the controller board.	Contact the Sakura Finetek Technical Support representative or local representative.
		An abnormal value was input to the signal amplification part of the controller board due to a disconnected temperature sensor.	
		The drying station connection harness connecting the temperature controller board and the controller board was disconnected.	
		A disconnected harness was recognized by CPU1 due to a signal amplification error of the controller board.	
531	Sensor Shorted (Drying Station 2)	Normal judgment was disabled due to a CPU1 error of the controller board.	Contact the Sakura Finetek Technical Support representative or local representative.
		An abnormal value was input to the signal amplification part of the controller board due to a shorted temperature sensor.	
		The drying station connection harness connecting the temperature controller board and the controller board shorted.	
		A shorted harness was recognized by CPU1 due to a signal amplification error of the controller board.	
532	Temperature Low (Drying Station 2)	Normal judgment was disabled due to a CPU1 error of the controller board.	Contact the Sakura Finetek Technical Support representative or local representative.
		Power could not be supplied to the heater due to a disconnection or connection failure of the heater.	
		Power could not be supplied to the heater due to a SSR error of the temperature controller board.	
		Power could not be supplied to the drying heater due to a disconnection of the drying station connection harness connecting the temperature controller board and the controller board.	
		Power could not be supplied to the heater due to a CPU1 error of the controller board.	
		The heater voltage selector switch is set incorrectly.	
533	Thermal Fuse (Drying Station 2)	The power-supply voltage selector switch is set incorrectly.	Contact the Sakura Finetek Technical Support representative or local representative.
		Short-circuit failure of the SSR4 on the temperature control board	
		The heater remained on due to a driver error of the controller board.	
		A false detection occurred due to an error of the SSR2 for power-supply voltage detection.	
		The drying station connection harness connecting the temperature controller board and the controller board was disconnected.	
		The instrument malfunctioned due to an over-temperature protection input error of the controller board.	
534	Fan Stop (Drying Station 2)	The heater voltage selector switch is set incorrectly.	Contact the Sakura Finetek Technical Support representative or local representative.
		The power-supply voltage selector switch is set incorrectly.	
		The fan is faulty.	
		An operation signal was not output due to a controller board error.	
		The connection cable connecting the temperature controller board and the controller board was disconnected.	
		The fan could not be driven due to a driver error of the temperature controller board.	
		The connection cable connecting the temperature controller board and the controller board was disconnected.	
		A stopped fan was recognized by CPU1 due to a signal input error of the controller board.	
Normal judgment was disabled due to a CPU1 error of the controller board.			

Troubleshooting

Error number	Name	Cause	Action
630	Sensor Opened (Heating Station 2)	An abnormal value was input to the signal amplification part of the controller board due to a disconnected temperature sensor. The heating station connection harness connecting the temperature controller board and the controller board was disconnected. A disconnected harness was recognized by CPU1 due to a signal amplification error of the controller board. Normal judgment was disabled due to a CPU1 error of the controller board.	Contact the Sakura Finetek Technical Support representative or local representative.
631	Sensor Shorted (Heating Station 1)	An abnormal value was input to the signal amplification part of the controller board due to a shorted temperature sensor. The heating station connection harness connecting the temperature controller board and the controller board shorted. A shorted harness was recognized by CPU1 due to a signal amplification error of the controller board. Normal judgment was disabled due to a CPU1 error of the controller board.	Contact the Sakura Finetek Technical Support representative or local representative.
632	Temperature Low (Heating Station 1)	Power could not be supplied to the heater due to a disconnection or connection failure of the heater. Power could not be supplied to the heater due to a SSR error of the temperature controller board. Power could not be supplied to the heater due to a disconnection of the heating station connection harness connecting the temperature controller board and the controller board. Power could not be supplied to the heater due to a CPU1 error of the controller board. Solution exchange to a cold solution occurred during the operation. The heater voltage selector switch is set incorrectly. The power-supply voltage selector switch is set incorrectly.	Contact the Sakura Finetek Technical Support representative or local representative.
633	Thermal Fuse (Heating Station 1)	The SSR3 on the temperature controller board experienced a short-circuit failure. The heater remained on due to a driver error of the controller board. A false detection occurred due to an error of the SSR1 for power-supply voltage detection. The heating station connection harness connecting the temperature controller board and the controller board was disconnected. The instrument malfunctioned due to an over-temperature protection input error of the controller board. The heater voltage selector switch is set incorrectly. The power-supply voltage selector switch is set incorrectly.	Contact the Sakura Finetek Technical Support representative or local representative.
730	Sensor Opened (Heating Station 2)	An abnormal value was input to the signal amplification part of the controller board due to a disconnected temperature sensor. The heating station connection harness connecting the temperature controller board and the controller board was disconnected. A disconnected harness was recognized by CPU1 due to a signal amplification error of the controller board. Normal judgment was disabled due to a CPU1 error of the controller board.	Contact the Sakura Finetek Technical Support representative or local representative.
731	Sensor Shorted (Heating Station 2)	An abnormal value was input to the signal amplification part of the controller board due to a shorted temperature sensor. The heating station connection harness connecting the temperature controller board and the controller board shorted. A shorted harness was recognized by CPU1 due to a signal amplification error of the controller board. Normal judgment was disabled due to a CPU1 error of the controller board.	Contact the Sakura Finetek Technical Support representative or local representative.
732	Temperature Low (Heating Station 2)	Power could not be supplied to the heater due to a disconnection or connection failure of the heater. Power could not be supplied to the heater due to a SSR error of the temperature controller board. Power could not be supplied to the heater due to a disconnection of the heating station connection harness connecting the temperature controller board and the controller board. Power could not be supplied to the heater due to a CPU1 error of the controller board. Solution exchange to a cold solution occurred during the operation. The heater voltage selector switch is set incorrectly. The power-supply voltage selector switch is set incorrectly.	Contact the Sakura Finetek Technical Support representative or local representative.

Troubleshooting

Error number	Name	Cause	Action
733	Thermal Fuse (Heating Station 2)	<p>The SSR3 on the temperature controller board experienced a short-circuit failure.</p> <p>The heater remained on due to a driver error of the controller board.</p> <p>A false detection occurred due to an error of the SSR1 for power-supply voltage detection.</p> <p>The heating station connection harness connecting the temperature controller board and the controller board was disconnected.</p> <p>The instrument malfunctioned due to an over-temperature protection input error of the controller board.</p> <p>The heater voltage selector switch is set incorrectly.</p> <p>The power-supply voltage selector switch is set incorrectly.</p>	Contact the Sakura Finetek Technical Support representative or local representative.
800	Coverslipper Not Connected	<p>The RS232C cable between the coverslipper and this instrument is not connected correctly.</p> <p>The coverslipper is not powered up.</p> <p>The instrument malfunctioned due to a communication driver error of the controller board.</p> <p>The instrument malfunctioned due to an electrical error of the coverslipper.</p>	<p>Connect the cable correctly.</p> <p>Turn on the coverslipper power.</p> <p>Contact the Sakura Finetek Technical Support representative or local representative.</p>
802	Coverslipper Operation Stop	<p>The cover of this instrument was opened while the start station of the coverslipper was moving into the instrument.</p> <p>The cover deformed and created a gap between the cover open/close sensor and the detection plate while the start station of the coverslipper was moving into the instrument.</p> <p>The instrument malfunctioned due to a cover open/close sensor error while the start station of the coverslipper was moving into the instrument.</p> <p>The instrument malfunctioned due to a controller board input error while the start station of the coverslipper was moving into the instrument.</p> <p>The control panel connection harness connecting the CPU board of the control panel and the controller board shorted while the start station of the coverslipper was moving into the instrument.</p> <p>The instrument malfunctioned due to an input error of the CPU board of the control panel while the start station of the coverslipper was moving into the instrument.</p> <p>The instrument malfunctioned due to a CPU error of the CPU board of the control panel while the start station of the coverslipper was moving into the instrument.</p> <p>An error occurred in the coverslipper.</p>	<p>Close the cover.</p> <p>Contact the Sakura Finetek Technical Support representative or local representative.</p>
900	File System Error (Drive: A)	The CF card inserted to CF drive A is not recognized.	Contact the Sakura Finetek Technical Support representative or local representative.
901	File Not Found (Drive: A)	<p>The operator attempted to export data to a file without inserting a CF card in CF drive A.</p> <p>The operator attempted to export data to a file by inserting a CF card in CF drive A in which the maximum number of files have been registered .</p> <p>The operator attempted to export data to a file by inserting in CF drive A a CF card that was formatted by a file system other than FAT.</p>	<p>Insert the CF card correctly, and then perform the operation.</p> <p>Contact the Sakura Finetek Technical Support representative or local representative.</p>
902	Space Not Available (Drive: A)	The CF card inserted to CF drive A is full.	Contact the Sakura Finetek Technical Support representative or local representative.
903	Write Protect Error (Drive: A)	If data is to be exported to a file, the file to be exported in CF drive A has a write-protected file whose name is exactly same as the file exported to CF drive A.	Contact the Sakura Finetek Technical Support representative or local representative.
905	File System Error (Drive: B)	The control file required for instrument operation is not found in the CF card set in CF drive B.	Contact the Sakura Finetek Technical Support representative or local representative.
906	File Not Found (Drive: B)	<p>The control file required for system operation is corrupted.</p> <p>The CF card inserted to CF drive B is full.</p> <p>The control file required for instrument operation is write-protected.</p>	Contact the Sakura Finetek Technical Support representative or local representative.

Troubleshooting

Instrument Conditions and Recovery Actions

The troubleshooting chart below lists problems that may occur during system operation. Probable causes and recovery actions are described for each problem. Use this information to identify the instrument condition and resolve the problem. If the problem persists after taking the specified action, or if the specific problem you have encountered is not listed in the chart, contact the Sakura Finetek Technical Support representative or local representative.

*** In the event of an unexpected problem for which you cannot find any solution, immediately turn off the power. Then, contact the Sakura Finetek Technical Support representative or local representative.**

Instrument condition	Check	Action
The power is not supplied even when the power switch is turned on.	Power is not supplied.	Check that the power supply condition on the facility side is normal. Check the power cord for abnormality.
The power indicator on the display does not illuminate.	<u>The circuit protector has tripped.</u> <u>The power input part is abnormal.</u> <u>The switching power supply is abnormal.</u> <u>The control panel LED board is faulty.</u>	Contact the Sakura Finetek Technical Support representative or local representative.
When the power switch is turned on, the built-in circuit protector in the power switch or the circuit protector installed in the control box trips.	<u>Mis-match of the trip current of the circuit protector.</u> <u>The power-supply voltage selector switch or heater voltage selector switch is set incorrectly.</u> <u>Short-circuited switching power input part.</u> <u>Short-circuited drying station drive part.</u> <u>Short-circuited heating station drive part.</u>	Contact the Sakura Finetek Technical Support representative or local representative.
When a metal part of the instrument is touched, tingling sensation is felt.	Insulation failure occurred when the enclosure was not grounded, and the leak current flowed through the operator's body.	Contact the Sakura Finetek Technical Support representative or local representative.
When the power is turned on, the screen remains dark.	<u>Power is not supplied to the control panel.</u> <u>The CPU board of the control panel is faulty.</u> <u>The LED backlight cable is faulty.</u> <u>The LED drive circuit unit is faulty.</u>	Contact the Sakura Finetek Technical Support representative or local representative.
The LCD turns on but the screen remains while when the power is turned on.	<u>The LCD FPC2 on the control panel is not connected all the way.</u> <u>The CPU board of the control panel is faulty.</u> <u>The LCD is faulty.</u>	Contact the Sakura Finetek Technical Support representative or local representative.
The program does not load after the power is turned ON.	<u>The program file is corrupted.</u> <u>The CPU board of the control panel is faulty.</u>	Contact the Sakura Finetek Technical Support representative or local representative.
The clock displayed at the top of the screen no longer keeps the correct time.	<u>The battery unit is faulty.</u> <u>The CPU board of the control panel is faulty.</u>	Contact the Sakura Finetek Technical Support representative or local representative.
When an effective button is touched on the control panel screen, the button does not react. Or, the input position is off.	<u>Calibration error of the touch panel detection position.</u> <u>The CPU board of the control panel is faulty.</u> <u>The LCD is faulty.</u>	Contact the Sakura Finetek Technical Support representative or local representative.
The LCD became much darker than before.	<u>Effect of environmental temperature when the power was turned on.</u> <u>The LED drive circuit unit is faulty.</u> <u>The LED backlight is faulty (deteriorated).</u>	After the power is on, wait for a while, and the display should become brighter. Contact the Sakura Finetek Technical Support representative or local representative.
The LCD suddenly flickers or colors become misaligned. This does not include a momentary freezing of the display when the screen is switched during a process.	<u>The LCD FPC2 on the control panel is not connected properly.</u> <u>The CPU board of the control panel is faulty.</u> <u>The LCD is faulty.</u>	Contact the Sakura Finetek Technical Support representative or local representative.
The buzzer at the back of the control panel does not sound.	Program setting. <u>The speaker is faulty.</u> <u>The CPU board of the control panel is faulty.</u>	Check the setting of the key input sound in "System Setup." Contact the Sakura Finetek Technical Support representative or local representative.
The control panel supported by the DP arm of the instrument does not operate smoothly and cannot be moved to an easy-to-operate position.	<u>Foreign matter is stuck between the control panel and the DP arm.</u> <u>The pin (washer) regulating the rotation range of the rotation mechanism broke and got bitten by the rotation part.</u>	Remove the foreign matter. Contact the Sakura Finetek Technical Support representative or local representative.
The control panel supported by the DP arm of the instrument can no longer be fixed at an easy-to-operate position.	<u>The tightening nut or hex socket head bolt of the rotation part is loose.</u> <u>The friction plate or swing mechanism collar used at the rotation part has worn.</u>	Contact the Sakura Finetek Technical Support representative or local representative.

Troubleshooting

Troubleshooting

Instrument	Condition Check	Action
A special staining tray can no longer be recognized.	The detection pin no longer moves.	Contact the Sakura Finetek Technical Support representative or local representative.
	False detection occurred due to a special staining tray detection sensor error.	
	False detection occurred due to a sensor input error of the controller board.	
	False detection occurred due to a CPU1 error of the controller board.	
Detection is still disabled after installing a 10-slide basket load adapter at the start station.	The detection part of the basket adapter deformed and can no longer detect properly.	Contact the Sakura Finetek Technical Support representative or local representative.
	False detection occurred due to a special staining basket sensor error.	
	False detection occurred due to a sensor input error of the controller board.	
	False detection occurred due to a CPU1 error of the controller board.	
The transfer arm still moves after the cover is opened.	The instrument was operated with the stopping of the transfer arm disabled in the service mode.	Contact the Sakura Finetek Technical Support representative or local representative.
	The cover open/close sensor is outputting a detection signal.	
	False detection occurred due to a cover open/close sensor error.	
	False detection occurred due to an input error of the controller board.	
	The control panel connection harness connecting the CPU board of the control panel and the controller board shorted.	
The door open/close sensor does not detect opening/closing of the door.	The door switch plate installed on the right side of the door has deformed.	Contact the Sakura Finetek Technical Support representative or local representative.
	The door open/close sensor suffered mechanical damage.	
	False detection occurred due to an electrical error of the door open/close sensor.	
	False detection occurred due to an input error of the controller board.	
	The control panel connection harness connecting the CPU board of the control panel and the controller board shorted.	
Stored instrument setup data was initialized.	The control file storing the instrument setup data was lost.	Contact the Sakura Finetek Technical Support representative or local representative.
	The control file storing the instrument setup data is corrupted.	
The saved password setting information was initialized.	The control file storing the password setting information was lost.	Contact the Sakura Finetek Technical Support representative or local representative.
	The control file storing the password setting information is corrupted.	
The saved solution names were initialized.	The control file storing the solution name information was lost.	Contact the Sakura Finetek Technical Support representative or local representative.
	The control file storing the solution name information is corrupted.	
The saved solution configuration was initialized.	The control file storing the solution configuration information was lost.	Contact the Sakura Finetek Technical Support representative or local representative.
	The control file storing the solution configuration information is corrupted.	
The saved staining program was initialized.	The control file storing the staining program information was lost.	Contact the Sakura Finetek Technical Support representative or local representative.
	The control file storing the staining program information is corrupted.	
Sakura solutions or kit solutions are not supported.	The definition file (solution, kit data base) provided by the dealer was lost.	Contact the Sakura Finetek Technical Support representative or local representative.
	The definition file (solution, kit data base) provided by the dealer is corrupted.	
The arm cannot be moved manually by hand when the power is off.	Foreign matters are attached to the slide shaft of each axis.	Contact the Sakura Finetek Technical Support representative or local representative.
	The bearing is too tight to move the slide shaft of each axis due to rust, scratches or deformation.	
	Each axis does not move due to an obstacle present in its moving direction.	
A basket cannot be placed in the correct position during operation.	Each axis does not move in the specified direction due to an abnormality of the axis drive wire and its transmission mechanism.	Contact the Sakura Finetek Technical Support representative or local representative.
	The solution reservoir is not installed in the correct position.	Install the solution reservoir in the correct position.
	The basket adapter has deformed.	Contact the Sakura Finetek Technical Support representative or local representative.
	The tray is not installed in the correct position.	Install the tray in the correct position.
	The home detection mechanism of each axis has not been adjusted correctly.	Contact the Sakura Finetek Technical Support representative or local representative.
	A solution reservoir different from the one set in the solution configuration was installed.	Install a solution reservoir matching the solution configuration.
	No solution reservoir is installed.	Install a solution reservoir matching the solution configuration.
	The staining process was aborted or ended, but the basket has not been removed.	Remove the basket.
	The lid of the solution reservoir remains on.	Remove the lid of the solution reservoir.
	The paraffin tray in the drying station is facing the wrong direction.	Install it in the correct direction.

Troubleshooting

Instrument	Condition Check	Action
The basket was placed in the reservoir, but does not come off from the adapter.	The solution reservoir is not installed in the correct position.	Install the solution reservoir in the correct position.
	The basket adapter has deformed.	Contact the Sakura Finetek Technical Support representative or local representative.
When the facility is operating under the conditions within the use environment of the instrument, water is not supplied by the water supply operation.	The home detection mechanism of each axis has not been adjusted correctly.	Open the flow rate adjustment valve.
	The water supply plug on the facility side, or the flow rate adjustment valve in the instrument, is closed.	Contact the Sakura Finetek Technical Support representative or local representative.
	Setting of the water flow time under control is too short.	Clean the strainer.
	The strainer is clogged.	Contact the Sakura Finetek Technical Support representative or local representative.
	The seat of the check valve is stuck (built-in check valve in the pressure relief valve or the check valve before the water supply manifold).	Clean the strainer.
	The water supply solenoid valve is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
The instrument power is off, but water is leaking from the water supply nozzle.	The valve seat of the vacuum breaker is stuck or leaking.	Contact the Sakura Finetek Technical Support representative or local representative.
	The float switch has actuated (error code: 90).	
When the facility was operating under the conditions within the use environment of the instrument, the water level reached by supplying water to the instrument is lower than before.	Contaminants entered the water supply solenoid valve seat.	Contact the Sakura Finetek Technical Support representative or local representative.
	The water supply solenoid valve is damaged.	
When the facility was operating under the conditions within the use environment of the instrument, the water level reached by supplying water to the instrument is more than before.	The flow rate adjustment valve is clogged.	Use the flow rate adjustment valve to adjust the amount of water supplied.
	The strainer is clogged.	Clean the strainer.
	The O-ring of the water supply nozzle is damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
	The wash reservoir is damaged.	
When the facility was operating under the conditions within the use environment of the instrument, the water level reached by supplying water to the instrument is more than before.	The wash reservoir is not installed properly.	
	The open angle of the flow rate adjustment valve was changed.	Use the flow rate adjustment valve to adjust the amount of water supplied.
	The pressure relief valve is damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
Water is leaking from the instrument due to a problem with water supply or drain.	The water supply capacity on the facility side has changed.	Use the flow rate adjustment valve to adjust the amount of water supplied.
	A component of the water supply part is damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
The solution odor is leaked from the instrument when the instrument is operating.	The component connection point at the water supply part is leaking water.	Contact the Sakura Finetek Technical Support representative or local representative.
	The connection of the drain hose is loose or clogged.	
	The sink is leaking water.	
	No fume cartridge is set.	Install a fume cartridge.
	The fume cartridge was set without unpacking it.	Unpack the fume cartridge.
	The fume cartridge is not installed properly.	Install the fume cartridge correctly.
	The fume cartridge has reached its life.	Replace the fume cartridge.
	The exhaust fan stopped.	Contact the Sakura Finetek Technical Support representative or local representative.
	The air inlet port in the system was blocked.	Check the air inlet port for foreign matters.
	Each gasket of the fume control unit deteriorated.	Contact the Sakura Finetek Technical Support representative or local representative.
	The exhaust outlet port is blocked.	Do not block the exhaust outlet port.
	When connecting the exhaust duct, it is not connected correctly.	Contact the Sakura Finetek Technical Support representative or local representative.
	The exhaust duct is too long.	
The exhaust duct was crushed.		
Solution odor leaks from the instrument when the operation is stopped with solution still in the instrument.	The exhaust system on the facility side is not operating.	Check the exhaust system on the facility side.
	The door is open.	Close the door.
	The cover is open	Close the cover.
	There is no lid on the solution reservoir.	Put the lid on the solution reservoir.
The cover does not open smoothly.	The cover was pushed open at an angle.	Contact the Sakura Finetek Technical Support representative or local representative.
	The cover (top plate) is bitten.	Check for foreign matters and remove them, if found.
	The rotary damper and rack are bitten or damaged.	Check for foreign matters and remove them, if found.
	The roller pack is bitten or damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
	The spring stay is damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
The cover does not close smoothly or fully.	The cover closes at an angle or does not close fully.	Contact the Sakura Finetek Technical Support representative or local representative.
	The cover is distorted.	Contact the Sakura Finetek Technical Support representative or local representative.
	The cover (top plate) is bitten.	Check for foreign matters and remove them, if found.
	The rotary damper and rack are bitten or damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
	The roller pack is bitten or damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
	The spring stay is damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
	The cover stay adjustment is inappropriate.	

Troubleshooting

Instrument	Condition Check	Action
The cover has deteriorated.	Dirty or solution is attached.	Clean the cover if dirty or solution is attached.
	Effect of ultraviolet light.	Contact the Sakura Finetek Technical Support representative or local representative.
	The temperature at the installation location has changed.	Use the instrument in an environment free from temperature difference.
The door does not move smoothly or close.	The oil damper is deformed or damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
	The stay is deformed or damaged.	
	The door hinge is deformed or damaged.	
	The door switch plate is deformed or damaged.	
The door would not open. The door would not close.	Entry of foreign matter	Check for foreign matters and remove them, if found.
	The one-touch open catch requires adjustment or is damaged.	Contact the Sakura Finetek Technical Support representative or local representative.
No error is displayed on the screen, but an external alarm is output.	The [Normal Signal] is not transmitted due to an external wiring error.	Contact the Sakura Finetek Technical Support representative or local representative.
	The [Normal Signal] is not output due to an external I/F board relay error.	
	The [Normal Signal] is not output due to an error in the controller board driver circuit.	
	The control panel connection harness is disconnected.	
	The [Normal Signal] is not output due to a CPU board error of the control panel.	
Rust generated.	Corrosive solution was spilled in the sink or instrument.	Clean the solution without delay.
	The instrument was stopped for a long period of time with corrosive and volatile solutions still left in the instrument.	Put the lids on the solution stations and store the solutions outside the instrument.
	The instrument was stopped for a long period of time with water still attached to the sink or tray.	Wipe off any water, etc., attached to the instrument.
Noises generated by each drive axis became louder than before.	Noise from the gear meshing part at the bottom of the Z-axis motor	Contact the Sakura Finetek Technical Support representative or local representative.
	Noise from the gear meshing part at the bottom of the X-axis motor	
	Noise from the gear meshing part at the bottom of the Y-axis motor	
The solution reservoir, basket or basket adapter has deteriorated or damaged.	Dropped.	Exercise caution when using the instrument.
	An attempt was made to install the instrument under unreasonable conditions.	Avoid using the instrument under unreasonable conditions.
	Effect of solution due to the staining process.	Contact the Sakura Finetek Technical Support representative or local representative.
Smoke came from the instrument.	The terminal block of the primary power line became loose and the terminals were overheated and smoke generated from electric wire.	Contact the Sakura Finetek Technical Support representative or local representative.
	Smoke generated due to an abnormal drying station.	
	Smoke generated due to an abnormal heating station.	
	The insulation of the power cable was lost and the leak current continued to flow, causing abnormal heat generation.	
Solution or water was spilled inside the instrument.	Human error	Contact the Sakura Finetek Technical Support representative or local representative.
The bar code cannot be read.	The bar code was read more than 10 seconds after touching the Read button.	Read the bar code within 10 seconds after touching the Read button.
	A bar code reader other than the specified product was used.	Use the specified bar code reader.
	The bar code reader is faulty.	Contact the Sakura Finetek Technical Support representative or local representative.
	Power is not supplied to the bar code reader.	
	The circuit on the instrument side is faulty.	
	The bar code is in poor state (not printed clearly, dirty, scratched, etc.)	Be sure to use an unexpired product.
	The bar code pertains to an expired product.	
	The product code DB file provided by the dealer is damaged.	
	The bar code reader is not set correctly.	Contact the Sakura Finetek Technical Support representative or local representative.



Please visit our website sakura.eu

Sakura Finetek Europe B.V., Flemingweg 10a, 2408 AV Alphen aan den Rijn,
P.O. Box 362, 2400 AJ Alphen aan den Rijn, The Netherlands

Sakura Finetek Japan Co., Ltd.
1-9 Honcho 3-chome Nihombashi Chuo-ku Tokyo, Japan

